**ANA 2000**<sup>®</sup>

High Copper Admixed Alloy  
Non Gamma 2

**ANA 70**

Dispersed Alloy  
Non Gamma 2

CE  
0344

**Duett**<sup>®</sup>

**Nordiska  
Dental**

## Alloy and Mercury acc to ISO 24234

### ANA 2000<sup>®</sup>

High Copper Admixed Alloy  
Non Gamma 2

#### Composition:

Ag 43,1%, Sn 30,8%, Cu 26,1%  
Lathe cut and spherical particles.

### ANA 70

Dispersed Alloy  
Non Gamma 2

#### Composition:

Ag 69,9%, Sn 18,8%, Cu 10,2%, Zn 1,1%  
Blend of spherical and lathe cut particles.

Danger

H330 Fatal if inhaled.

H360 May damage fertility. May damage the unborn child.

H372 Causes damage to organs through prolonged or repeated exposure.

H410 Very toxic to aquatic life with long lasting effects.

P201 Obtain special instructions before use.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P273 Avoid release to the environment.

P284 Wear respiratory protection.

P310 Immediately call a POISON CENTER or doctor/physician.

P501 Dispose of contents/container to an approved waste facility.

Contents Duett,

Mercury 50% CAS No. 7439-97-6

### **Effect of mercury on metals**

"Mercury reacts with and embrittles particular metals and their alloys. Avoid unnecessary contact between mercury and those metals (and their alloys)."

### **Spillages**

"Mercury presents a health hazard if handled incorrectly. Mercury is toxic by vapour inhalation and the effect is cumulative. Spillages of mercury should be removed immediately, including from places which are difficult to access. Use a plastic syringe to draw it up.

Smaller quantities can be covered by sulfur powder and removed. Individual small droplets can be picked up by tin (i.e Sn) foil and removed. Avoid inhalation of the

vapour. During and after a clean-up thoroughly ventilate the area where the spill has occurred. Vacuum cleaners should not be used"

### **Moisture contamination**

"If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance can be affected adversely. If the alloy contains zinc, such contamination can result in an excessive expansion (delayed expansion). Use a dry field whenever it is possible."

### **Waste disposal**

"Waste material and all primary containers that have held mercury shall be disposed of following appropriate management practice."

 **Manufactured by:**  
Wykle Research Inc. E. College Parkway 2222,  
Carson City, NV 89706, USA

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Reusable capsules can be precharged with tablet(s) and sachet(s), as the mercury is not released until the capsule is vibrated. Use tweezers when charging.

Trituration takes place in the reusable capsule. No pestle is needed. After trituration knock the capsule on the desk before opening. After pouring out the ready-mixed amalgam, remove the empty mercury sachet from the capsule.

### MANIPULATION

Insert in small portions and condense well. Overpack the cavity. Remove amalgam with excess of mercury. Carve back for occlusal adaption.

**To avoid weight loss from vibration, store this product on a rigid surface.**

Recommended storage:  25°C

For dental use only. Do not use in patients with known allergies.

The reusable capsules included are intended and appropriate to be used for the included material only.

No special cleaning between the mixings are needed, but all mixed amalgam must be removed before next usage!

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