# Activator universal Plus

Optosil®/Xantopren®

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GB Instructions for use

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SE Bruksanvisning

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NO Bruksanvisning

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LV Lietošanas instrukcija

LT Naudojimo instrukcija

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Manufacturer:  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)  
Made in Romania

Distributed in USA/  
Canada exclusively by:  
Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785

Caution: Federal law restricts this  
device to sale by or on the order  
of a dentist.

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## Technical Information

Silicon based precision condensation curing impression material.

Instructions for use (GB)

Optosil®/ Xantopren®	EN ISO 4823	Colours	Dosage**		Mixing- time	Working- time**	Time in mouth		
			BASE					Activator universal Plus	
			Liquid	Paste				s	min : s
Optosil® Comfort®	Typ 0	yellow	1 spoon (10,7 g/9,5 ml)	6-8 drops (0,14 g/0,13 ml)	6-8 graduations (0,32 g/0,29 ml)	30	1:00	4:00	
Optosil® P Plus	Typ 0	yellow	1 spoon (14,4 g/9,5 ml)	4-6 drops (0,09 g/0,09 ml)	4-6 graduations (0,21 g/0,19 ml)	30	1:00	4:00	
Xantopren® H green	Typ 1	green	6 graduations (11,0 g/8,8 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:00	4:00	
Xantopren® M mucosa	Typ 2	orange	12 graduations (9,0 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® L <sup>+</sup> blue	Typ 3	blue	12 graduations* (8,6 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® VL <sup>+</sup> Plus	Typ 3	blue	12 graduations* (8,5 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	

\* At room temperature of 23°C (73°F), 50% rel. humidity.  
Higher temperatures shorten, lower temperatures prolong these times.  
\* 1 graduation = 1,3 cm (applies to mixing pad available)  
\* 3 Bottom ring on mixing dish is equivalent to 7,6 ml  
\* 4 When using the dispenser: 1 stroke (4,5 g / 4,0 ml) + 6 drops (0,14 g / 0,13 ml)  
The stated dosage should be observed.

Typ 0 = putty consistency  
Typ 1 = heavy bodied consistency  
Typ 2 = medium bodied consistency  
Typ 3 = light bodied consistency

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## User Advice

The Optosil®/Xantopren® assortment is a complete system. Components can be individually combined for all impression techniques. For best results usage with other silicon materials and Activators is not recommended. Visually inspect for damage before first use. Damaged products must not be used. Close carefully after use.

Pouring the model:

Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days.

Recommended model material:

All commercially available modeling stones can be used.  
Our suggestion:  
Class III Gips, MOLDANO®/MOLDADUR®  
Class IV Gips, MOLDASTONE®/MOLDASYN®  
Class V Gips, MOLDASTONE®/MOLDADUR®  
Class IV Gips, MOLDASTONE®/MOLDASYN®  
Class IV Gips, MOLDASTONE®/MOLDADUR®  
Impressions may be copper or silver plated in the usual way.  
Impressions should generally be treated by immersion for 10 minutes\* in 5,25% Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the product manufacturer.  
\* Gelson Luis Adabo, DDS, PhD, Elaine Zananotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD  
Effect of disinfectant agents on dimensional stability of elastic impression materials. J Prosthet Dent, 81, 621-4, 1999

Adhesion:

We always recommend the use of our Universal Adhesive.  
Do not store above 25°C (77°F).  
Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact.  
Wear protective gloves/protective clothing/eye protection/face protection (EN 374, cat III, e.g. Dermatrill from KCL, Germany). Do not swallow and ingest. If health problems arise after swallowing impression material, seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Do not use in pregnant or nursing patients or in children under 6 years of age. For use by dental personnel only.

Storage conditions:

Warning and precaution for Activator universal Plus:

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