

**Indications for use**

Tray adhesive for use with Silicones with standard and custom trays.

**Colour:** blue

**Application**

- Apply the Cavex Adhesive in a thin even layer to the impression tray. Please ensure that all surfaces are clean and dry before application of the adhesive.
- Allow to dry for 2 minutes after application before loading impression material into the tray.

**Remarks**

The drying process can be reduced by using an air blower.  
Cavex Adhesive can be removed with ethyl alcohol.

**Safety Precautions**

- Avoid spillages on clothing as removal of adhesive may be difficult.
- Avoid contact with skin and eyes – in the event of contact with skin or eyes wash immediately with clear water.
- Keep away from flames – do not inhale the fumes.
- Replace cap on bottle and tighten securely after use.

**Storage**

Do not store above 25°C (77°F).

**Shelf life:** see information on package.

**Warnings**

Highly flammable.

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

11.02.2020