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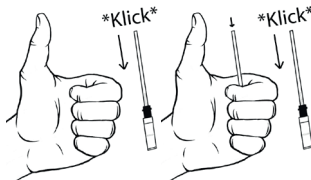
## Contents:

- Single Use Brush
- Quantity 0,15 ml

## General description and purpose of the medical device

Arti-BruX® Quick is a red indicator paint which is applied with a brush (single use product) to the occlusal surfaces of restorative and prosthetic appliances as well as hard splints. After drying, Arti-BruX® Quick forms a thin and even lacquer coating. The lacquer coating is meant for short term use (up to max. 24 hours). It is saliva resistant and is abraded by tooth contacts (occlusal surfaces). Arti-BruX® Quick can be used as an Occlusogram that shows contacts and movement traces of teeth. Contact surfaces as well as traces of movement can be clearly identified by means of the removed lacquer. **For professional use only.**

### Indication

Arti-BruX® Quick BK 88 is a red liquid contact color:

- To control dynamic occlusion on restorative and prosthetic restorations as well as on bite splints
- For short-term control on occlusion (max. 24 h) – Occlusogram
- For visualization of enamel fissures as well as margin fissures of fillings and crowns

### Contraindication

- Arti-BruX® Quick consists mainly of concentrated alcohol (ethanol 96%). Arti-BruX® Quick should not be applied directly invasively to patients who are addicted to alcohol, pregnant women or children. Also contact with injured mucosa should be prevented.
- Arti-BruX® Quick must not be used in case of iodine allergy.
- Arti-BruX® Quick must not be applied to open dentin or prepared teeth.

### Side Effects

- When applied to insufficiently polymerized acrylics or vacuum splints, made of different thermoplastics, discoloration of the plastic may occur.
- For surfaces that are not properly polished or sand blasted, the lacquer can penetrate into fine micro cracks of the plastic or ceramic surfaces and cause discoloration.
- When Arti-BruX® Quick is applied, the color can penetrate into enamel cracks or margins of fillings and cause temporary or even permanent discoloration.
- After longer use of Arti-BruX® (up to max. 24 hours) discoloration of the saliva may also occur; the lacquer consists of a natural resin and red food coloring. After use, the mouth should be rinsed with water.

### Use

- Clean and dry the surfaces to be tested (no saliva should be on the occlusal surfaces);
- Apply the lacquer with the brush to the occlusal surfaces that are to be tested.
- To check the extent of bruxism, especially sleep bruxism, the patient should insert the splint shortly before going to bed.

### Effect and diagnosis

- Check the surfaces to be tested after the observation period
- The coating is rubbed off by tooth contacts (occlusal surfaces). These contact patterns are used to check the occlusal contacts in static and dynamic
- A uniform wear on night-guards for example is an indication that the individual track is evenly loaded. Strong one-sided wear suggests parafunctions (e.g. bruxism) or an incorrectly adjusted splint.
- The same applies to prosthetic restorations.

### Cleaning

After successful inspection remove the coating from the surface. Appropriate tools are:

- Tooth brush and appropriate tooth paste
- Alcohol bath or soaked wadding with alcohol. The lacquer dissolves well with concentrated alcohol (ethanol 96%, un-denatured).
- Splints can be polished or cleaned with standard dental field polishing products
- Prosthetic restorations can be cleaned with a toothbrush or usual practice equipment for the professional tooth cleansing.

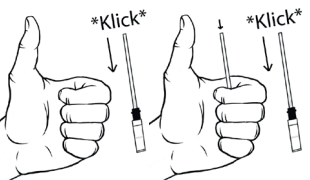
### Disposal

Arti-BruX® Quick consists of a red food color and a natural resin, which is also approved for food and pharmaceutical applications. Arti-BruX® Quick can be disposed of with the usual practical waste.

### Reporting of incidents

Serious occurring incidents in connection with the product must be reported to the manufacturer and the competent authority of the Member State.

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