ENGLISH

Cavitron® Select™ SPS™ Ultrasonic Scaler

Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System

Directions For Use

Please read carefully and completely before operating unit.

Introduction

Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System

DENTSPLY Professional is an ISO13485 certified company. The Cavitron® Select™ System is classified by Underwriters Laboratories Inc. with respect to electric shock, fire, mechanical hazards only in accordance UL 2601-1 1 and Can/CSA C22.2 NO. 601.1.

The Scaling System operates by converting ordinary AC house current into high frequency current. The ultrasonic system consists of two parts: an insert, and the SPS™ (Sustained Performance System) electronic system. The system incorporates two closed loops. One loop provides automatic tuning (operating frequency is adjusted to be at resonance for each insert), the second loop automatically controls the tip stroke over different working conditions. The scaler produces 30,000 microscopically small strokes per second at the insert's working tip. This combined with acoustic effects of the coolant water, produces a synergistic action that literally "powers away" the heaviest calculus deposits while providing exceptional operator and patient comfort.

The Pump/Reservoir accessory for the Cavitron® Select™ SPS™ is an optional lavage dispenser. Footswitch plug, handpiece plug, water tube and power supply plug are removed from the Scaler before mating with the Pump/Reservoir. The Scaler is snapped together with the Pump/Reservoir. The footswitch plug, handpiece plug and power supply plug are inserted into mating receptacles at the rear of the Pump/Reservoir. Ultrasonic signal, footswitch and DC power connections between the units are made via an edgcard connector at the rear of the Scaler and the front of the Pump module. The Reservoir has a capacity of 450 ml. Fluid from the

Reservoir is propelled from the Reservoir Bottle with 15–19 psig compressed air generated by an internal diaphragm pump. ON/OFF of the lavage is controlled through an internal solenoid valve. Fluid to the Handpiece passes directly from the Pump/Reservoir to the Handpiece. No fluid passes through the Cavitron® Select™ SPS™ when used with this accessory. Flow through any insert in the Handpiece will be greater than 40 cc/m water when adjusted to maximum.

Technical Support

For technical support and repair assistance in the U.S., call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY representative.

Supplies & Replacement Parts

To order supplies or replacement parts in the U.S., contact your local DENTSPLY Distributor or call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY representative.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a license dental professional.

For dental use only.

Table of Contents

Section Number	Section Title Description of Contents	Page Number
1	Indications for use.....	3
2	Contraindications and Warnings	3
	2.1 Contraindications	
	2.2 Warnings	
3	Precautions.....	3
	3.1 Precautions for All Systems	
	3.2 Precautions for Ultrasonic Prophylaxis Procedures	
	3.3 Precautions for Cavitron® Select™ Reservoir System	
4	Infection Control.....	4
	4.1 General Infection Control Recommendations	
	4.2 Water Supply Recommendations	
5	Installation Instructions for Cavitron® Select™ SPS™ Ultrasonic Scaler.....	4–5
	5.1 General Information	
	5.2 Water Line Requirements	
	5.3 Electrical Requirements	
	5.4 Unpacking the System	
	5.5 Power Cord/Power Connection	
	5.6 Foot Control Cable Assembly Connection	
	5.7 Water Supply Line Connection	
	5.8 Lavage Control Detachable Handpiece Cable Assembly Connection	
6	Installation Instructions for Cavitron® Select™ SPS™ Scaler with Cavitron® Select™ Reservoir System6–7	
	6.1 General Information	
	6.2 Electrical Requirements	
	6.3 Unpacking the System	
	6.4 Assembling the System	
	6.5 Power Cord/Power Connection	
	6.6 Foot Control Cable Assembly Connection	
	6.7 Lavage Control Detachable Handpiece Cable Assembly Connection	
7	Cavitron® Select™ SPS™ Ultrasonic Scaler Description	8–10
	7.1 System Controls for Cavitron® SPS™ Select™ Ultrasonic Scaler	
	7.2 System Controls for Cavitron® Select™ SPS™ Scaler with Cavitron® Select™ Reservoir System	
	7.3 Handpiece	
	7.4 Cavitron® 30K™ Ultrasonic Inserts	
	7.5 Foot Control Information & Operation	
8	Accessories	11
	8.1 Accessories List	
	8.2 Medicaments	
9	Techniques for Use.....	11
	9.1 Patient Positioning	
	9.2 Performing Ultrasonic Scaling Procedures	
	9.3 Patient Comfort Considerations	
10	System Care	12–13
	10.1 Daily Maintenance — Start-Up Procedures, Between Patients, Shut-Down Procedures	
	10.2 Weekly Maintenance	
	10.3 Water Line Filter Maintenance	
	10.4 Pump Lavage Filter Maintenance	
11	Troubleshooting.....	14
	11.1 Troubleshooting Guide	
	11.2 Technical Support and Repairs	
12	Specifications	15
13	Classifications	15
14	Disposal of Unit.....	15

Section 1: Indications for Use

The Dentsply® Cavitron® SPS Scaler System is designed for use in general prophylaxis and periodontal treatments and other areas of operative dentistry. It is used to debride light to heavy calculus and plaque deposits from tooth and root surfaces.

Section 2: Contraindications and Warnings

2.1 Contraindications

- Ultrasonic Systems should not be used for restorative dental procedures involving the condensation of amalgam.

2.2 Warnings

- The use of High Volume Saliva Evacuation to reduce the quantity of aerosols released during treatment is highly recommended.
- Prior to beginning treatment, patients should rinse with an antimicrobial such as Chlorhexidine Gluconate 0.12%. Rinsing with an antimicrobial reduces the chance of infection and reduces the number of microorganisms released in the form of aerosols during treatment.
- It is the responsibility of the Dental Healthcare Professional to determine the appropriate uses of this product and to understand:
 - the health of each patient,
 - the dental procedures being undertaken,
 - and applicable industry and governmental agency recommendations for infection control in dental healthcare settings,
 - requirements, and regulations for safe practice of dentistry; and
 - these Directions for Use in their entirety, including Section 4 Precautions, Section 6 Infection Control, and Section 10 System Care.
- Where asepsis is required or deemed appropriate in the best professional judgment of the Dental Healthcare Professional, this product should not be used, unless used in combination with a sterile lavage kit (P/N 81340).
- Handle Cavitron insert with care. Improper handling of insert, specifically the insert tip, may result in injury and/or cross contamination.
- Failure to follow properly validated sterilization processes and approved aseptic techniques for Cavitron inserts may result in cross contamination.
- Persons fitted with cardiac pacemakers, defibrillators and other active implanted medical devices, have been cautioned that some types of electronic equipment might interfere with the operation of the device. Although no instance of interference has ever been reported to DENTSPLY, we recommend that the handpiece and cables be kept 6 to 9 inches (15 to 23 cm) away from any device and their leads during use. There are a variety of pacemakers and other medically implanted devices on the market. Clinicians should contact the device manufacturer or the patient's physician for specific recommendations. This unit complies with IEC 60601 Medical Device Standards.

- Insufficient water flow could result in elevated water and tip temperature. When operated at the input water temperature specified in the Water Line Requirements Section 7.1 and with sufficient water flow, the water and tip temperature should not exceed 50o C (122o F). Failure to follow recommendations for environmental operating conditions, including input water temperature, could result in injury to patients or users. If temperature is elevated, increase water flow. If temperature remains elevated, discontinue use.
- During boil-water advisories, this product should not be operated as an open water system (e.g. connected to a public water system). A Dental Healthcare Professional should disconnect the system from the central water source. The Cavitron DualSelect system can be attached to this unit and operated as a closed system until the advisory is cancelled. When the advisory is cancelled, flush all incoming waterlines from the public water system (e.g. faucets, waterlines and dental equipment) in accordance with the manufacturer's instructions for a minimum of 5 minutes.

Section 3: Precautions

3.1 Systems Precautions

- Do not place the System on or next to a radiator or other heat source. Excessive heat may damage the System's electronics. Place the System where air is free to circulate on all sides and beneath it.
- The System is portable, but must be handled with care when moving.
- Equipment flushing and dental water supply system maintenance are strongly recommended. See Section 5.2 Water Supply Recommendations.
- Close the water shut-off valve to the System every night before leaving the office.
- When used as an open water system, use of an in-line water filter is recommended.
- Always ensure that the electrical connections on the handpiece cable and the Steri-Mate® handpiece are clean and dry before assembling them for use.
- Never operate scaler without fluid flowing through handpiece.

WARNING: No modification of this equipment is allowed.

3.2 Procedural Precautions

- The Cavitron Select SPS unit works with Cavitron inserts as a system, and was designed and tested to deliver maximum performance for all currently available Cavitron brand ultrasonic inserts. Companies that manufacture, repair or modify inserts carry the sole responsibility for proving the efficacy and performance of their products when used as a part of this system. Users are cautioned to understand the operating limits of their insert before using in a clinical setting.
- Like a toothbrush, ultrasonic inserts "wear out" with use. Inserts with just 2 mm of wear lose about 50% of their scaling efficiency. In general, it is recommended that ultrasonic inserts be discarded and replaced after 2 mm of

wear to maintain optimal efficiency and avoid breakage.

- If excessive wear is noted, or the insert has been bent, reshaped or otherwise damaged, discard the insert immediately.
- Ultrasonic insert tips that have been bent, damaged, or reshaped are susceptible to in-use breakage and should be discarded and replaced immediately.
- Retract the lips, cheeks and tongue to prevent contact with the insert tip whenever it is placed in the patient's mouth.
- Water should be used for all scaling procedures. Water flow is adjustable from less than 10ml to greater than 60ml per minute.
- The DENTSPLY power supply shipped with this unit is approved for use with devices and must be used to operate this system.

3.3 Precautions for Cavitron® Select™ Reservoir System

- Do not place the System on or next to a radiator or other heat source. Excessive heat may damage the System's electronics. Place the System where air is free to circulate on all sides and beneath it.
- The System is portable, but must be handled with care when moving.
- Equipment flushing and dental water supply system maintenance are strongly recommended. See Section 10: System Care.
- Pour liquid into the cap opening of the reservoir bottle. To prevent potential leaking at the cap, do not overfill bottle with liquid into the neck of the bottle. Ensure cap is tightly sealed.
- Turn unit off before refilling reservoir bottle to prevent potential leaking of fluid. Turn cap a 1/4 revolution to release air pressure and then remove the cap completely.

Section 4: Adverse Reactions

None known.

Section 5: Infection Control

5.1 General Infection Control Recommendations

- For operator and patient safety, carefully practice the infection control procedures detailed in the Infection Control Information Booklet accompanying your System. Additional booklets can be obtained by calling Customer Service at 1-800-989-8826, Monday through Friday, 8:00 A.M. to 5:00 P.M. (Eastern Time). For areas outside the U.S., contact your local DENTSPLY Professional representative.
- As with high speed handpieces and other dental devices, the combination of water and ultrasonic vibration from the Cavitron Select SPS Ultrasonic Scaler will create aerosols. Following the procedural guidelines in Section 9 of this manual can effectively control and minimize aerosol dispersion.

5.2 Water Supply Recommendations

- It is highly recommended that all dental water supply systems conform to applicable CDC (Centers for Disease Control and Prevention) and ADA (American Dental Association) standards, and that all recommendations be followed in terms of flushing, chemical flushing, and general infection control procedures. See Section 6.1 and 10.
- As a medical device, this product must be installed in accordance with applicable local, regional, and national regulations, including guidelines for water quality (e.g. drinking water). As an open water system, such regulation may require this device to be connected to a centralized water control device. The Cavitron® Select Reservoir System may be installed to allow this unit to operate as a closed water system.

Section 6: Installation Instructions for Cavitron® Select™ SPS™ Ultrasonic Scaler

6.1 Water Line Requirements

- A water supply line with user-replaceable filter is supplied with your system. See Section 10 System Care for replacement instructions.
- Incoming water supply line pressure to the system must be 20 psi (138 kPa) to 40 psi (275 kPa). If your dental water system's supply line pressure is above 40 psi, install a water pressure regulator on the water supply line to your Cavitron Plus Ultrasonic Scaler.
- A manual shut-off valve on the dental water system supply line should be used so that the water can be completely shut-off when the office is unoccupied.
- In addition to the water filter supplied, it is recommended that a filter in the dental water system supply line be installed so that any particulates in the water supply will be trapped before reaching the Cavitron system.
- After the above installations are completed on the dental water supply system, the dental office water line should be thoroughly flushed prior to connection to the Cavitron system.
- Incoming water temperature to the Cavitron System should not exceed 25°C (77°F). If needed a device should be installed to maintain a temperature within this specification, or a Cavitron DualSelect Dispensing System attached to allow this system to be operated as a closed water system.

6.2 Electrical Requirements

- Incoming power to the system must be 100 volts AC to 240 volts AC, single phase 50/60 Hz capable of supplying 1.0 amps.
- The system power should be supplied through the AC power cord provided with your system.

WARNING: To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

Refer to Section 12: Specifications.


6.3 Unpacking the System

Carefully unpack your Cavitron® Select™ SPS™ Scaler and verify that all components and accessories are included:

1. Cavitron® Select™ SPS™ Scaler.
2. Detachable AC Power Cord set.
3. Steri-Mate Detachable Sterilizable Handpiece.
4. Steri-Mate Flow Control Detachable Handpiece Cable Assembly with Swivel.
5. Dual Position Foot Control Assembly.
6. Hose / Filter / Quick Disconnect Assembly.
7. Additional Water Line Filter.
8. Additional Pump Lavage Filters.
9. Medically Approved Power Supply.
10. Literature Packet.
11. Cavitron Ultrasonic Insert (quantity optional)


6.4 Power Cord/Power Connection

- Verify the Power Control knob is set to the OFF position before proceeding.
- Plug the detachable AC cord into the back of the Medically Approved Power Supply.
- Plug the power supply outlet into the back of the System.
- Plug the 3 prong grounded plug into an approved outlet.
- The Cavitron System should not be positioned such that access to the power input and AC power cords are limited.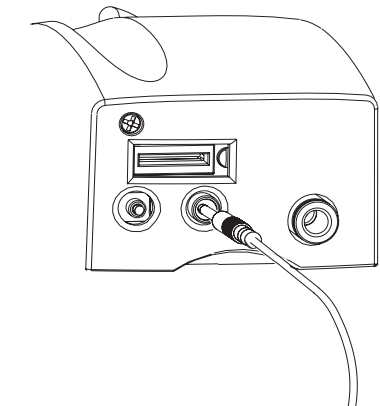

6.5 Foot Control Cable Assembly Connection

Align the Foot Control plug with the receptacle on the back of the System and push it in until firmly seated.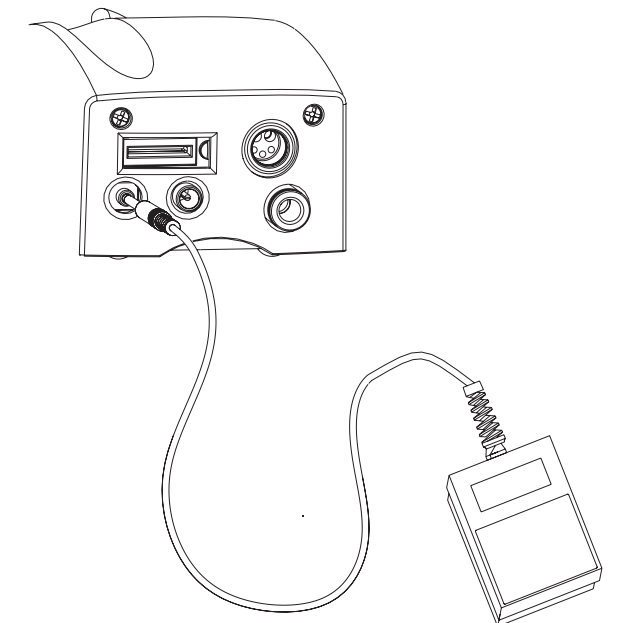


6.6 Water Supply Line Connection

- Push the blue hose into the orange water receptacle until the hose cannot be pushed in any further.
- Connect the free end of the system's water supply line to the office water supply line.
- Inspect all connections to make certain there are no leaks.
- To remove water line from Cavitron® Select™ SPS™ Scaler, turn off or disconnect water supply line **FIRST**, then push on outer orange ring of the outlet and gently pull out water hose.

6.7 Lavage Control Detachable Handpiece Cable Assembly Connection

- Plug the Handpiece outlet into the receptacle on the back of the System.
- Holding the Handpiece Cable Assembly seat male connector into appropriate female connector.
- Once seated, snap male connector into female connector by pushing on the end of the male connector.
- Rotate Lavage Control on the handpiece to middle setting.
- Turn on the on/off switch (power adjustment).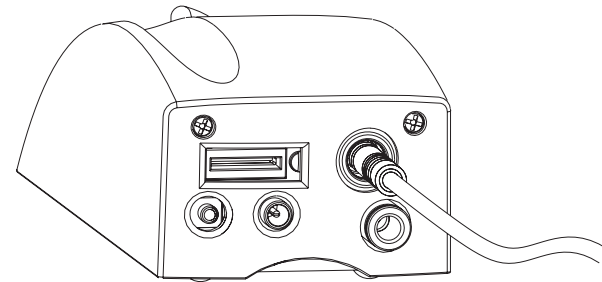

Section 7: Installation Instructions for Cavitron® Select™ SPS™ Scaler with Cavitron® Select™ Reservoir System

7.1 Water Line Requirements

See Section 6.1.

7.2 Electrical Requirements

See Section 6.2.


7.3 Unpacking the System

Carefully unpack your Cavitron® Select™ SPS™ Scaler and verify that all components and accessories are included:


1. Cavitron® Select™ SPS™ Scaler.
2. Cavitron® Select™ Reservoir Pump.
3. Cavitron® Select™ Reservoir Bottle.
4. Detachable AC Power Cord set.
5. Steri-Mate Detachable Sterilizable Handpiece.
6. Steri-Mate Flow Control Detachable Handpiece Cable Assembly with Swivel.
7. Dual Position Foot Control Assembly.
8. Hose / Filter / Quick Disconnect Assembly.
9. Additional Water Line Filter.
10. Additional Pump Lavage Filters.
11. Medically Approved Power Supply.
12. Literature Packet.
13. Cavitron Ultrasonic Insert (quantity optional).

7.4 Assembling the System


- Remove protective sticker from edge card connector on the scaler.
- Rinse Cavitron® Select™ Reservoir Bottle before using.
- Insert Cavitron® Select™ Reservoir Bottle into Cavitron® Select™ Reservoir Pump and rotate clockwise until bottle locks into place.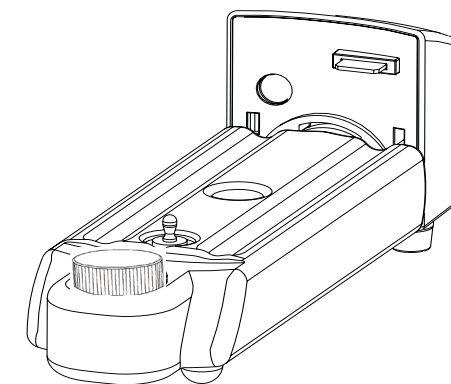


- Connect Cavitron® Select™ SPS™ Scaler to Cavitron® Select™ Reservoir System. Insert male edge card connector on Cavitron® Select™ Reservoir Pump into Cavitron® Select™ SPS™ Scaler opening on the back of the Cavitron® Select™ SPS™ Scaler. Press down on the top of the Cavitron® Select™ SPS™ Scaler (directly above power setting knob) until the black striker knob located on the Reservoir Bottle locks into place.

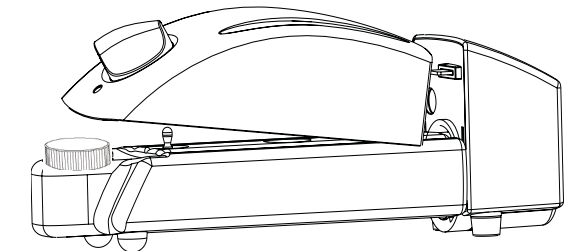- To detach Cavitron® Select™ SPS™ Scaler from Cavitron® Select™ Reservoir Pump and Bottle, lift up on the front of the Cavitron® Select™ SPS™ Scaler (below power setting knob and Power/Boost light). After the striker knob is unsnapped, pull the Cavitron® Select™ SPS™ Scaler away from the Cavitron® Select™ Reservoir Pump.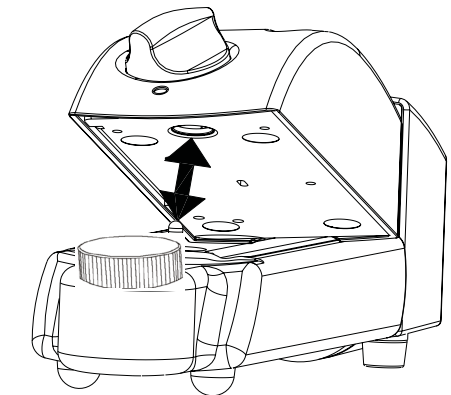


NOTE: Always disengage striker knob before pulling the Cavitron® Select™ SPS™ Scaler away from the Cavitron® Select™ Reservoir Pump.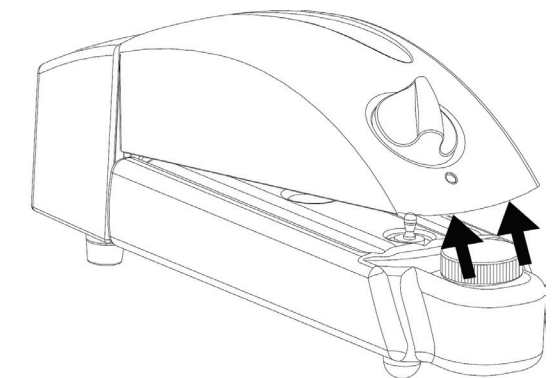

Section 7: Installation Instructions for Cavitron® Select Scaler with Cavitron® Select™ SPS™ Reservoir System cont'd.


7.5 Power Cord/Power Connection

- Verify the Power Control knob is set to the OFF position before proceeding.
- Plug the detachable AC cord into the back of the Medically Approved Power Supply.
- Plug the power supply outlet into the back of the Cavitron® Select™ Reservoir Pump.
- The system should not be positioned such that access to the power input and AC power cord are limited.
- The Cavitron System should not be positioned such that access to the power input and AC power cords are limited.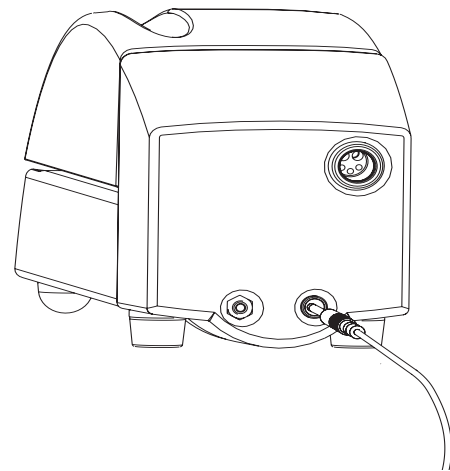


7.7 Lavage Control Detachable Handpiece Cable Assembly Connection

- Plug the Handpiece outlet into the receptacle on the back of the Cavitron® Select™ Reservoir Pump.
- Holding the Handpiece Cable Assembly seat male connector into appropriate female connector.
- Once seated, snap male connector into female connector by pushing on the end of the male connector.
- Rotate Lavage Control on the handpiece to middle setting. Turn on the on/off switch (power adjustment). This energizes the system and the ultrasonic unit is ready for operation.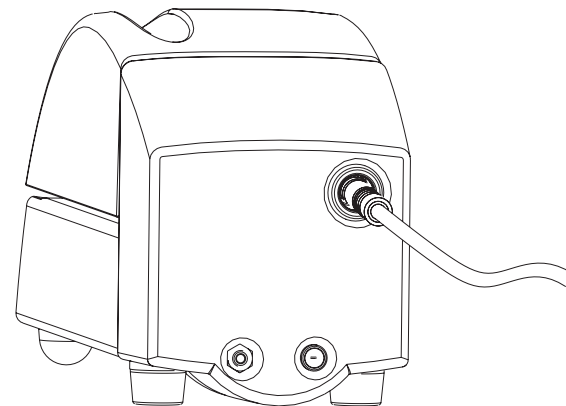


7.6 Foot Control Cable Assembly Connection

Align the Foot Control plug with the receptacle on the back of the Cavitron® Select™ Reservoir Pump and push it in until firmly seated.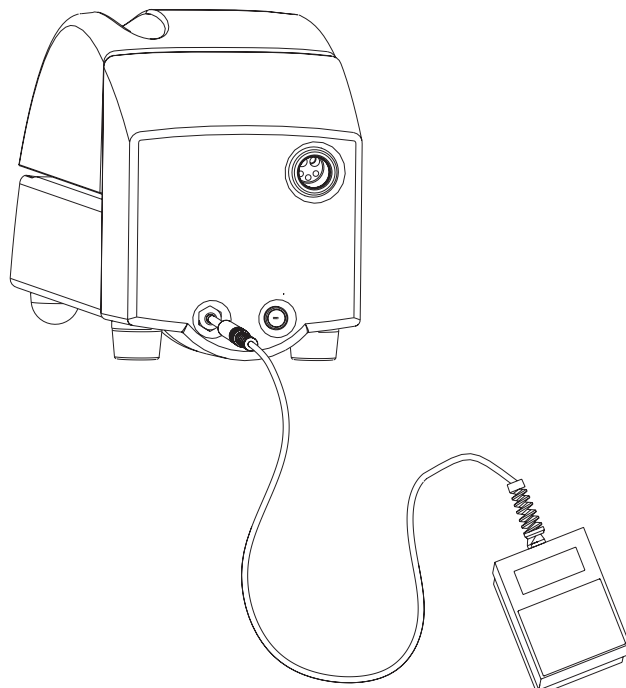


Section 8: Cavitron® Select™ SPS™ Ultrasonic Scaler Description

8.1 System Controls: Cavitron® Select™ SPS™ Ultrasonic Scaler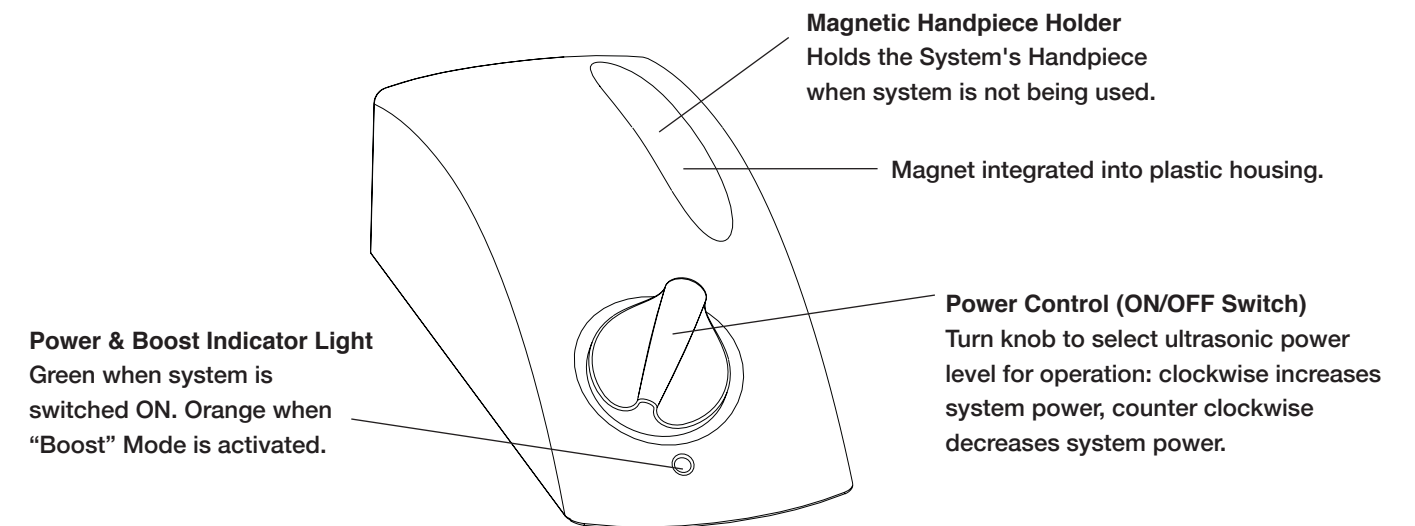



8.2 System Controls: Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System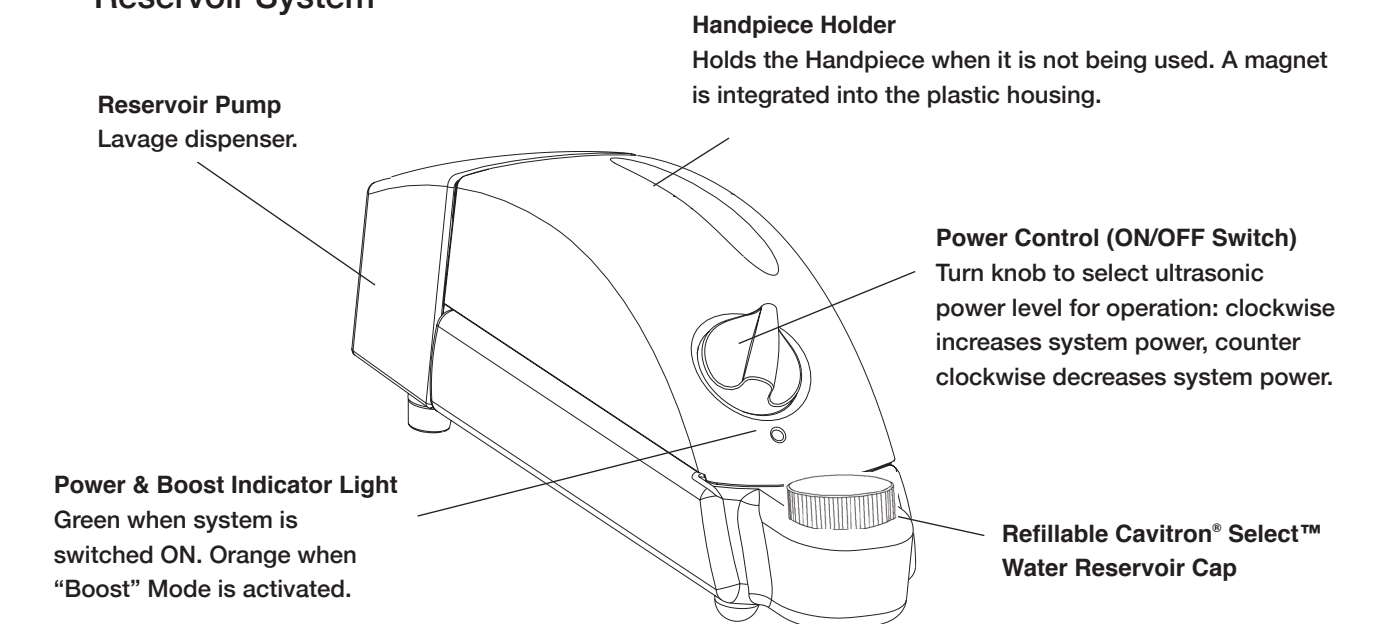

8.3 Handpiece

The handpiece accepts all Cavitron® 30K™ Ultrasonic Inserts.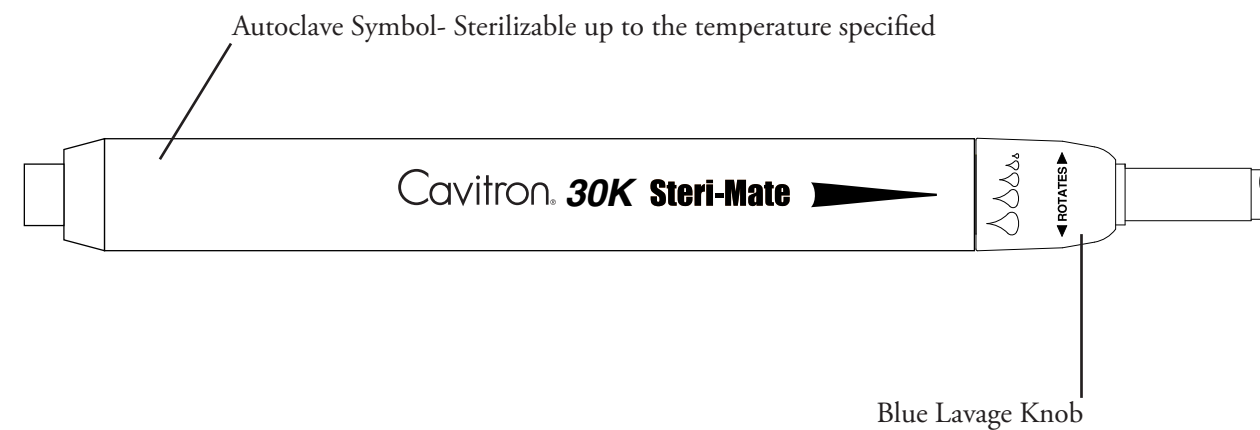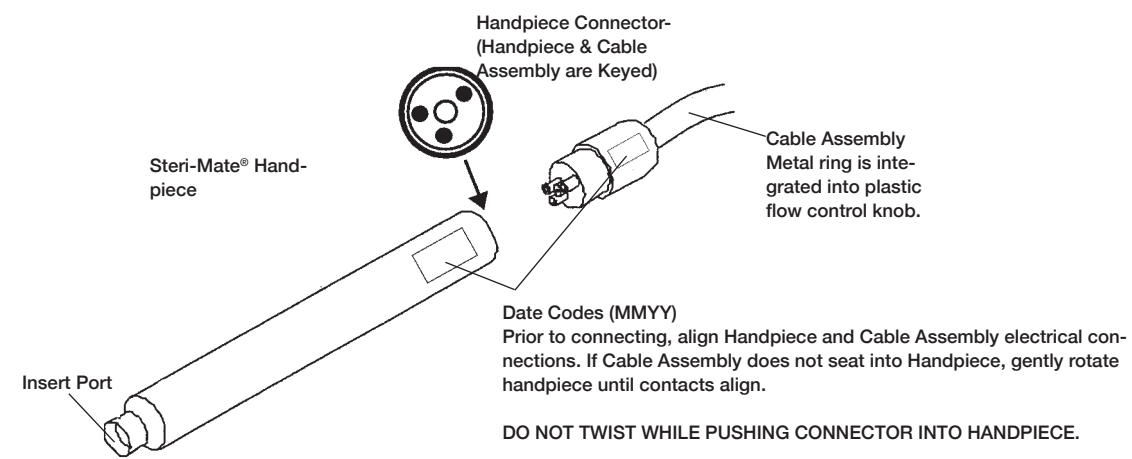


Lavage Control Adjustment

Holding open end of handpiece upright, lavage can be increased by holding the handpiece and gently twisting the blue lavage adjustment knob clockwise.

8.4 Cavitron® 30K™ Ultrasonic Inserts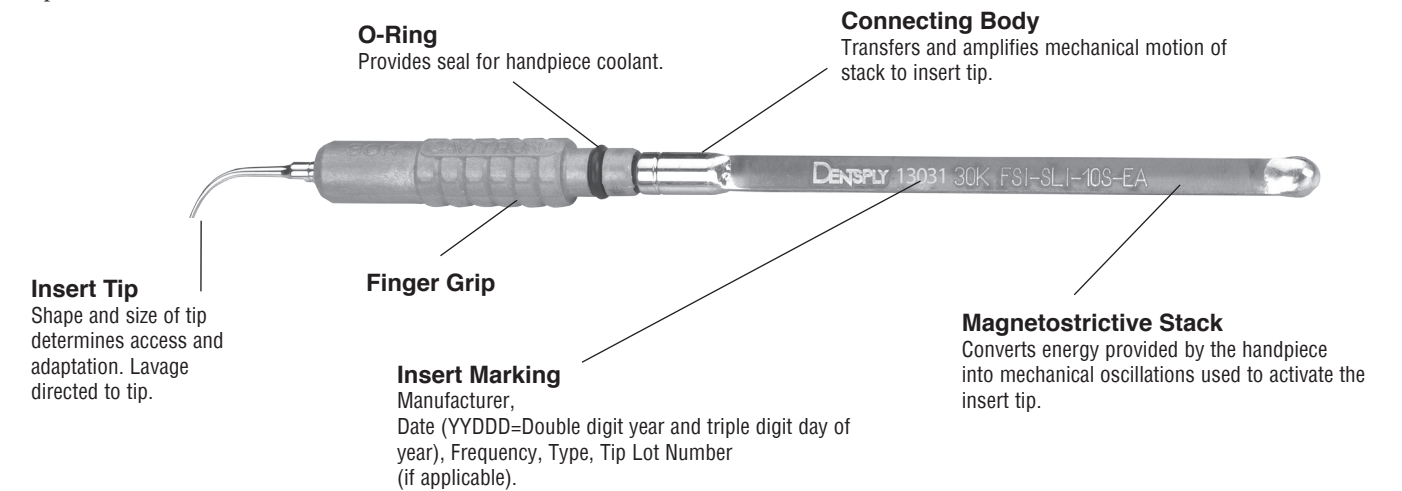


The many styles of Cavitron® 30K™ Ultrasonic Inserts are easily interchangeable for various procedures and applications. See enclosed booklet for specific information.

Hold the handpiece in an upright position. Activate the Foot Control to bleed any air bubbles that might be trapped inside the handpiece. Lubricate the rubber O-ring on the insert with water before placing it into the handpiece. Fully seat insert with a gentle push-twist motion. DO NOT FORCE.

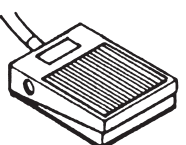8.5 Foot Control Information & Operation

The Foot Control is a two position momentary switch, which activates both ultrasonic energy and Lavage Flow at the insert tip.

<p>Foot Control released...</p> 	<p>Foot Control depressed half way... (1st position)</p> 	<p>Foot Control fully depressed... (2nd position)</p> 
<p>Both ultrasonic activation and irrigating flow stop.</p> 	<p>The ultrasonic insert tip is activated and irrigating water flows.</p> 	<p>This activates the "Boost" Mode.</p> 

Section 9: Accessories

9.1 Accessories List

1. Cavitron[®] Select™ SPS™ Lavage Control Detachable Handpiece Cable Assembly with Swivel
2. Steri-Mate Detachable Sterilizable Handpiece
3. Medically Approved Power Supply
4. AC Power Cord Set
5. Cavitron[®] Ultrasonic Inserts
6. Dual Position Foot Control Assembly
7. Cavitron[®] Select™ Reservoir Bottle
8. Cavitron[®] Select™ Pump
9. Steri-Mate Grip

*For detailed information, contact your local DENTSPLY Representative or authorized DENTSPLY Distributor.

9.2 User Replaceable Part Kits

1. Cavitron[®] Insert Replacement O-ring Kits, 12/packs
Part Number 62351 (Black) for plastic and soft grips
Part Number 62625 (Green) for metal grips
2. Handpiece Cable O-ring, Part Number 79397
3. Lavage (Water) Filter, 10/pack, Part Number 90158

*For detailed information, contact your local DENTSPLY Representative or authorized DENTSPLY Distributor.

9.3 Medicaments

The following medicaments are compatible with the Cavitron[®] Select™ SPS™ when used with the Cavitron[®] Select™ Reservoir Pump:

- Chlorhexidine Gluconate 0.12% (Oris™ CHX Antimicrobial Irrigating Solution)
- Cetylpyridinium Chloride (Cepacol[®])
- Essential Oils (Listerine)
- Hydrogen Peroxide, 3% USP
- Povidone Iodine, 10% Solution
- Saline Solution
- Sanguinaria Extract (Viadent[®])
- Sterile Water, USP

(For irrigating and System maintenance purposes).

Section 10: Techniques for Use

10.1 Patient Positioning

- For optimal access to both the upper and lower arches, the backrest of the chair should be adjusted as for other dental procedures. This assures patient comfort and clinician visibility.

- Have the patient turn their head to the right or left. Also position their chin up or down depending upon the quadrant and surface being treated. Evacuate irrigant using either a saliva ejector or High Volume Evacuator (HVE).

10.2 Performing Ultrasonic Scaling Procedures

NOTE: Refer to the Infection Control Information card supplied with your System and Section 11 of this manual for general procedures to be followed at the beginning of each day and between patients.

- Follow precautions listed in the general and ultrasonic sections of 3.2 Procedural Precautions
- The edges of Cavitron[®] Ultrasonic Inserts are intentionally rounded so there is little danger of tissue laceration with proper ultrasonic scaling technique. Whenever the insert tip is placed in the patient's mouth, the lips, cheek and tongue should be retracted to prevent accidental prolonged contact with the activated tip.
- Turn power level control to select ultrasonic power level for operation. Clockwise increase the system power. Power level will increase throughout the full range of the control. Hold the handpiece over a sink or drain. Depress foot pedal to first position to activate the system. Check water spray to verify fluid is reaching the working end of the insert tips. Adjust the water lavage control under the water (lavage) flows with a rapid drip or small spray. Higher water flow setting provide cooler irrigation.
- It may be necessary to adjust lavage with the system in "Boost" mode (Foot Control fully depressed) so adequate fluid will be available to cool tip to tooth interface.
- In general, it is suggested that a "feather-light-touch" be used both supra- and subgingivally. The motion of the activated tip and acoustic effects of the irrigating fluid, in most cases, is adequate to remove even the most tenacious calculus.

- Periodically check the Cavitron[®] Ultrasonic Insert for wear with the DENTSPLY Insert Efficiency Indicator.
- The use of a saliva ejector or High Volume Evacuator (HVE) is recommended during all procedures.
- Set the System's Power Adjustment knob to the lowest power setting for the application and the selected insert.

10.3 Patient Comfort Considerations

Reasons for sensitivity

- Incorrect tip placement. Point should be directed away from root surfaces.
- Not keeping tip in motion on tooth. Do not allow the insert to remain in a static position on any one area of the tooth. Change the insert's path of motion.
- Applying pressure. Use extremely light grasp and pressure, especially on exposed cementum.
- If sensitivity persists, decrease power setting and/or move from the sensitive tooth to another and then return.

Section 11: System Care

It is recommended that you perform the following maintenance procedures.

11.1 Daily Maintenance

Start-Up Procedures at the beginning of the day (Cavitron® Select™ SPS™ Ultrasonic Scaler):

1. Open the manual shut-off valve on the dental office water supply system.
2. Turn the System ON using the Power Control knob (see illustration in Section 7). Verify the Power Indicator Light is lighted.
3. Connect a freshly sterilized handpiece to its matching cable connector. Set the Power Control knob to full power.
4. Set the Lavage Control on Handpiece to maximum.
5. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain. Activate the Foot Control and flush the water line for at least 2 minutes.
6. Place a sterilized Cavitron® Insert into the Handpiece. Set the Power and Lavage Control knobs to your preferred operating position.

Start-Up Procedures at the beginning of the day (Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System):

1. Carefully lock Cavitron® Select™ Reservoir Bottle into Cavitron® Select™ Reservoir Pump.
2. Fill Cavitron® Select™ Reservoir Bottle with tap, distilled water, or medicament. Replace Cavitron® Select™ Bottle cap.
3. Connect Cavitron® Select™ Scaler to Cavitron® Select™ Reservoir Pump and Bottle (see Section 6 for complete instructions).
4. Turn the System ON using the Power Control knob (see illustration in Section 7). Verify the Power Indicator Light is lighted.
5. Connect a freshly sterilized handpiece to its matching cable connector. Set the Power Control knob to full power.
6. Set the Lavage Control on the Handpiece to maximum.
7. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain. Activate the Foot Control and flush the water line for at least 2 minutes.
8. Place a sterilized Cavitron® Insert into the Handpiece. Set the Lavage Control on the Handpiece and Power control knobs to your preferred operating position.

Between Patients (Cavitron® Select™ SPS™ Ultrasonic Scaler):

1. Remove all ultrasonic inserts and handpiece used. Clean and sterilize inserts and handpiece and clean and disinfect all the surfaces of the cabinet, power cord, handpiece cable and foot control and cable assembly as instructed in the Cavitron Systems Infection Control Procedure booklet (#81158). To maintain sterility of the Steri-Mate® handpiece, store it in the steam sterilization pouch until needed.
2. Connect a freshly sterilized handpiece to its matching cable connector. Hold the Handpiece over a sink or drain and flush the water line of the unit as above at maximum water flow for 30 seconds.
3. Place a freshly sterilized insert into the handpiece.

* NOTE: Water-based disinfection solutions are preferred. Some alcohol-based disinfectant solutions may be harmful and may discolor plastic materials.

Between Patients (Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System):

1. Follow steps 1–2 listed in *Between Patients (Cavitron® Select™ SPS™ Ultrasonic Scaler)*.
2. With the power OFF, unscrew cap and fill Cavitron® Select™ Reservoir Bottle with tap, distilled water, or medicament.
3. Do not overfill Bottle with liquid into the neck of the Bottle. Securely screw cap back on Bottle.
4. Place a freshly sterilized insert into the handpiece.

Shut-Down Procedures at the end of the day (Cavitron® Select™ SPS™ Ultrasonic Scaler):

1. Turn the System OFF.
2. Remove all ultrasonic inserts and handpiece used. Clean and sterilize inserts and handpiece and clean and disinfect all the surfaces of the cabinet, power cord, handpiece cable and foot control and cable assembly as instructed in the Cavitron Systems Infection Control Procedure booklet (#81158). To maintain sterility of the Steri-Mate® handpiece, store it in the steam sterilization pouch until needed.
3. Close the manual shut-off valve on the dental water supply system.

Shut-Down Procedures at the end of the day (Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System):

1. Remove Cavitron® Insert from Handpiece.
2. Turn the System OFF.
3. Detach Cavitron® Select™ SPS™ Scaler from Reservoir Pump and Bottle.
4. Detach Cavitron® Select™ Reservoir Bottle from Pump (see Section 6 for complete instructions).
5. Empty entire contents of Cavitron® Select™ Reservoir Bottle.
6. Replace empty Cavitron® Select™ Reservoir Bottle to Pump.
7. Carefully lock Cavitron® Select™ Reservoir Bottle into Cavitron® Select™ Reservoir Pump.
8. Connect Cavitron® Select™ SPS™ Scaler to Cavitron® Select™ Reservoir System (see Section 6 for complete instructions).
9. Turn unit ON.
10. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
11. To air purge the water line, activate foot control to operate the Cavitron® Select™ SPS™ Scaler for at least 30 seconds. (This will eliminate fluid to prevent it from settling in the Cavitron® Select™ SPS™ System for extended periods of time. This may also help decrease the chance of bio-film formations in the Cavitron® Select™ SPS™ water lines.)
12. Follow steps 2–3 listed in *Shut-Down Procedures at the end of the day (Cavitron® Select™ SPS™ Ultrasonic Scaler)*.

11.2 Weekly Maintenance: Cavitron® Select™ SPS™ Ultrasonic Scaler with Cavitron® Select™ Reservoir System

1. Remove Cavitron® Insert from Handpiece.
2. Turn the System OFF.
3. Detach Cavitron® Select™ SPS™ Scaler from Reservoir System.
4. Detach Cavitron® Select™ Reservoir Bottle from Pump (see Section 6 for complete instructions).
5. Empty entire contents of Cavitron® Select™ Reservoir Bottle.
6. Replace empty Cavitron® Select™ Reservoir Bottle to Pump.
7. Carefully lock Cavitron® Select™ Reservoir Bottle into Cavitron® Select™ Reservoir Pump.
8. Connect Cavitron® Select™ SPS™ Scaler to Cavitron® Select™ Reservoir System (see Section 6 for complete instructions).
9. Turn the System ON.
10. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
11. To air purge the water line, operate the Cavitron® Select™ SPS™ Scaler for at least 30 seconds. (This will eliminate fluid to prevent it from settling in the Cavitron® Select™ SPS™ Scaler for extended periods of time. This may also help decrease the chance of bio-film formations in the Cavitron® Select™ SPS™ water lines.)
12. Turn the System OFF.
13. Prepare a 1:10 hypochlorite (NaOCL) solution by mixing 1 part 5.25% sodium hypochlorite with 10 parts tap water.
14. Remove cap and pour this 1:10 solution into the Cavitron® Select™ SPS™ Reservoir Bottle. Do not overfill with solution (see Section 3.3 for complete instructions).
15. Securely replace cap on Bottle.
16. Turn the System ON.
17. Hold the Handpiece (without the Cavitron® Insert installed) upright over a sink or drain.
18. Operate the Cavitron® Select™ SPS™ Scaler for at least 10 seconds.
19. Turn the System OFF.
20. *Leave the unit undisturbed for at least 10 minutes, BUT FOR NO MORE THAN 30 MINUTES to allow sodium hypochlorite solution to disinfect the system. As a suggestion, place a sign which states that the System is being disinfected and should not be used.*
21. Unscrew cap and discard disinfectant solution.
22. Fill Cavitron® Select™ Reservoir Bottle with tap or distilled water. Do not overfill with liquid (see Section 3.3 for complete instructions).
23. Turn the System ON.
24. Operate the Cavitron® Select™ SPS™ Scaler for at least 60 seconds.
25. Turn off system. Detach Cavitron® Select™ SPS™ scaler from Select™ Reservoir System.

26. Detach Cavitron® Select™ Reservoir Bottle from Pump (see Section 6 for complete instructions).
27. Autoclave Cavitron® Select™ Reservoir Bottle.

NOTE: ALL CHEMICALS MUST BE FLUSHED FROM THE SYSTEM BEFORE IT IS READY FOR PATIENT USE.

11.3 Water Line Filter Maintenance

When the water line filter becomes discolored, the filter should be replaced to prevent reduced water flow to the Cavitron® Select™ SPS™ System. A 10-pack of replacement filters is available by ordering Part Number 90158 from your local DENTSPLY distributor.

1. Verify the system is turned off.
2. Disconnect the water supply hose from the dental office water supply. If a quick-disconnect connector is attached to the end of the hose, relieve the water pressure by pressing the tip of the connector in an appropriate container and drain the water.
3. Grasp the fittings on either side of the filter disk and twist counterclockwise. Remove the filter section from either side of the water hose.
4. Install the replacement filter onto the water hose fittings. The filter should be positioned to match up with the correct hose fitting.
5. Hand tighten the two hose fittings in a clockwise direction. Reconnect the water supply hose, operate the unit to bleed the air and test for leaks.

11.4 Pump Lavage Filter Maintenance

When the pump lavage filter becomes discolored, the filter should be replaced to prevent reduced water flow to the Cavitron® Select™ SPS™ System. A 10-pack of replacement filters is available by ordering Part Number 90191 from your local DENTSPLY distributor.

1. Empty the contents of the Cavitron® Select™ Reservoir bottle and disconnect it from the pump.
2. Remove the old filter and discard.
3. Insert the small end of the new filter until filter is seated.

11.5 O-Ring Maintenance

Before placing Cavitron® Inserts into the handpiece, performing the following steps will increase the O-ring life on Cavitron® Inserts:

- Fill entire handpiece with water.
- Lubricate O-ring (black or green rubber material) with water.
- Gently twist the insert down into the handpiece until the insert is fully seated in place.

Section 12: Troubleshooting

Although service and repair of the Cavitron® Select™ SPS™ should be performed by DENTSPLY personnel, the following are some basic trouble shooting procedures that will help avoid unnecessary service calls. Generally, check all lines and connections to and from the System, a loose plug or connection will often create problems. Check the settings on the System's knobs.

12.1 Troubleshooting Guide

System will not operate:

(Power Indicator Light is not lighted.)

1. Check that the Power Control knob is in the ON position, and that the detachable Power Cord is fully seated in the receptacle on back of the Cavitron® Select™ SPS™ System.
2. Check that the System's three-prong plug is fully seated in an appropriate AC receptacle, and that AC current is present.
3. Check that the DC power plug is properly inserted into the DC input located on the back of the System.

(Power Indicator Light is lighted.)

1. Check that the Foot Control Connector is fully seated in the Foot Control Receptacle on the back of the System.
2. Check that the Handpiece Cable Assembly is fully seated in the Handpiece Receptacle.

System operates:

(No lavage flow at insert tip.)

1. Assure that Lavage control is properly adjusted.
2. Check that irrigant supply control valve/s (dental office water supply) are open.
3. Check that the Handpiece Cable Assembly is fully seated in the Handpiece Receptacle.
4. Check to see if Water Line or Pump Lavage Filter is discolored—if so, replace.
5. Check to assure that liquid is available in reservoir bottle.

12.2 Technical Support and Repairs

For technical support and assistance call 1-800-989-8826 Monday through Friday, 8:00 AM to 5:00 PM (Eastern Time). For other areas, contact your local DENTSPLY representative.

Section 13: Specifications

Cavitron® Select™: 100-240 VAC Systems

Electrical Voltage	30 VDC +/- 5%	
Current	2 amps DC Max	
Frequency	50/60 Hertz	
Water Pressure	20 – 40 psig	
Flow Rate	Minimum setting (CCW) 0 – 15 ml/min Maximum setting (CW) >45 ml/min	
Weight	425 g / 15 oz.	
Dimensions	Height:	2.4 in.
	Width:	3-1/8 in.
	Depth:	7 1/2 in.
	Handpiece Cable:	72 in.
	Foot Control Cable:	72 in.
	Power Cord:	2 m
	Water Supply Line:	96 in.

Cavitron® Select™ Power Supply (674022001):

Electrical Voltage	100 – 240 VAC	
Current	1,6 amps Max @ 100 – 240 VAC	
Insulation Requirement	Reinforce	
Frequency	47 – 63 Hertz	
Weight	550 g / 19.4 oz. Max	
Dimensions	Height:	14,3 cm / 5.7 in.
	Width:	7,6 cm / 3.0 in.
	Depth:	14,3 cm / 5.7 in.
	Power Cord:	213 cm / 84 in.

Footswitch
Not for operating theatres
Protection Class IPX1

Cavitron® Select™ Reservoir Pump:

Electrical Voltage	30 VDC +/- 5%	
Pump Pressure	15 – 19 psig	
Flow Rate	40 ml/min	
Weight	513 g / 18 oz.	
Dimensions	Height:	10,2 cm / 4.0 in.
	Width:	8,4 cm / 3.3 in.
	Length:	30 cm / 11.8 in.

Operating Environment:

Temperature:	15 to 40 Deg. Celsius
Relative Humidity:	30% to 75% (non-condensing)

Transport and Storage Conditions:

Temperature:	0 to 70 Deg. Celsius
Relative Humidity:	10% to 95% (non-condensing)
Atmospheric Pressure:	500 to 1060 hPa

Symbology

 DIRECT CURRENT

 TYPE B EQUIPMENT

 FOOTSWITCH

 CAUTION, CONSULT ACCOMPANYING DOCUMENTS

 CONSULT INSTRUCTIONS FOR USE

 This symbol is a mandatory marking for devices entering the European market to indicate conformity with the essential health and safety requirements set out in European Directives. The symbol may be accompanied by a four-digit identification number of the notified body.

 Dispose of in accordance with the Waste Electrical and Electronic Equipment Directive 2002/96/EC of the European Parliament and the Council of the European Union

WEEE

 EMC
EUR - USA


Section 14: Classifications

• Type of protection against electric shock:	Class 1
• Degree of protection against electric shock:	Type B
• Degree of protection against the harmful ingress of water:	Ordinary
• Mode of operation:	Continuous
• Degree of safety of application in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide:	Equipment not suitable for use in the presence of flammable anesthetics or oxygen.
• According to medical device directive:	IIA (rule 9)

Section 15: Disposal of Unit

• Accordance with local and state laws.

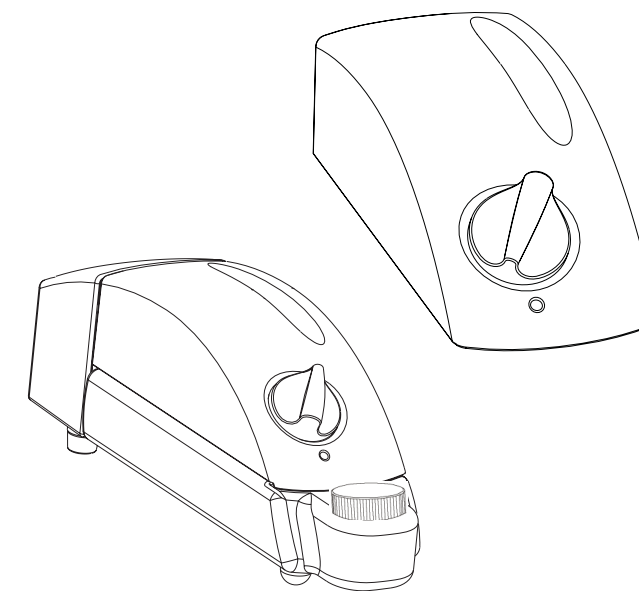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Technical Support and Repairs

For technical support and repair assistance call Dentsply Sirona Cavitron Care Factory Certified Service at 1-800-989-8826 Monday through Friday, 8:00 A.M. to 5:00 P.M. (Eastern Time). For areas outside the U.S. contact your local Dentsply Sirona representative.

Printed in U.S.A. Form #90295 Rev. 6 (0317)

DENTSPLY
PROFESSIONAL


Manufactured by
DENTSPLY Professional
1301 Smile Way
York, PA 17404 USA

CE
0086

EC REP

DENTSPLY DeTrey GmbH
De-Trey-Str.1
78467 Konstanz,
Germany

Distributed by
DENTSPLY Canada
Woodbridge, Ontario
L4L 4A3

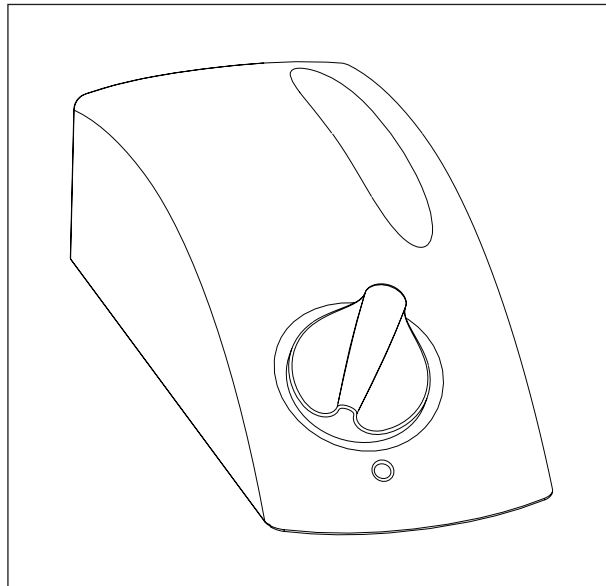**Cavitron® Select™ SPS™ Ultrasonic Scaler
with Cavitron® Select™ Reservoir System**
Directions For Use

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