

## DIRECTIONS

1. Stir two capfuls (2 oz.) of powder into one quart of warm water until completely dissolved.
2. Strip bulk of material from instruments or trays and completely immerse in solution.
3. Immerse the tray in a solution of tray cleaner and warm water for 15-30 minutes. Hot solution removes residue in a few minutes.
4. Stubborn particles may be removed with slight brushing. Rinse instruments and trays with running water before processing or storing.

**CAUTION:** Clean nickel and chrome instruments and trays in a separate solution from that used for aluminum instruments. Do not leave instruments in solution longer than is necessary for cleaning. Prolonged immersion may cause some corrosion, particularly on worn areas.

Contains/Enthält:  
Sodium carbonate/Natriumcarbonat

XI

C



*Irritant/Reizend*



*Corrosive / Ätzend*

- Causes burns / Verursacht Verätzungen
- Irritating to eyes and respiratory system / Reizt die Augen und die Atmungsorgane
- Do not breathe dust / Staub nicht einatmen
- Avoid contact with skin and eyes / Berührung mit den Augen und der Haut vermeiden
- In case of contact with eyes, rinse immediately with plenty of water and seek medical advice / Bei Berührung mit den Augen sofort gründlich mit Wasser abspülen und Arzt konsultieren
- Wear suitable protective clothing and gloves / Bei der Arbeit geeignete Schutzkleidung und Schutzhandschuhe tragen



GC AMERICA INC.  
ALSIP, IL 60803  
MADE IN U.S.A.

GC EUROPE N.V.  
Interleuvenlaan 13 - B-3001 LEUVEN  
Tel.: +32.16.39.80.50 - Fax +32.16.40.02.14

5930204