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**Processing instructions according to
EN ISO 17664:201**

Product: Del-Ject Amalgam Carrier

Medical device**REF** 355 035 – 355 038Manufacturer: HAGER & WERKEN GmbH & Co. KG
Ackerstraße 1, 47269 Duisburg, Germany

Warnings: Attention! The product must be cleaned, disinfected and sterilized according to these processing instructions before its first use as well as after every subsequent use. Disinfection alone is not enough. Thorough cleaning and disinfection are decisive prerequisites for effective sterilization. Processing should be started as soon as possible but not later than 2 hours after use. During the processing, the product should not be unnecessarily exposed to wetness or humidity. Please also observe the legal regulations applicable in your country as well as the hygiene regulations of the medical practice and/or the hospital.

Limitations in the processing: Frequent processing has only minor effects on this product. The end of the product's useful life is mainly determined by wear and damage caused by use. In case of doubt, the products should always be sorted out and replaced early. The user alone is responsible for making the decision regarding repeated use of the product. If the product is used too frequently, the manufacturer does not accept any warranty for the function, performance and safety of the product. **Personal protective equipment:** For reasons of occupational safety and to minimize infections/cross infections, suitable personal protective equipment (protective clothes, protective gloves, protective goggles and nose/mouth protection) must be worn during the entire processing period.

INSTRUCTIONS:

INITIAL TREATMENT AT THE PLACE OF USE

Remove surface contamination using a disposable cloth/paper cloth when you are still at the place of use. Rinse the products with water (at least drinking water quality) no later than 2 hours after the first use. Avoid drying of residues or contamination of any kind on the product. When using the product for the first time at the place of use, do not use any agents containing aldehyde or alcohol as they may cause protein fixation.

PREPARATION BEFORE CLEANING

Disassemble the product into its individual components. Perform a visual inspection for damage and wear.

CLEANING & DISINFECTION: BY MACHINE (RECOMMENDED)

Only perfectly maintained, checked, calibrated and released cleaning and disinfection equipment according to EN ISO 15883 with valid certification mark (CE mark) may be used. Make sure that during machine cleaning and disinfection of several different products/individual components, the cleaning device is not overfilled and that the products/individual components do not contact each other. We recommend completing the cleaning and disinfection cycle according to the specifications of the device manufacturer and the manufacturer of the cleaning agent and/or disinfectant. Make sure that the cleaning agent is compatible with the products and disinfectants that might be used. The pH of the cleaning agent and disinfectant should be between 5.5 and 8.5. Do not use organic solvents (e.g. alcohols, ether, ketones, petrol), oxidizing agents (e.g. peroxides), halogens (chlorine, iodine, bromine) or aromatic/halogen hydrocarbons.

Application with the following agents is recommended:

Cleaning agent - Dr. Weigert, neodisher MA

Neutralizer - Dr. Weigert, neodisher Z

The instructions for use of the manufacturer of cleaning agent and disinfectant must strictly be observed. In particular, the concentrations and contact times must be observed.

Description of the recommended cleaning process:

Pre-cleaning: The pre-cleaning should be completed in a water bath (at least drinking water quality) without other cleaning additions using a cleaning brush below the water surface to achieve both sufficient cleaning of the products/individual components avoiding protein fixation, and to protect the environment from contamination with splash water. The duration should be based on the level of contamination of the product/individual component; it should, however, not be less than 1 minute. Cleaning tools for pre-cleaning: Mirasuc Brushes REF 605 216, Chiru-Cleaner 23 REF 605 215.

Information on the cleaning and disinfection device: Type: Miele G7881

Description of the recommended program:

Cleaning program: VarioTD

Cleaning temperature: 50.0-56.6°C

Hold time: 11:03 min

Disinfection temperature: 65.0 - 92.3°C

Hold time: 09:59 min

Flushing technology: Fresh water system/demineralized water

Cleaning agent: Dr. Weigert, neodisher MA, Dosing: 30 mg (3g/l)

Neutralizer: Dr. Weigert, neodisher Z, Dosing: 1 - 2 ml/L

The water should be of drinking water quality. Flushing and drying should be completed by machine. The machine cleaning should not exceed a temperature of 60 °C. Ensure that visual contamination has been completely removed. If there is still contamination after cleaning and disinfection, these processes must be repeated. Before further processing, the product must be free from any residues and dry.

CLEANING: MANUAL

The manual cleaning should be completed in a water bath (at least drinking water quality) with the cleaning agent specified below using a brush below the water surface to achieve both sufficient cleaning of the products/individual components avoiding protein fixation, and to protect the environment from contamination with splash water. The duration should be based on the level of contamination of the product/individual component; it should, however, not be less than 1 minute. Afterwards, rinse the products/individual components with water (at least drinking water quality) for at least 20 seconds.

Description of the recommended cleaning process:

- Pre-cleaning: Remove surface contamination using a disposable cloth
Cleaning: Insert the instrument parts into the strainer bowl. Suspend the strainer bowl into the cleaning bath with cleaning solution, remove contamination by means of cleaning brushes.
Cleaning agent: ID 213, company Dürr Dental, concentration: 2 %
Cleaning tools: Mirasuc Brushes REF 605 216, Chiru-Cleaner 23 REF 605 215
Rinsing: Rinse with water (at least drinking water quality) for 20 sec.
Drying: Dry at room temperature

The manual cleaning should not exceed a temperature of 45 °C. Ensure that visual contamination has been completely removed. If there is still visible contamination after the cleaning process, cleaning must be repeated.

DISINFECTION: MANUAL

For manual disinfection, please use only approved disinfectants with tested effectiveness (CE mark, VAH-/DGHM-listed). Put the products/individual components into corresponding disinfection baths according to the disinfectant manufacturer's specifications. Ensure that the products are sufficiently covered and do not come into contact times must be observed. The instructions for use of the disinfectant manufacturer must be strictly observed. In particular, concentrations and contact times have to be observed. Use with the following agents is recommended: Dürr Dental ID 213 instrument disinfection, concentration: 2 %.

Description of the recommended disinfection process:

- Completion of proper cleaning preparation and manual cleaning
Disinfection: Insert the instrument parts into the strainer bowl. Hang the strainer bowl into the disinfection bath with disinfection solution
Disinfectant: ID 213, company Dürr Dental, concentration: 2 %
Cleaning tools: Mirasuc Brushes REF 605 216, Chiru-Cleaner 23 REF 605 215.
Contact time: 5 min
Rinsing: Rinse with water (at least drinking water quality) for 20 sec.
Drying: Dry at room temperature

Make sure that the disinfectant is compatible with the products and cleaning agents that might be used. The pH of the disinfectant should be between 5.5 and 8.5. Do not use organic solvents (e.g. alcohols, ether, ketones, petrol), oxidizing agents (e.g. peroxides), halogens (chlorine, iodine, bromine) or aromatic/halogen

hydrocarbons. Please observe any thermal restrictions that might apply while taking the disinfectant manufacturer's specifications into consideration. Afterwards, rinse the products/individual components with water (at least drinking water quality) for at least 20 seconds. Before further processing, the product must be free from any residues and dry.

DRYING

If the products are processed manually, the products/individual components may be dried by blowing them off with filtered, oil-free compressed air according to DIN ISO 8573-1 (medical cleanliness class) or at room temperature. If you process the products by machines, drying should be performed directly by the relevant equipment using hot air. For machine drying, a temperature of 100°C should not be exceeded.

MAINTENANCE, CONTROL AND INSPECTION

The products do not require special maintenance. Reassemble disassembled products/individual components. Always perform a visual inspection for contamination, damage, wear and deformation before and after the individual work steps. Damaged or corroded products must no longer be used. If the product/individual component is not visibly clean, the entire processing must be repeated or the product/individual component must be disposed of properly.

PACKAGING

The product is to be packed in suitable and standardized transparent sterile packaging (sterilization bags) and sealed. Observe the instructions of the sterilization bag and sealing machine manufacturers as well as the current, normative requirements. Products/individual parts that have not been sterilized in bags must be used immediately.

STERILIZATION

Only tested steam vacuum autoclaves may be used. Make sure that during the sterilization of several different products/individual components, the autoclave is not overfilled and that the products/individual components do not come into contact with each other. The following sterilization cycles may be completed: Steam sterilization, 134 °C, hold time 5 minutes or steam sterilization, 121 °C, hold time 15 minutes. To dry the products/individual components, the drying cycle of the autoclave should be configured accordingly. Please observe the autoclave manufacturer's usage instructions

STORAGE

To maintain the sterility, the products should be stored in standardized sterilization bags at a dry, clean place until they are used. If the sterile packaging is damaged, the products must be processed again before their use.

ADDITIONAL INFORMATION

Mark the sterilized products according to the legal and national regulations. The recommended storage duration for sterile medical devices is described in standard DIN 58953-8 and depends on external influences and effects during storage, transport and handling. The user must ensure within the scope of their quality management system that specified processing cycles that might apply (see Limitations in the processing) are not exceeded. The manufacturer and the competent authority of your member state must be immediately notified of all serious incidents occurring in connection with the product. Recommendations and regulations for disposal of the products vary by region. Ask your competent disposal company for the current regulations in your region.

TO CONTACT TO THE MANUFACTURER

Phone: (+49) 203/99269 -0

Fax: (+49) 203/299283

Email: info@hagerwerken.de

The instructions specified above are validated by the medical device manufacturer as suitable for the preparation of a medical device for its reuse. The processor is responsible for ensuring that the completed processing with the equipment, materials and personnel used in the preparation facility achieves the desired result. To this end, verification and/or validation and routine monitoring of the procedure are required.

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HAGER & WERKEN GmbH & Co. KG · Ackerstraße 1 · 47269 Duisburg, Germany