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trayloc® A 03098
17 ml botella con pincel

Detaseal[®] hydroflow Xlite regular / fast

VPS precision impression silicone – extra light bodied consistency

Fig. 1

1. Tray preparation

Select a tray type suited for desired application (standard-size or custom, in metal or hard plastics). To avoid teeth contact with the tray, place stops from wax or light-curing resin into the tray at non-relevant areas of the impression. For optimal adhesion of impression material, we recommend applying a thin layer of **trayloc[®] A** to tray (Fig. 1). Let dry for approx. 3 minutes.

Fig. 2

2. Mixing and dispensing

Insert cartridge into Automix2-System mixing gun (Fig. 2). Remove and dispose of closure cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved (Fig. 3). Mount the mixing cannula by using the guides on the cannula and the cartridge and lock in place by rotating in the opposite direction (Fig. 4). Feed material applying even pressure. Prior to use, dispense a small amount of material and visually check that the base and catalyst paste are mixed omogeneously (Fig. 5). Only then dosing can be performed individually. After each use leave the used mixing cannula in place to seal the cartridge.

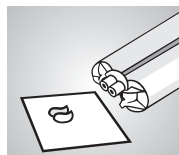
Fig. 3

3. Application

Apply **Detaseal[®] hydroflow Xlite** according to impression technique (e.g. double-mix technique, Fig. 5). For applications in the groove area use intra-oral-tip or disposable syringe (Fig. 6) Then seat the impression until **Detaseal[®] hydroflow Xlite** has set.

Fig. 4

4. Disinfecting

After removing tray from the mouth, rinse impression under running, lukewarm water. If desired, impression can then be disinfected by immersing into 2% glutaraldehyde or other usual solutions for 15 minutes.

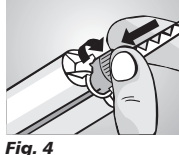
Fig. 5

5. Pouring the model

We recommend waiting at least 30 minutes before pouring the model. The impression will retain its dimensional stability for up to 14 days. All class III and IV dental plasters and standard modelling resins can be used.

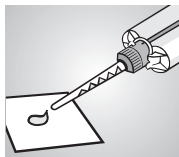
Fig. 6

6. Cleaning the tray

After setting, material can be removed with a blunt instrument. Remaining film of **trayloc[®] A** is dissolved by immersion in acetone or isopropanol (be sure to use only in well-ventilated rooms). Tray can then be cleaned and disinfected in the usual manner.

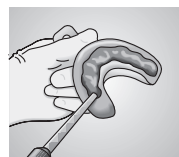
Fig. 7

Important working hints


- Substances such as retraction liquids can adversely affect the setting reaction and should therefore be thoroughly rinsed off before taking the impression.
- Do not combine with condensation curing silicones.
- Contact with latex examination gloves or other latex material may impair the setting reaction of **Detaseal[®] hydroflow Xlite**.
- Be sure to remove all traces of impression material from patient's mouth.
- Cured impression materials are chemically inert – spots on clothing should be avoided.
- We recommend use of **Detaseal[®] hydroflow putty, -heavy** or **-Jumbo putty** as tray material.
- Please follow the instructions of the safety data sheet!

Further information:


Silicone based materials are proven a million times. On condition of a proper application, **undesired effects** are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

Only for dental use by qualified personnel.

Detaseal[®] hydroflow Xlite regular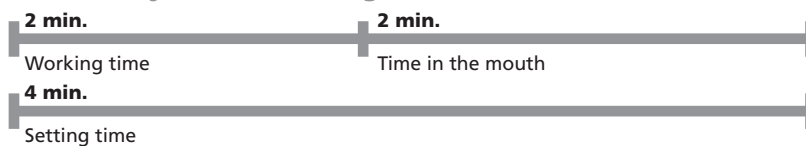

Detaseal[®] hydroflow Xlite fast

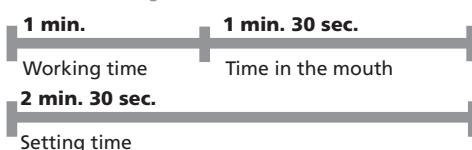Indications for use:

- Impressions with sandwich and double mix technique
- Corretion impressions
- Reline impressions

Technical data:

- DIN EN ISO 4823 - Type 3
- Mixed volume:** 50 ml cartridge
- Mixing ratio:** 1:1
- Product colour:** base: blue catalyst: white
- Mixing time:** omitted (Automix2-System)
- Working time:** **regular:** approx. 2 min.* **fast:** approx. 1 min.*
- Setting time:** **regular:** approx. 4 min.* **fast:** approx. 2 min. 30 sec.*
- Time in the mouth:** **regular:** approx. 2 min. **fast:** approx. 1 min. 30 sec
- Strain in compression:** approx. 6,5 %
- Recovery from deformation:** approx. 99,8 %
- Linear dimensional change:** approx. 0,2 %
- Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity.
- Storage:**

* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard these times.

Ordering information:

- Detaseal[®] hydroflow Xlite**
Standard packing
2 cartridges of 50 ml
12 mixing cannulas, yellow
regular set 02741
fast set 02744
- Detaseal[®] hydroflow lite**
Standard packing
2 cartridges of 50 ml
12 mixing cannulas, yellow
regular set 02730
fast set 02733
- Mixing cannulas **02706**
yellow, 48 pcs.
- Detaseal[®] hydroflow putty**
Standard packing **02727**
2 jars of 250 ml
2 measuring scoops
- Jumbo packing 03283**
base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow
- Detaseal[®] hydroflow heavy**
Jumbo packing **02729**
base + catalyst 380 ml
10 dynamic mixers, yellow
1 rotating ring, yellow
- trayloc[®] A**
17 ml bottle with brush **03098**
15 ml spray bottle **03392**

DETAX

GmbH & Co. KG
Postfach 100225 · 76256 Ettlingen / Germany
Telefon 07243/510-0 · Fax 07243/510-100
www.detax.de · mail@detax.de

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personnel).

Made in Germany

10/2017

Detaseal® hydroflow Xlite regular / fast

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