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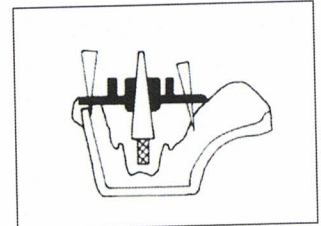
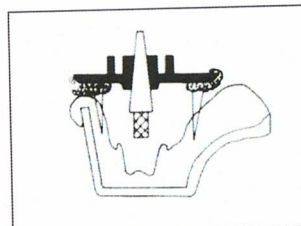
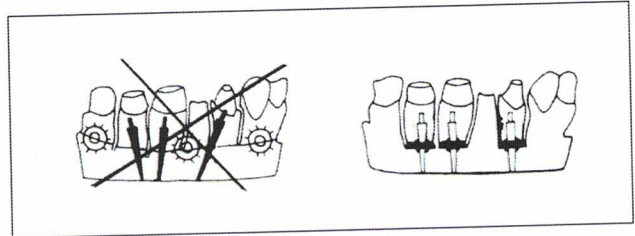
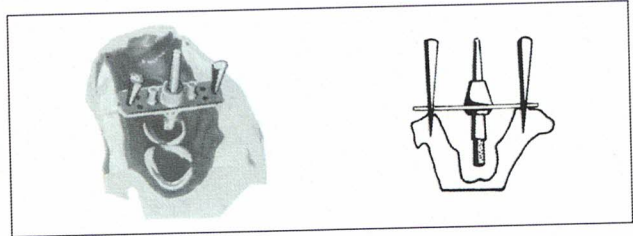
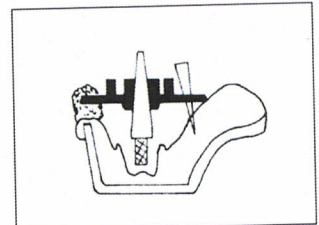
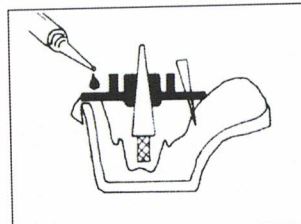
Contains plastic and dye.

Fields of application:

- To produce saw dies
- Guarantees separation of the model die from the complete module
- Serves to provide exact and solid guidance and acts as a firm base
- For dental use only

Contra-indications:

None

Side effects/interactions:

None known if used according to intended purpose.

Dosage, type and duration of treatment:

Procedure:

Detach the fixing pin from the guide plate. Insert the Dowel-Pin securely in the guide plate and place in the impression so that the Dowel-Pin projects into the prepared stump.

If the mucolabial fold is too high, remove some of the impression material so that the Dowel-Pin projects far enough into the tooth stump to be poured.

Fix the guide plate with the help of fixing pins so that it is shakeproof or, with very delicate bases, secure using sticky wax.

The complete impression is poured in a single step (tooth row/model base). As such, pouring twice is not required nor is it necessary to isolate between the several layers and fix the retention rings. After the plaster has set, remove the impression material.

Saw the model die out up to the guide plate.

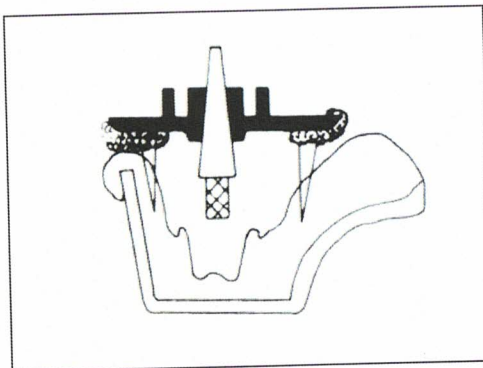
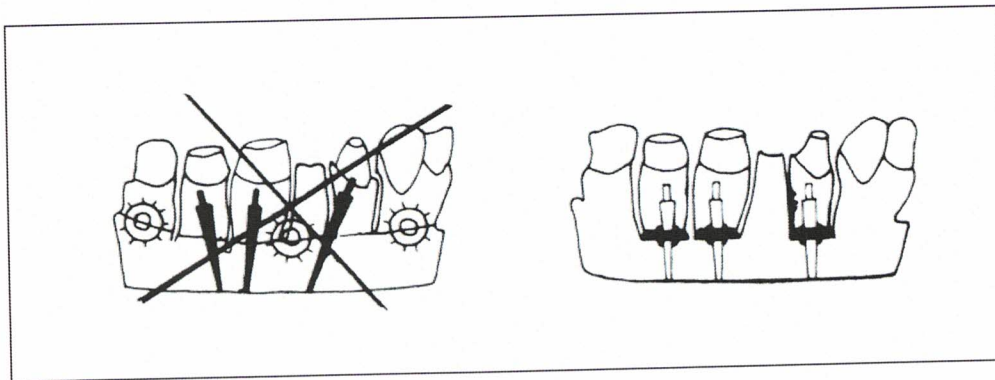
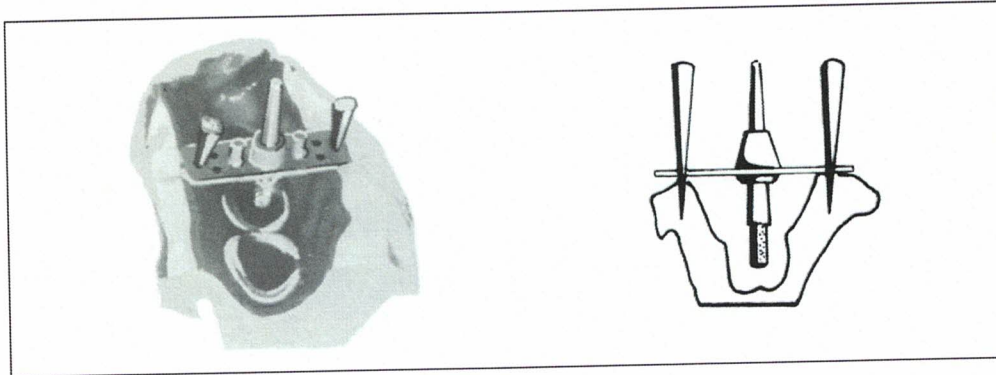
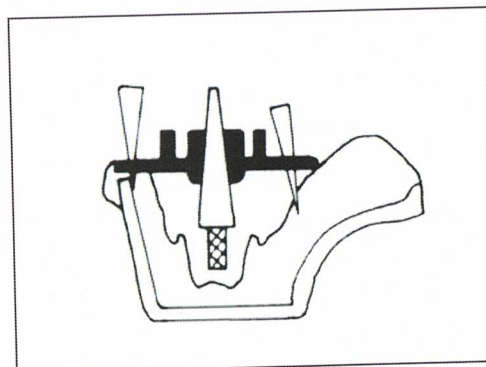
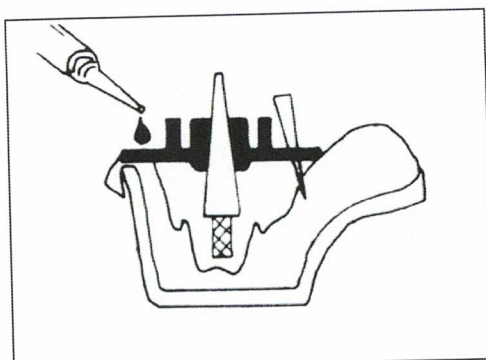
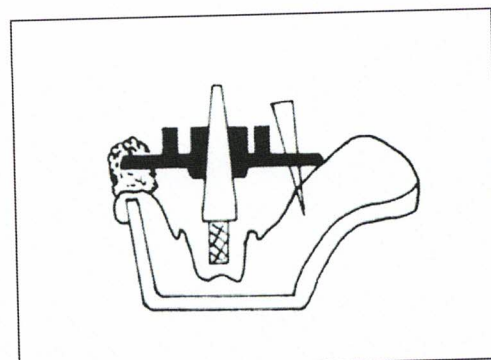
The guide plate ensures that the model die is separated from the complete module and also serves as an exact and solid guide as well as a firm base.

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