CE SPEIKO Easy Endo Tips

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CE SPEIKO Easy Endo Tips

Composition:

Contains polypropylene, transparent.

Fields of application:

- Irrigation of root canals
- For dental use only

Contra-indications:

- None known if used as intended

Side effects/interactions:

None known if used as intended.

Dosage, type and duration of treatment:

Connect the Easy Endo Tip with the thread to the connection of a previously filled Luer or Luer-Lock syringe.

The Easy Endo Tip is secured by turning firmly in the case of a Luer-Lock connection or pressing on firmly in the case of a Luer connection.

Ensure that the connection is secure by pulling on it briefly and firmly.

Use the syringe and Easy Endo Tip for root canal irrigation.

After use, separate the Easy Endo Tip from the syringe and dispose of it according to the disposal instructions.

Hypochlorite-SPEIKO 5.25 %, Hypochlorite-SPEIKO 3 %, Hypochlorite-SPEIKO 1 %, SPEIKO EDTA Solution, SPEIKO Citric Acid Solution 10 % are compatible liquids.

The SPEIKO Easy Quick withdrawal system can be used to draw off the compatible liquids.

When using the Easy Endo Tip for root canal irrigation, it must be ensured that the cannula can move freely to guarantee a constant backflow of irrigation liquid.

Notes:

SPEIKO Easy Endo Tips are intended for single use. Do not reuse.

The device is manufactured ready for use.

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