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## GC Fuji™ IX GP EXTRA

RADIOPAQUE POSTERIOR GLASS IONOMER RESTORATIVE CEMENT IN CAPSULES

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- Class I and II restorations in non-indicous teeth.
- Class I and Class II restorations in permanent teeth.
- Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- Class V and root surface restorations.
- Core build-up.

### CONTRAINDICATIONS

- 1. Pulp capping.
- 2. Avoid use of this product in patients with known allergies to glass ionomer cement.

### DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)	0.40 / 0.12
Mixing Time (sec.) <sup>a</sup>	10 <sup>a</sup>
Time of initial extrusion after mixing	10 seconds maximum
Working Time (min., sec.) <sup>a</sup>	1'15"
Net Setting Time (min., sec.) <sup>a</sup>	2'00"
Final Finishing Commencing Time (37°C, after starting mix) (min., sec.) <sup>a</sup>	2'30"
Test conditions: Temperature (23 +/-1°C), Relative humidity (50 +/-5%)	

1. CAVITY PREPARATION

- a) Prepare tooth using standard techniques. Extensive mechanical preparation is unnecessary. For pulp capping use calcium hydroxide.
- b) Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).
- c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. **DO NOT DESCALICATE.** Prepared surfaces should remain moist.

### 2. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).
- b) To activate the capsule, push the plunger until it is flush with the main body (Fig. 3).

Note: The capsule should be activated just before mixing and used immediately.

d) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/-4.000RPM) (Fig. 5).

### 3. RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and tap it onto a GC Capsule Applier.
- b) Make two clicks to prime the capsule then synge (Fig. 6).

The working time is 1 minute 15 seconds from start of mixing at 23°C (74°F). Higher temperatures will shorten working time.

c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 7).

d) Form the preliminary contour, and cover with a matrix if required (Fig. 8).

e) During the first 2 minutes 30 seconds from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply or GC Fuji Varnish (blow dry) or GC Fuji Coat LC (light cure).

Note: 1) To adjust the direction of the nozzle, hold the applier with one capsule towards you and turn the capsule body.

2) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

### 4. FINISHING

- a) Final finishing under water spray using standard techniques can begin approximately 2 minutes 30 seconds after start of mixing (Fig. 9).
- b) Apply a final coat of G-COAT PLUS and light cure (alternatively apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) Refer to the respective instructions for use.

c) Instruct the patient not to apply pressure for 1 hour.

### STORAGE

Recommended for optimal performance, store the capsules in the original aluminum foil in a cool and dark place (42°C) (39.2-77.0°F).

### SHADES

A1, A2, A3, A3.5, B1, B2, B3, C4  
Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vito Zahnfabrik, Bad Säckingen, Germany.

### PACKAGES

Single shade package : 5 capsules.

2 Assortment package : 50 capsules (10 each of A2, A3, A3.5, B1, B3).

Average contents per capsule : 0.40g powder and 0.12g (0.10mL) liquid.

Minimum net volume of mixed cement per capsule : 0.14mL.

Option: GC CAPSULE APPLIER (1 piece).

### CAUTION

- 1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.
- 2. In case of contact with eyes, flush immediately with water and seek medical attention.

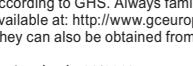
3. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

4. In rare cases the product may cause sensitivity in some people. If any such reaction are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com>

They can also be obtained from your supplier.

Last revised : 06/2018

MANUFACTURED by  
GC CORPORATION  
76-1 Hasuruma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasuruma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haasrode-einde 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

PRINTED IN BELGIUM

Prior to use, carefully read  
the instructions for use.

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- 1. Class I and II restorations in non-indicous teeth.
- 2. Intermediate restorations I and Class II restorations in permanent teeth.
- 3. Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
- 4. Class V and root surface restorations.
- 5. Core build-up.

### CONTRAINDICATIONS

- 1. Pulp capping.

2. Avoid use of this product in patients with known allergies to glass ionomer cement.

### DIRECTIONS FOR USE

Powder / Liquid Ratio (g/g)

0.40 / 0.12

Mixing Time (sec.)<sup>a</sup>

10<sup>a</sup>

Time of initial extrusion after mixing

10 seconds maximum

Working Time (min., sec.)<sup>a</sup>

1'15"

Net Setting Time (min., sec.)<sup>a</sup>

2'00"

Final Finishing Commencing Time (37°C, after starting mix) (min., sec.)<sup>a</sup>

2'30"

Test conditions: Temperature (23 +/-1°C), Relative humidity (50 +/-5%)

Fig. 1

Fig. 2

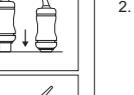
Fig. 3

Fig. 4

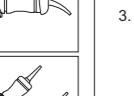
Fig. 5

Fig. 6

Fig. 7

Fig. 8

Fig. 9

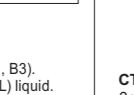
Fig. 10

1. CAVITY PREPARATION

- a) Prepare cavity using standard techniques. Extensive mechanical preparation is unnecessary. For pulp capping use calcium hydroxide.

b) Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).

c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. **DO NOT DESCALICATE.** Prepared surfaces should remain moist.

### 2. CAPSULE ACTIVATION AND MIXING

- a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).

b) To activate the capsule, push the plunger until it is flush with the main body (Fig. 3).

Note: The capsule should be activated just before mixing and used immediately.

d) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/-4.000RPM) (Fig. 5).

### 3. RESTORATIVE TECHNIQUE

- a) Immediately remove the mixed capsule from the mixer and tap it onto a GC Capsule Applier.

b) Make two clicks to prime the capsule then synge (Fig. 6).

The working time is 1 minute 15 seconds from start of mixing at 23°C (74°F). Higher temperatures will shorten working time.

c) Within 10 seconds maximum after mixing, start to extrude the mixture directly into the preparation (Fig. 7).

d) Form the preliminary contour, and cover with a matrix if required (Fig. 8).

e) During the first 2 minutes 30 seconds from start of mix extra care should be taken to avoid moisture contamination or drying-out. In case this cannot be guaranteed, immediately apply or GC Fuji Varnish (blow dry) or GC Fuji Coat LC (light cure).

Note: 1) To adjust the direction of the nozzle, hold the applier with one capsule towards you and turn the capsule body.

2) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

### 4. FINISHING

- a) Final finishing under water spray using standard techniques can begin approximately 2 minutes 30 seconds after start of mixing (Fig. 9).

b) Apply a final coat of G-COAT PLUS and light cure (alternatively apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) Refer to the respective instructions for use.

c) Instruct the patient not to apply pressure for 1 hour.

### STORAGE

Recommended for optimal performance, store the capsules in the original aluminum foil in a cool and dark place (42°C) (39.2-77.0°F).

### SHADES

A1, A2, A3, A3.5, B1, B2, B3, C4  
Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vito Zahnfabrik, Bad Säckingen, Germany.

### PACKAGES

Single shade package : 5 capsules.

2 Assortment package : 50 capsules (10 each of A2, A3, A3.5, B1, B3).

Average contents per capsule : 0.40g powder and 0.12g (0.10mL) liquid.

Minimum net volume of mixed cement per capsule : 0.14mL.

Option: GC CAPSULE APPLIER (1 piece).

### CAUTION

- 1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

2. In case of contact with eyes, flush immediately with water and seek medical attention.

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**RECOMMENDED INDICATIONS**

1. Class I and II restorations in deciduous teeth.
2. Non-load bearing Class I and Class II restorations in permanent teeth.
3. Intermediate restorative and base material for heavy stress situation in Class I and Class II cavities using sandwich laminate technique.
4. Class V and root surface restorations.
5. Core build-up.

**CONTRAINdications**

1. Pulp capping.
2. Avoid use of this product in patients with known allergies to glass ionomer cement.

**DIRECTIONS FOR USE**

Powder / Liquid Ratio (g/g)	0.40 / 0.12
Mixing Time (sec.)	10"
Time of initial extrusion after mixing	10 seconds maximum
Working Time (23 °C/73 °F) (min., sec.)	1'15"
Net Setting Time (37 °C/99 °F) (min., sec.)	2'00"
Finishing Time (37 °C/99 °F, from start of mixing) (min., sec.)	2'30"

**1. CAVITY PREPARATION**

a) Prepare tooth using standard techniques. Extensive mechanical retention is unnecessary. For pulp capping use calcium hydroxide.

b) Apply GC CAVITY CONDITIONER (10 seconds) or GC DENTIN CONDITIONER (20 seconds) to the bonding surfaces using a cotton pellet or sponge (Fig. 1).

c) Rinse thoroughly with water. Blot away excess water with a cotton pellet or dry by gently blowing with an air syringe. DO NOT DESICCATE. Prepared surfaces should appear moist (glistening).

**2. CAPSULE ACTIVATION AND MIXING**

a) Before activation, shake the capsule or tap its side on a hard surface to loosen the powder (Fig. 2).

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c) Immediately place the capsule into a metal GC Capsule Applier and click the lever once (Fig. 4). The capsule is now activated.

Note :  
The capsule should be activated just before mixing and used immediately.

d) Immediately remove the capsule and set it into a mixer (or an amalgamator) and mix for 10 seconds (+/- 4.000RPM) (Fig. 5).

**3. RESTORATIVE TECHNIQUE**

a) Immediately remove the mixed capsule from the mixer and load it into the GC Capsule Applier.

b) Make two clicks to prime the capsule then syringe (Fig. 6). The working time is 1 minute 15 seconds from start of mixing at 23°C (73.4°F). Higher temperatures will shorten working time.

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d) Form the preliminary contour, and cover with a matrix if required.

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Note :

1) To adjust the direction of the nozzle, hold the applier with the capsule towards you and turn the capsule body.  
2) To remove the used capsule, push the applier release button. Twist the capsule and pull upwards.

**4. FINISHING**

a) Final finishing under water spray using standard techniques can begin approximately 2 minutes 30 seconds after start of mixing (Fig. 8).

b) Apply a final coat of GC G-COAT PLUS and light cure for 20 seconds (Fig. 9,10).

Alternatively, apply GC Fuji VARNISH (blow dry) or GC Fuji COAT LC (light cure) Refer to the respective instructions for use.

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**STORAGE**

Recommended for optimal performance, store the capsules in the original aluminium foil in a cool and dark place (4-25°C (39.2-77.0°F)).

**SHADES**

A1, A2, A3, A3.5, B1, B2, B3, C4

Shade numbers according to Vita® shade guide.

Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

**PACKAGES**

1. Single shade package : 50 capsules.

2. Assortment package : 50 capsules (10 each of A2, A3, A3.5, B1, B3).

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Option : GC CAPSULE APPLIER (1 piece).

**CAUTION**

1. In case of contact with oral tissue or skin, remove immediately with a sponge or cotton soaked in alcohol. Flush with water.

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4. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gc-europe.com/>

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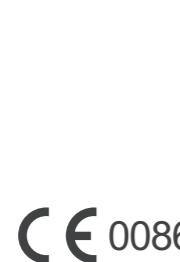
Last revised : 10/2018

MANUFACTURED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haasrode-Leeuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

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