GC Fuji LINING™ LC PASTE PAK

RADIOPAQUE LIGHT CURED GLASS IONOMER LINING CEMENT

For use only by a dental professional in the recommended indications.

GC Fuji LINING LC PASTE PAK is the first radiopaque, light cured resin-modified glass ionomer lining cement available in paste presentation. GC Fuji LINING LC PASTE PAK is available in a cartridge pack and can be dispensed easily and correctly using a special dispenser called Paste Pak Dispenser.

RECOMMENDED INDICATIONS
As a base or liner in prepared cavities.

CONTRAINDICATIONS
1. Pulp capping.
2. Avoid use of this product in patients with known allergies to glass ionomer cement, methacrylic monomer or methacrylate polymer.

IDENTIFICATION OF PARTS
(Paste Pak Cartridge)
1. Cover 2. Inner plunger 3. Cartridge tip (Paste Pak Dispenser)
4. Piston 5. Adjusting guide 6. Lever 7. Sliding block 8. Piston release (Fig. 1)

DIRECTIONS FOR USE

Paste / Paste Ratio (g/g)	1.0/1.1
Mixing Time (sec.)	10*
Working Time (min., sec.)	2'15"
Light Curing Time (sec.) (Halogen)	20*
Depth of Cure (mm)	1.3

1. PASTE PAK CARTRIDGE LOADING
a) Make sure that the piston is completely retracted into the dispenser. (Fig. 2)
Note:
To retract the piston, hold both the sliding block and piston release (on the underside of the dispenser) with fingers and pull them back together.
b) Load the cartridge into the Paste Pak Dispenser. (Fig. 3)
1) Ensure the mark on the cartridge lines up with the mark on the dispenser.
2) Then turn the cartridge fully to the right position.
Note:
1) If the cartridge is stored in a refrigerator, before you remove it, room temperature (23°C / 73°F) for 30 minutes.
2) To remove the cartridge, retract the piston fully and turn the cartridge back to the marked position.

c) After cartridge loading, push the piston release forward until it stops. (Fig. 4)
1) Push the piston release forward.
2) The piston will advance to the upper side.

2. DISPENSING
a) Remove the cartridge cover. (Fig. 5)
b) Depress the lever to dispense the required amount of paste onto the mixing pad.
Note:
1) Before dispensing the first material from a new cartridge or when dispensing after a long interval, bleed the paste in order to prevent the incorporation of air bubbles into the material.
2) The adjusting guide can be moved forwards or backwards to change the quantity of material to be dispensed. Position of adjusting guide for lining. (Fig. 6)
3) While depressing the lever, level and cut off the extruded material from the tips of the cartridge by moving the cartridge and dispenser to an upright position on the mixing pad. (Fig. 7)
1. Level and cut off material.
2. Depressing the lever.
When the lever is released after dispensing, residual air at the tips are retracted into the cartridge. Replace the cartridge cover.
Note:
1) If any surplus paste is retained on the cartridge after dispensing, remove with gauze, etc.
2) The amount of residual material in the cartridge can be confirmed by the position of the sliding block on the underside of the dispenser. As the cartridge has a translucent tip on one of the barrels, the residual amount can also be confirmed by the position of the inner plunger seen through the cartridge tip.
3) Do not drop the dispenser with the cartridge. Otherwise the cartridge may be damaged.

3. MIXING
After dispensing, incorporate and spread the material on a thin layer on the mixing pad using a plastic spatula. Mix thoroughly, with lapping strokes, for 10 seconds. Be careful not to incorporate air bubbles.
Note:
1) The working time is 2 minutes 15 seconds from the start of mixing at 23°C (73°F). Higher temperatures will shorten working time.
2) Do not mix paste with other materials.

4. RESTORATIVE TECHNIQUE
a) Prepare tooth using standard techniques.
Extensive mechanical retention is not necessary.
b) Wash and dry but DO NOT DESICCATE. For pulp capping use calcium hydroxide.
c) Mix the cement (as above).
d) Remove surface moisture, but DO NOT DESICCATE.
e) Transfer cement to the preparation using a syringe or suitable placement instrument, covering dentine up to the dentino-enamel junction.
f) Light cure with a halogen, plasma arc or LED light curing device*. Place light source as closely as possible to the cement surface.
Note:
If application exceeds 2 mm in depth, use a layering technique.

g) Use standard standard composite or amalgam restorative techniques.
*Irradiation time:
Halogen/LED 20 sec.
Plasma arc 10 sec.
G-Light 20 sec.

Cleaning / Sterilization of Paste Pak Dispenser
1. Clean the dispenser and the adjusting guide with gauze soaked in alcohol.
2. The dispenser may be steam autoclaved at 121-135°C (250-275°F) for 10-30 minutes or disinfected using alcohol. (Please remove the adjusting guide and the cartridge before autoclaving the dispenser.)
The use of various chemical disinfectants / sterilants may cause damage to the dispenser and therefore is not recommended.
Note:
After autoclaving, the piston release of the dispenser may be sticky and difficult to operate by fingers. In such a case, slide the piston release forward until it stops. Apply a very small quantity of petroleum jelly onto the entire surface of the piston.
Do not apply petroleum jelly. Then move the piston release back and forth a few times to ensure smooth movement.

SHADE
Universal

STORAGE
Recommended for optimal performance, store in a cool and dark place (4-25°C) (39.2-77.0°F).

PACKAGE
Paste Pak Cartridge (7.0 g / 4.7 mL) (1) with mixing pad (No. 22)
Option: Paste Pak Dispenser (1)

CAUTION
1. In case of contact with oral tissues or skin, remove immediately with cotton, gauze or a sponge soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. In rare cases metal may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of Paste Pak Dispenser (nickel-plated aluminum) and refer to a physician.
4. Store Paste Pak Dispenser away from chemicals such as iodine tincture containing halogen element.
5. Otherwise the dispenser may discolor.
6. Persons wearing protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
7. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:
https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental
In this way you will contribute to improve the safety of this product.

Manufactured by:
GC CORPORATION
76-1, Hatanuma-cho, Tabatah-ku, Tokyo 146-8585, Japan
RESPONSIBLE MANUFACTURER IN CANADA:
GC AMERICA INC.
3727 West 127th Street, Aisle, IL 60603 U.S.A.
3031
DISTRIBUTED BY:
GC CORPORATION
76-1 Hatanuma-cho, Tabatah-ku, Tokyo 146-8585, Japan

EU: GC EUROPE NV
Sie die richtige Adresse, telefonieren unter 00 33, B-3001 Leuven, Belgium. TEL: +32 16 74 10 00
GC AMERICA INC.
3727 West 127th Street, Aisle, IL 60603 U.S.A.
TEL: +1 708-587-0000
www.gcamerica.com

GC SOUTH AMERICA
Rua Heliodora, 309, Santana - São Paulo, SP, BRASIL.
CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 07.812.888/0001-00
RESP. TEC: Mayra de Satis Ribeiro - CROSP 105.982

GC ASIA DENTAL PTE. LTD.
5 Tampines Central 1, #06-01 Tampines Plaza, Singapore 529541
Tel: +65 6546 7588

GC AUSTRALASIA DENTAL PTY. LTD.
1753 Botany Rd, Bankstown NSW 2019, Australia
TEL: +61 2 9391 8200

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