

# Quick guide for foot control S-NW


## CAN-Dongle



### Pairing

The foot control S-NW and the CAN dongle are paired by default.

### Connecting CAN dongle

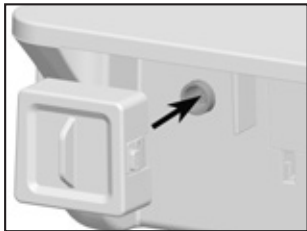


❶ Plug in the CAN dongle.



Pay attention to the positioning!

### Removing CAN dongle

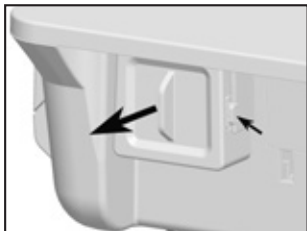

❷ Press the side lock and remove the CAN dongle.

### CAN dongle activated



Icon visible on display

- > CAN dongle inserted
- > Control unit switched on
- > Foot control actuated


## SPI-Dongle



### Pairing

The foot control S-NW and the CAN dongle are paired by default.

### Connecting and disconnecting the SPI dongle

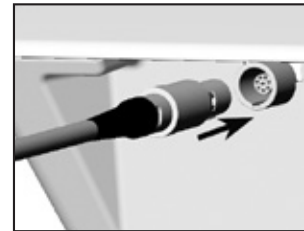

Pay attention to the positioning!

❶ Plug in the SPI dongle or disconnect the SPI dongle from the control unit.



❷ Attach the SPI dongle to the irrigant support or remove the SPI dongle from the irrigant support.



### GREEN – SPI dongle activated

LED on if the SPI dongle is connected and the control unit is switched on.

### ORANGE – battery

LED flashes if the batteries on the foot control need to be replaced.

### BLUE – pairing



# Quick guide for foot control S-NW

