



# HOFFMANN'S GUSSWACHS

blau/elfenbein, normalhart

---

## Gebrauchsanweisung für den Zahnarzt / Zahntechniker

### Zusammensetzung:

Gusswachs enthält: Paraffin, Carnaubawachs, Dammarharz, Stearin

### Anwendungsgebiete:

blau: Formen von Modellen für Goldgussfüllungen

elfenbein: Formen von Modellen für Kunststoffarbeiten

Gegenanzeigen: nicht bekannt

Nebenwirkungen: nicht bekannt

Wechselwirkungen mit anderen Mitteln: nicht bekannt

### Verarbeitungsempfehlung:

Gusswachs wird mit Hilfe des Bunsenbrenners erwärmt, auf das Modell getropft und in Zahnform modelliert.

Packungsgröße: 20 Stangen - 30g

Hoffmann's Gusswachs

- nach Ablauf des Verfalldatums nicht mehr anwenden

---

**HOFFMANN DENTAL MANUFAKTUR GMBH**, Wangenheimstraße 37/39,  
14193 Berlin, Germany, [www.hoffmann-dental.com](http://www.hoffmann-dental.com)



# HOFFMANN'S CASTING WAX

blue/ivory, normal hardness

---

## Instruction for the dentist/ dental technician

### Composition:

casting wax contains: paraffin, carnauba wax, dammar resin, stearin

### Area of Application:

blue: Moulding of models for gold casting fillings

ivory: Moulding of models for composite work

Contra-indications: No known contra-indications

Side effects: No known side effects

Interactions with other substances: No known interactions

### Processing Recommendation:

Soften casting wax with a Bunsen burner. Drip onto model and shape.

Package Size: 20 sticks - 30 g

### Hoffmann's Casting Wax

- Do not use after the expiration date

---

**HOFFMANN DENTAL MANUFAKTUR GMBH**, Wangenheimstraße 37/39,  
14193 Berlin, Germany, [www.hoffmann-dental.com](http://www.hoffmann-dental.com)