Hera AM 99

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DE Gebrauchsinformation

GB Instructions for use

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Distributed in USA/
Canada exclusively by:
Kulzer, LLC
4315 South Lafayette Blvd.
South Bend, IN 46614-2517
1-800-431-1785

Caution: Federal law restricts
this device to sale by or on the
order of a dental professional.

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Hera AM 99

(GB) Instructions for use

Pickling agent for precious metal dental alloys

With the ideal pickling agent in powder form oxides and other impurities can be easily and safely removed from precious gold casting alloys.

Mixing ratio:

To mix the solution use the content of 1 bag (65 g) to 250 ml of warm water in a ceramic-, glas- or acid-resistant container.

Warnings:

When using Hera AM 99 solution be sure to observe the safety regulations for handling acids.

Warning

Hazard statements

Causes skin irritation. Causes serious eye irritation. Harmful to aquatic life with long lasting effects.

Precautionary statements

Wear protective gloves / protective clothing / eye protection / face protection. Avoid release to the environment. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Specific treatment (see on this label). Take off contaminated clothing and wash before reuse. **If skin irritation occurs:** Get medical advice / attention.