INSTRUMENTS

GREEN&CLEAN ID N

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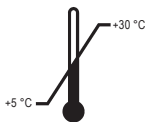
METASYS

GREEN&CLEAN ID N

DE Gefahr	SK Nebezpečstvo
EN Danger	SL Nevarno
FR Danger	HR Opasnot
IT Pericolo	HU Veszély
ES Peligro	ET Ettevaatust
PT Perigo	LV Bīstami
NL Gevaarlijk	LT Pavojinga
FI Vaara	RO Pericol
NO Faretype	MY Bahaya
DA Farlig	IS Hætta
SV Fara	HE הַנַּחֲטָה
EL Κίνδυνος	KA საფრთხე
PL Niebezpieczeństwo	ZH 危險
CS Nebezpečí	AR خطر

C €0044

Produktidentifikatoren / product identifiers:

Ethandiol/ethanediol [107-21-1], 2-Amino-ethanol [141-43-5],
N,N-Didecyl-N-methyl-poly-(oxethyl)-ammoniumpropionat [94667-33-1],
N-(-3-Aminopropyl)-N-docecylpropan-1,3,-diamin [2372-82-9]

Artikelnummern GREEN&CLEAN ID N Order numbers GREEN&CLEAN ID N

Artikelnummern | Order numbers

REF 60030033	4 x 500 ml Flaschen + Dosierbecher 4 x 500 ml bottles + measuring beaker
REF 60030034	1 x Kanister 5 l 1 x canister 5 l
REF 40200003	Dosierbecher für GREEN&CLEAN ID N Measuring beaker for GREEN&CLEAN ID N
REF 40200004	Auslaufhahn 5 l Kanister Water tap 5 l canister

Dosiertabelle Dosing table

Liter	1,0%	2,0%	7,5%
1 l	10 ml	20 ml	75 ml
2 l	20 ml	40 ml	150 ml
3 l	30 ml	60 ml	225 ml
4 l	40 ml	80 ml	300 ml
5 l	50 ml	100 ml	375 ml

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** For optimum results we recommend using a 2% solution with a contact time of 30 minutes or a 1% solution with a contact time of 60 minutes.

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EN GREEN&CLEAN ID N

Highly effective concentrate for the disinfecting preliminary cleaning of invasive and non-invasive instruments, including surgical instruments and endoscopes. Suitable for use in ultrasonic cleaners and for heat-sensitive and -insensitive instruments. An added corrosion inhibitor provides further protection for your instruments. Aldehyde- and phenol-free.

Use restriction: GREEN&CLEAN ID N is NOT suitable for the final disinfection or sterilization of surgical instruments or endoscopes. Further reprocessing steps must be followed including rinsing, washing and final disinfection/sterilization in accordance with the instrument manufacturer's instructions.

Material Compatibility: Samples of typical materials use for medical devices which were tested for material compatibility: Anodized aluminium, Aluminium coated with powder technology, Nickel plated mild steel, Polished martensitic steel, Stainless steel coated with gold, Polyethylene, Polymethacrylmethacrylate, Composite material from tungsten carbide and nickel, Polyvinylchloride flooring, Two types of butyl rubber, Optical glasses made from polycarbonate, Optical glasses made from silicate. Product test concentration: 3.0 %. Test conditions: submersion of material samples at 20 °C for up to 19 days.

Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, HSV-1): 2% dilution: 30 min.; bactericide, fungicide (C. albicans): 0.25%: 30 min., 1.5%: 5 min.; tuberculocide: 1%: 1 hour, 2%: 15 min., 7.5%: 5 min.

Application: **1** + **2** To prepare a ready-to-use solution, mix GREEN&CLEAN ID N with cold water. Please refer to the information in the dosing table. For optimal efficacy, we recommend using a 2% solution (= 20 ml concentrate + 1 liter of water) with an action time of 30 minutes. The solution has a standing time of 7 days; however, it must be replaced immediately when heavily contaminated.

3 Remove coarse contaminants from instruments with a suitable soft sponge or brush before immersion.

4 Immerse instruments completely and carefully in the solution to prevent forming of air bubbles. **5** Cover the disinfection tray and soak for the recommended action time. **6** After soaking, remove instruments from the solution and rinse thoroughly with water. Further reprocessing must be performed according to the instrument manufacturer's instructions.

DANGER: Harmful if swallowed. Causes severe skin burns and eye damage. Harmful if inhaled. Very toxic to aquatic life with long lasting effects. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store locked up.

Composition: Preparation made from quaternary ammonium compounds, alkylamine, non-ionic tensides, corrosion inhibitors and water

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**METASYS Medizintechnik GmbH**