SPECIAL AREAS

GREEN&CLEAN M2

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METASYS

GREEN&CLEAN M2

DE Gefahr
EN Danger
FR Danger
IT Pericolo
ES Peligro
PT Perigo
NL Gevaarlijk
FI Vaara
NO Faretype
DA Farlig
SV Fara
EL Κίνδυνος
PL Niebezpieczeństwo

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EN GREEN&CLEAN M2

Dual, actively defoaming concentrate for the daily cleaning, disinfection and deodorisation of suction systems and amalgam separators.

Spectrum of efficacy/action times: HBV/HIV/HCV: 1% dilution: 1 hour; bactericide*, fungicide (C. albicans)*: 1% dilution: 15 minutes; tuberculocide*: 1% dilution: 4 hours. (*under high load)

Refilling and use of the GREEN&CLEAN M2 dispenser: Each refill set contains the same amount of pouches of GREEN&CLEAN M2-red as well as of GREEN&CLEAN M2-green. Always alternate the colours, meaning that once you have used up GREEN&CLEAN M2-red in the dispenser, refill it with GREEN&CLEAN M2-green and vice versa. **1** Remove the dosing unit from the dispenser. Turn the empty container clockwise to remove it from the pump head. **2** Pour the concentrate into the dispenser. **3** Turn the filled container counterclockwise until it stops at the pump head. Put the dosing unit back into the dispenser.

Daily use: It is recommended to clean and disinfect the dental unit twice a day. After surgical procedures an additional cleaning should be carried out. **4** Dosage: Press dispenser pump twice (= 6 ml of GREEN&CLEAN M2 concentrate). Warning: Overdosing may lead to malfunctions! **5** Mixing: Take the mixing container and fill it with water up to the indicator (= 600 ml of 1% solution). **6 + 7** Disinfecting: Hold small suction tube on opening no. 1 until air is sucked in (= 200 ml). Hold large suction tube on opening no. 2 until air is sucked in (= 200 ml). **8** Cleaning and disinfecting the spittoon: Pour the remaining 200 ml from the mixing container into the spittoon bowl. Do not rinse (wait for action time).

To avoid any adverse chemical reactions, please do not use any other cleaning agents or disinfectants to clean either the spittoon bowl or the suction system. After every treatment, rinse the suction system with large quantities of cold water. Make sure that GREEN&CLEAN M2 is only used after rinsing with cold water; otherwise, the active chemicals would be flushed out.

DANGER: Harmful if swallowed. Causes severe skin burns and eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Very toxic to aquatic life. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. **IF ON SKIN (or hair):** Take off immediately all contaminated clothing. Rinse skin with water/shower. **IF INHALED:** Remove person to fresh air and keep at rest in a position comfortable for breathing. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If experiencing respiratory symptoms: Call a

POISON CENTER or doctor. Store locked up.

Composition GREEN&CLEAN M2 green: quaternary ammonium compounds, alkylamine

Composition GREEN&CLEAN M2 red: quaternary ammonium compounds, alkylamine, guanidinium compounds

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METASYS Medizintechnik GmbH