
GREEN&CLEAN MT

METASYS

DE	Achtung
EN	Warning
FR	Attention
IT	Attenzione
ES	Atención
PT	Atenção
NL	Waarschuwing
SV	Varning
FI	Varoitus

NO	Advarsel
DA	Advarsel
EL	Προσοχή
PL	Uwaga
CS	Varování
SK	Pozor
SL	Pozor
HR	Pozor
HU	Figyelem
ET	Hoiatus
LV	Uzmanību
LT	Atsargiai
IS	Varúð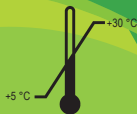

Extragroße Desinfektionstücher
Extra-large disinfectant wipes
Teli disinfettanti extra grandi
Suurikokoinen desinfectiopyyhe

70

Desinfektionstücher
 disinfecting wipes
 teli disinfettanti
 lingettes de désinfection

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EN Extra-large disinfecting wipes. Alcoholic disinfectant solution for disinfecting large surfaces of medical devices. Spectrum of efficacy/action times: limited virucidal (e.g. HBV/HCV/HIV, HSV-1): 30 seconds; bactericide, fungicide (C. albicans): 1 minute; tuberculocide: 5 minutes. Application: 1. Remove lid of dispenser bucket (security seal), place wipes inside. 2. Pour 750 ml from the canister or one bottle of the disinfectant solution GREEN&CLEAN SD evenly over the wipes. 3. Pull first wipe from the inside of the roll through the opening and close dispenser bucket. 4. Write the batch number and expiry date of the disinfectant solution on the enclosed label, fill in the rest of the information completely, and stick it onto the lid of the dispenser. 5. Ready to use as soon as all the liquid has been soaked up. WARNING: Flammable liquid and vapour. Causes serious eye irritation. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Composition: 45% ethanol, 15% 2-propanol, 0.2% quaternary ammonium compounds, 0.2% glyoxal

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