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## Instructions for use English

**Definition**  
Inspiro Nano-Hybrid Composite is a light cured, radiopaque, highly filled homogenous nano-hybrid composite filling material for restorations that meets the highest physico-mechanical and esthetic standards.

**The system consists of 2 components:**

Dentin „Body“ (B) in 6 different intensity levels (shades: B10 to B15) and Enamel „Skin“ (S) in 5 shades: Skin Bleached (SB), Skin White (SW), Skin Neutral (SN), Skin Transparent (ST), Skin Ivory (SI).

**Composition**

- Barium dental glass
- Nano-particles
- Bis-GMA based resin matrix
- Pigments
- Additives
- Catalyst

**Technical data**

Filler ratio: 82% by weight or 65% by volume. The range of inorganic filler particle size is from 0.02 to 3 µm.

**Indications**

**Direct applications:**

- Filling of cavity classes I, II, III, IV, V
- All other types of direct restorations (e.g., form corrections, diastema closure, etc. ...)

**Indirect applications:**

- Inlays
- Onlays
- Veneers
- Cementation of edelweiss DIRECT VENEAR & OCCLUSION VD

Dentin „Body“ (B) is specially indicated for: Reconstruction of natural dentin

Enamel „Skin“ (S) is specially indicated for: Reconstruction of natural enamel

**Recommended application**

- Select the shade using the Inspiro Body & Skin Shade Guide.
- Provide appropriate isolation. It is recommended to use a dental dam.
- Prepare the cavity according to the general requirements for adhesive techniques.
- Clean the cavity using water spray.
- Dry the cavity using air spray.
- Apply a pulp protection (e.g. calcium hydroxide), if necessary; then, only cover areas close to the pulp and apply a pressure-resistant cement.
- Apply matrix / interproximal dental wedge if needed.
- Condition the tooth and apply the selected bonding agent according to manufacturer's instruction
- Apply Inspiro Nano-Hybrid Restorative Composite in layers of max. 2mm.
- Polymerize each layer for 20s using a light output intensity of at least 500mW/cm<sup>2</sup>. With a light output intensity of 1100mW/cm<sup>2</sup>, the individual layers can be cured in 10s. When using a metal matrix (e.g. OpraMatrix), also polymerize from the buccal or lingual/palatal sides after removing the matrix.
- Finish the restoration using suitable finishing instruments or fine grit diamonds.
- Check the occlusion.
- Polish using silicone polishers as well as polishing discs and strips.

**Essential notes**

- Light curing units should have an output of 450nm and should be checked regularly. The curing light output intensity should be at least 500mW/cm<sup>2</sup>. Place the curing light as close as possible to the restorative material.
- Do not remove the oxygen-inhibition layer, since this is needed to create a sufficient bond to the next layer.
- Follow the instructions for use for all materials used together with Inspiro Nano-Hybrid Composite.

**Contraindications**

Application of Inspiro Nano-Hybrid Composite is contraindicated if a dry working area or the recommended application technique is not possible, if the pulp is exposed, or if there are any known allergies to any components in the Inspiro Nano-Hybrid Composite.

**Side effects**

Side effects are not known to date. In individual cases, hypersensitivity to components of the material cannot be excluded. Avoid contact of the paste with the skin. If accidental contact occurs, wash immediately using soap and water. In case of contact with eyes, rinse immediately using plenty of water, and consult a physician.

**Interactions**

Avoid materials containing eugenol, moisture or oily air, since they can inhibit polymerization at the contact area. The use of cationic mouthwashes, plaque indicators or chlorhexidine may lead to discolorations.

**Classification**

Inspiro Nano-Hybrid Composite fulfills the requirements of ISO 4049:2000. The specified curing depth of 2mm (20s) for Inspiro Nano-Hybrid Composite is prepared according to ISO 4049:2000. The specified, layer thickness layer of 2mm and light curing time of 20s for each layer is recommended for all cases.

**Storage**

Store at normal room temperature (15-25°C / 59-77°F). Please close syringe/composite tip immediately after each use to prevent premature setting from ambient light.

**Shelf life and marking**

Expiration date and the lot number are marked on the syringes as well as the packaging. Do not use after the date of expiration.

**Note**

Only to be supplied to dentists and dental laboratories or upon their request. Keep out of the reach of children! For dental use only!

**Important**

To ensure for optimal clinical results, we recommend that all Inspiro components in this system are used in conjunction with each other.

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