# LuxaPick-up

DMG Chemisch-Pharmazeutische Fabrik GmbH  
Elbgaustraße 248 · 22547 Hamburg · Germany  
www.dmg-dental.com  
092008/#1\_2016-02

CE 0482

DE	Gebrauchsinformation	DA	Brugsanvisning
EN	Instructions for use	SV	Användningsinformation
FR	Mode d'emploi	PL	Wskazówki dotyczące użycia
IT	Istruzioni per l'uso	RU	Инструкция по применению
ES	Instrucciones de uso	ZH	使用说明
PT	Instruções de uso	JA	取扱説明書
NL	Gebruiksaanwijzing	KO	사용 설명서

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Deutsch

### Produktbeschreibung

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## English

### Product description

LuxaPick-up is a self-curing relining composite based on multifunctional methacrylates. The material is available in the Automix cartridge.

### Indications

- Fixation of secondary parts and connection elements in dentures/prostheses
- Relining of partial or/and full dentures
- Modification of intermediate dentures
- Repair of prostheses
- Fabrication of gingival masks
- Customization of temporaries to create natural red-white esthetics, especially in the case of implant-borne dentures

### Contraindications

Do not use the material in cases of allergies to any of the ingredients or, as can occur in rare cases, in the event of contact allergies.

### Basic safety information

- For dental use only.
- Keep out of the reach of children.
- Avoid contact with the skin! In the event of accidental skin contact, immediately wash the affected area thoroughly with soap and water.
- Avoid eye contact! In the event of accidental contact with the eyes, immediately rinse thoroughly with plenty of water and consult a physician if necessary.

### Side effects

There are no known systemic side effects to date. Hypersensitivity to components of the material or contact allergies cannot be excluded in individual cases.

### Timing

0:00–0:45 min	Place into the mouth.
2:00 –3:00 min	Remove from the mouth.
6:00–7:00 min*	Setting completed

\*The indicated times apply at a room temperature of 23°C/73°F and a normal relative humidity of 50%. Higher temperatures shorten these times while lower temperatures increase them.

### Recommended use

#### Prepare secondary parts and connection elements

- Mill a circular undercut groove into the base of the prosthesis in order to support mechanical retention.
- Check whether the secondary part/connection element fits into the corresponding groove avoiding touching the base of the prosthesis.
- Block out undercuts on the secondary part/connection element, e.g. with silicone impression materials or wax.

#### Prepare defective prostheses

- Grind cracks or abrade broken sub-areas.

**Note:** Only repair small breaks or cracks temporarily. Long cracks which affect the stability of the restoration or complete breaks should be repaired conventionally.

#### Using LuxaPick-up intraorally

- Abrade areas which are to be joined using LuxaPick-up.
- Apply Luxatemp-Glaze & Bond to the abraded areas according to the manufacturer's instructions and allow to work.
- Light-cure Luxatemp-Glaze & Bond according to the manufacturer's information.

**Note:** Ensure that the conditioned surfaces are not contaminated with saliva or blood.

- Apply LuxaPick-up in the restoration and passively fit and fix into place using moderate pressure no more than 45 seconds after mixing has begun.
- Remove the restoration from the mouth during the elastic phase, i.e. in the time period from 2:00 to 3:00 min after the start of mixing.
- Leave LuxaPick-up to cure fully.
- Trim junctions and grind down excess. Emulate the anatomic shape using a rotating instrument.
- Polish abraded areas using a suitable instrument or seal with a suitable conditioner and then cure according to the manufacturer's instructions.

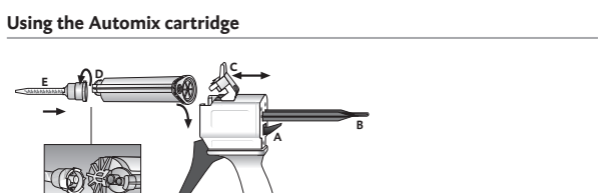
#### Using LuxaPick-up extraorally

- Take intraoral functional impressions of the resorbed areas using the prosthesis and silicone impression materials.
- Plaster in the prosthesis including the functional impressions in a fixing device. As soon as the plaster has partly hardened, create a counter for later fixation.
- Allow the plaster to harden completely according to manufacturer's instructions.
- Open the fixing device and remove the prosthesis.
- Remove the residuals of the functional impressions from model and prosthesis.
- Abrade areas of the prosthesis which are to be joined using LuxaPick-up.
- Apply Luxatemp-Glaze & Bond to the abraded areas according to the manufacturer's instructions and allow to light cure.

**Note:** Ensure that the conditioned surfaces are not contaminated with liquids (e.g. saliva or blood).

- Carefully paint the model and the counter with plaster plastic insulation.
- Fix the prosthesis on the plaster counter (e.g. using sticking wax).
- Apply LuxaPick-up to the areas to be relined and assemble in the fixing device within 45 s after beginning to mix.
- Remove coarse excess using a spatula, then polymerize in a pressure pot at 45°C for 10 min.
- Deflask the prosthesis after removal and cooling, trim junctions and grind down any excess using a plastic milling cutter.
- Polish abraded areas using a suitable instrument or seal with a suitable conditioner and cure according to the manufacturer's instructions.

#### Using the Automix cartridge

#### Inserting the cartridge

- Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] all the way.
- Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

#### Attaching a mixing tip

- Remove the cartridge cap [D] or used mixing tip after turning it 90° counterclockwise, and discard it.
- Attach a new mixing tip [E].

**Note:** In order to achieve an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available in refill packs.

**Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.

- Twist the mixing tip clockwise through 90° to lock it.

## Applying material

- ▶ When using a cartridge for the first time, squeeze out an approximately pea-sized amount of material and discard it.
- ▶ Mix the material in the mixing tip using the lever [F] on the Automix-Dispenser and apply directly.

**Note:** After application, leave the used mixing tip on the cartridge as a seal!

## Removing the cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] all the way.
2. Lift up the plastic lever [C] and remove the cartridge.

## Disinfection of the Automix-Dispenser

The Automix-Dispenser can be reused after disinfection, although it should be replaced if there are visible signs of wear.

Disinfection can be carried out using a commercially available dipping disinfectant. DMG recommends using only disinfectant solutions listed by the RKI (Robert Koch Institute). The Automix-Dispenser is also autoclavable.

## Storage and shelf life

- ▶ Store at room temperature (15–25°C/59–77°F) in a dry place.
- ▶ Do not store in the refrigerator.
- ▶ Keep away from direct sunlight.
- ▶ Do not use after the expiration date.

## Composition

Glass filler material in a matrix of multifunctional methacrylates; catalysts, stabilizers, and additives. Does not contain methyl methacrylate or peroxides. Filler content: 47 wt.% = 26 vol.%. (0.02 to 1.5 µm)

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