Periotome

| | |
|------|------|
| L1S | 1 mm |
| L2S | 2 mm |
| L3C | 3 mm |
| L3S | 3 mm |
| L3CA | 3 mm |
| L3IC | 3 mm |
| L5C | 5 mm |
| L5S | 5 mm |

Short Periotome

| | |
|------|------|
| S2S | 2 mm |
| S3C | 3 mm |
| S3S | 3 mm |
| S3CA | 3 mm |
| S5S | 5 mm |

Dual Edge Periotome

| | |
|-----|----------|
| DE3 | 3/1.5 mm |
| DE5 | 5/3 mm |

Forte Elevator

| | |
|-----|---------|
| F25 | 2.50 mm |
| F32 | 3.25 mm |
| F40 | 4.00 mm |

**Luxator Titanium
L2S TiN, DE3 TiN,
L3A TiN, L3S TiN**

Thank you for choosing Luxator®. The guarantee will not cover instrument with damaged tip caused by incorrect use.

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DIRECTA INC
64 Barnabas Road, Unit 3
Newtown, CT 06470, United States
203.491.2273

Luxator® is a registered trademark own by Directa.

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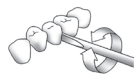
EN INSTRUCTION FOR USE

Intended use: Luxator® Extraction Instruments are specially designed for extraction of teeth.

Luxator® Periotome, Luxator® Short Periotome, Luxator® Titanium, Luxator® Dual Edge Periotome (1S, 2S, 3C, 3S, 3CA, 3IC, 5C, 5S, DE3, DE5)

Caution

The tips of all types of Luxator Periotomes are extremely fine and sharp and can be damaged if exposed to excessive forces. These tips are not designed for strong forces.

The handle is ergonomically designed for optimal facility and control.

- Place your index finger along the axis of the blade, marking the depth of required penetration. Thus the index finger serves as an occlusal stop.
- Place the tip axially in the periodontal space.
- Proceed with a slight twisting action to gently drive the tip into the socket. The thin and sharp tip cuts ligaments while compressing the bone and raising the tooth gently from the socket.
- Continue to two-thirds of the depth of root keeping close to the root wall. Avoid snagging of the root. If root remains firm, repeat on the other side.

Luxator® Forte Elevator (F25, F32, F32C, F40)

To be used for elevation.

Note TiN. Do not sharpen this instrument as it will damage the titanium coating.

Sharpening of instrument

To obtain optimal result, sharpen the instrument on a regular basis using a Luxator sharpening stone. Sharpen the instrument after thorough cleaning, before sterilization.

Sterilisation

Recommended sterilisation method: steam autoclave at 134°C.
Other suitable sterilization method: dry heat maximum 180°C.

Material

Stainless Steel, Polyamid PA6.6.
Luxator Titanium: titanium nitride coated stainless steel.

Examples of use

Periotome/Titanium/Short Periotome (shaft 10 mm shorter than Periotome)

L1S * Apical and/or Interproximal
L2S, S2S Apical and/or Interproximal
L3C, S3C General use/Starter
L3S, S3S, L5S, S5S Interproximal
L3CA, S3CA Lingual and/or Distal – Molars
L3IC General, Lingual and/or Distal
L5C Large molar roots – general use

Dual Edge Periotome

DE3 * DE5 Deep fractured and decayed roots

Forte Elevator

F25, F32, F32C, F40 Elevation

*) Extremely sharp and delicate, use with caution!

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