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Light curing composite resin for the fabrication of custom trays and base plates

GB/IR/US/CA

1. Name and description of the medical device
TOPDENT LUXTRAY consists of a matrix of polyfunctional methacrylates and inorganic fillers. The resin is blue or pink.
The thickness of the TOPDENT LUX-TRAY tray material is 2.2 mm.

TOPDENT LUXTRAY has

- less shrinkage,
- resistance to water absorption
- high dimensional stability.

2. Indication

Light-cure impression tray material for the fabrication of individual trays, bite plates and base plates for needle-point tracing.

3. Application

The plaster model must be insulated with alginate insulation (TD MasterCast separator) before using TOPDENT LUX-TRAY. The cover-vlies from the wafer has to be removed, then TOPDENT LUX-TRAY is moulded onto the model. The tray can be cut to the desired rim. Use excess material to form the handle. The wafer is polymerised in an UVA- or blue-light unit (e.a. TOPLIGHT MINI) for 5 minutes. The wafer is taken off the model and the reverse side of the tray is cured for another 5 minutes. Individual tray can be trimmed immediately. After polymerising the inhibition layer caused by the airoxygen is easily removed by a tissue moistened with an alcoholic cleaner. Use a dust collector for trimming with carbide burrs.

4. Indication of risk

TOPDENT LUX-TRAY contains monomers for polymerisation (e.g. methacrylic esters) which can cause skin allergies, especially with sensitive patients. Do not use this product where allergic reactions are observed, or for patients with methacrylate/acrylate-based resin allergy history.

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GB/IR/US/CA

1. Name and description of the medical device

Light curing resin in form of contoured wafers for the fabrication of base plates

TOPDENT LUX-TRAY Base plates consists of a matrix of polyfunctional methacrylates and inorganic fillers.

TOPDENT LUX-TRAY Base plates has

- less shrinkage
- resistance to water absorption
- high dimensional stability
- perfect thickness of the plates, 1.4 mm

2. Indication

For the fabrication of bite plates, plates for wax up and base plates for needle-point tracing.

3. Application

After use, close the box directly to prevent a pre curing of the material inside.

The plaster model must be insulated with alginate insulation before using the Megatray Basisplatte material. The protecting layer at the wafer has to be removed, than applicate the resin plate onto the plaster model. The base plate can be cut to the desired rim. Remove excess material can be storage in the box for other plates or to form a handle. The base plate is polymerized in a UVA- or blue-light unit for 5 minutes. The base plate is taken off the model and the reverse side of the tray is cured for another 5 minutes.

After polymerization, the inhibition layer caused by the air oxygen is removed by a tissue moistened with an alcoholic cleaner easily. Use a dust extractor for trimming with carbide burs.

4. Warnings

TOPDENT LUX-TRAY base plate contains monomers for polymerization (e.g. methacrylic esters) which can cause skin allergies, especially with sensitive patients. Do not use this product where allergic reactions are observed, or for patients with methacrylate/acrylate-based resin allergy history.

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