GC MI Paste Plus

TOPICAL CREAM WITH CALCIUM, PHOSPHATE AND FLUORIDE

For use by or under the supervision of a dental professional in the recommended indications.

DESCRIPTION
GC MI Paste Plus is a water based cream containing RECALDENT™* with incorporated fluoride (CPP-ACP): Casein Phosphopeptide-Amorphous Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which means that in adult teeth it strengthens tooth enamel. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium, phosphate and fluoride.

IMPORTANT :
Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium, phosphate and fluoride

- Provides extra protection for teeth.
- Helps neutralize acid challenges from acidogenic bacteria in plaque.
- Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS
Casein phosphopeptides are derived from milk casein.
Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoteze preservatives.

DIRECTIONS FOR USE

INDICATIONS
a. In children in-office bleaching procedure.

b. After orthodontic scaling and planning.

c. Following professional tooth cleaning (PTC).

d. Hypersensitivity prevention and control.

e. An alternative means of applying fluoride topically in children aged 6 and above.

f. During orthodontic treatment.

g. For high risk caries patients.

h. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögren's syndrome.

i. For special needs adult patients.

In-office application:

- 1 Before placing the custom tray thoroughly under running water.
- 2 Apply a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
- 3 Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
- 4 Remove the tray.
- 5 Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to hold for as long as possible (another 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- 6 Ask patient to rinse to expectorate thoroughly and if possible avoid rinsing.
- 7 Ask GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- 8 Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1 If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with a cotton roll or pellet.

2 Apply a GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Cleaning Brush.

3 Leave GC MI Paste Plus undisturbed for a minimum of 5 minutes.

4 Instruct patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.

5 Ask patient to rinse to expectorate thoroughly and if possible avoid rinsing.

6 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

III. Day time application after tooth brushing as recommended by a dental professional:

1 Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth.

2 A pea size amount of the tooth surface using a clean dry finger or cotton tip.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional:

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12 years old. The material should be applied to the tooth surface using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Cleaning Brush or dental floss coated with GC MI Paste Plus.

2 Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

FLAVOURS
Strawberry (S), Melon (M), Vanilla (V), Mint (I), Tutti-Frutti (T)

STORAGE
Store in a cool dry low humidity area away from direct sunlight (8-25°C) (46-77°F). (Shelf life: 2 years from date of manufacture.)

PACKAGES
Tube containing 40g (35ml)

1 Assorted Package

1 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)

2 Single Flavour Package

1 x 40g of each flavor

Vanilla, Strawberry, Melon, Tutti-Frutti, Mint

3 Refill Package

1 x 40g tube in each flavour

NOTE

GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia. "RECALDENT" is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada and USA.

CAUTION

1 Always replace the cap after use and remove any residual paste or debris from the tube teeth or inside the cap.

2 It is not recommended to use GC MI Paste Plus on patients under the age of 6.

3 The overnight at home application is not recommended for children under 12.

4 In case of contact with eyes, flush immediately with water and seek medical attention.

5 If any angioneurotic symptoms are experienced, this may indicate sensitivity or allergy to the benzoteze preservatives, or to some other component of the product such as a flavouring agent. In this event, discontinue the use of the product and refer to a physician.

Last revised : 02/2011

GC MI Paste Plus is a water based cream containing RECALDENT™* with incorporated fluoride (CPP-ACP): Casein Phosphopeptide-Amorphous Calcium Phosphate Fluoride). The level of fluoride is 0.2% w/w (900ppm), which means that in adult teeth it strengthens tooth enamel. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localising bio-available calcium, phosphate and fluoride.

IMPORTANT :

Saliva will enhance the effectiveness of CPP-ACP and the flavour will help stimulate saliva flow. The longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

A topical paste containing bio-available calcium, phosphate and fluoride

- 1 Provides extra protection for teeth.
- 2 Helps neutralize acid challenges from acidogenic bacteria in plaque.
- 3 Helps neutralize acid challenges from other internal and external acid sources.

CONTRAINDICATIONS

Casein phosphopeptides are derived from milk casein.
Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzoteze preservatives.

DIRECTIONS FOR USE

INDICATIONS

a. In children in-office bleaching procedure.

b. After orthodontic scaling and planning.

c. Following professional tooth cleaning (PTC).

d. Hypersensitivity prevention and control.

e. An alternative means of applying fluoride topically in children aged 6 and above.

f. During orthodontic treatment.

g. For high risk caries patients.

h. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögren's syndrome.

i. For special needs adult patients.

In-office application:

- 1 Before placing the custom tray thoroughly under running water.
- 2 Apply a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
- 3 Leave the tray undisturbed in the mouth for a minimum of 3 minutes.
- 4 Remove the tray.
- 5 Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to hold for as long as possible (another 1-2 minutes) avoiding expectoration and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
- 6 Ask patient to rinse to expectorate thoroughly and if possible avoid rinsing.
- 7 Ask GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- 8 Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.

II. Non tray application:

1 If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with a cotton roll or pellet.

2 Apply a GC MI Paste Plus to the tooth surfaces using an application swab, gloved finger or in difficult interproximal areas using an Interproximal Cleaning Brush.

3 Leave GC MI Paste Plus undisturbed for a minimum of 5 minutes.

4 Instruct patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

5 Ask patient to rinse to expectorate thoroughly and if possible avoid rinsing.

6 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.

III. Day time application after tooth brushing as recommended by a dental professional:

1 Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth.

2 A pea size amount of the tooth surface using a clean dry finger or cotton tip.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing.

5 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional:

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12 years old. The material should be applied to the tooth surface using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Cleaning Brush or dental floss coated with GC MI Paste Plus.

2 Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing.

5 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional:

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12 years old. The material should be applied to the tooth surface using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Cleaning Brush or dental floss coated with GC MI Paste Plus.

2 Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing.

5 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional:

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12 years old. The material should be applied to the tooth surface using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Cleaning Brush or dental floss coated with GC MI Paste Plus.

2 Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.

The longer GC MI Paste Plus and saliva are maintained in the result.

4 Expectorate thoroughly and if possible avoid rinsing.

5 Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

IV. Night time application after tooth brushing as recommended by a dental professional:

The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12 years old. The material should be applied to the tooth surface using a clean dry finger or cotton tip.

For difficult areas (between the teeth) use an Interproximal Cleaning Brush or dental floss coated with GC MI Paste Plus.

2 Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.

3 Then use your tongue to tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectoration (spitting out) and delaying swallowing.</p

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DESCRIPTION
GC MI Paste Plus is a water based cream containing RECALDENT™ with incorporated CPP-ACP: Casein Phosphopeptide-Amorphous Calcium Phosphate Fluoride.

The level of fluoride is 0.2% w/w (900ppm), which approximates that at adult strength toothpastes. When CPP-ACP is applied in the oral environment, it will bind to biofilms, plaque, bacteria, hydroxyapatite and soft tissue localizing bio-available calcium, phosphate and fluoride.

IMPORTANT:
Saliva will enhance the effectiveness of CPP-ACP and the flavor will help stimulate the tongue. Longer CPP-ACP and saliva are maintained in the mouth, the more effective the result.

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1. Provides extra protection for teeth.
2. Helps neutralize acid challenges from acidogenic bacteria in plaque.
3. Helps neutralize acid challenges from other internal and external acidic sources.

CONTRAINDICATIONS

Do not use in patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzaldehyde preservatives.

DIRECTIONS FOR USE INDICATIONS

- a. Following in-office bleaching procedures.
- b. After Ultrasonic, hand scaling or root planing.
- c. Following professional tooth cleaning (P.T.C.).
- d. As an alternative method of applying fluoride topically in children aged 6 and above.
- e. During orthodontic treatment.
- f. For high risk caries patients.
- g. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögren's syndrome.
- h. For patients who need adult patients.

In-office application

- i. **Custom tray application**
 1. Before use rinse the custom tray thoroughly under running water.
 2. Extrude a generous layer of GC MI Paste Plus into the tray and apply to the upper and / or lower teeth.
 3. Leave undisturbed in the mouth for a minimum of 3 minutes.
 4. Remove the tray.
 5. Instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Instruct the patient to retain for as long as possible (another 1-2 minutes) avoiding expectation and delaying swallowing. The longer GC MI Paste Plus remains on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
 6. Any residual GC MI Paste Plus in the tray should be rinsed or brushed off under running water immediately after use.
 - ii. **Custom tray application**
 1. After rinsing remove any excess salivary on the tooth surface with a cotton roll or pellet. However, it is NOT necessary to dry the teeth with compressed air.
 2. Apply GC MI Paste Plus to the tooth surfaces using an Interproximal Tooth Cleaning Brush.
 3. Apply GC MI Paste Plus to the tooth surfaces using an Interproximal Tooth Cleaning Brush.
 4. Then instruct the patient to use the tongue to spread the remaining GC MI Paste Plus throughout the mouth. Request the patient to hold in the mouth for as long as possible (an additional 1-2 minutes) avoiding expectation and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 5. Reste vorsichtig ausspucken, dabei Spülung wenn möglich vermeiden. Eventuell zurückbleibende GC MI Paste Plus löst sich schrittweise selber auf. Der Patient soll fröhlestens 30 Minuten nach der Behandlung wieder essen oder Trinken.
- j. **Day time application after tooth brushing as recommended by a dental professional**
 1. Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
 2. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
 3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Advise the patient not to eat or drink for 30 minutes following application.
- k. **Night time application after tooth brushing as recommended by a dental professional**
 1. The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
 2. Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
 3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

At-home application

- l. **Day time application after tooth brushing as recommended by a dental professional**
 1. Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
 2. Leave GC MI Paste Plus on the teeth undisturbed for a minimum of 3 minutes.
 3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 4. Reste vorsichtig ausspucken, dabei Spülung wenn möglich vermeiden. Reste von GC MI Paste Plus können Probleme in den Mundhöhlen verbleiben. Nach der Anwendung 30 Minuten nicht essen oder trinken.
- m. **Night time application after tooth brushing as recommended by a dental professional**
 1. The overnight use is especially recommended for high-risk adult patients, but not recommended for children under 12.
 2. Apply a sufficient amount of GC MI Paste Plus to upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean dry finger or cotton tip. For difficult areas (between the teeth) use an Interproximal Tooth Cleaning Brush or dental floss coated with GC MI Paste Plus.
 3. Then use your tongue to spread the remaining GC MI Paste Plus throughout the mouth. Hold in the mouth for as long as possible (a further 1-2 minutes) avoiding expectation (spitting out) and delaying swallowing. The longer GC MI Paste Plus and saliva are maintained in the mouth, the more effective the result.
 4. Expectorate thoroughly and if possible avoid rinsing. Any GC MI Paste Plus remaining on the surface can be left to gradually dissipate. Do not eat or drink for 30 minutes following application.

FLAVORS

Strawberry, Melon, Vanilla, Mint, Tutti-Frutti

Recommended for optimal performance, store in a cool dry low humidity area away from direct sunlight (8-25°C, 46-47.7°F) (Shelf life: 2 years from date of manufacture)

PACKAGES

- Tube containing 40g (35ml).
1. Assorted Package
2. Single Flavor Package
10 x 40g in each flavor (Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)

2. Single Flavor Package
10 x 40g in each flavor (Vanilla, Strawberry, Melon, Tutti-Frutti, Mint)

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NOTE

GC MI Paste Plus with its unique CPP-ACP formulation was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia.

RECALDENT is a trademark used under license. CPP-ACP technology has related patents or pending patents in Australia, NZ, Europe, Canada and USA.

CAUTION

1. Always replace the cap after use and remove any residual paste or moisture around the tube neck or inside the cap.

2. It is not recommended to use GC MI Paste Plus on patients under the age of 6.

3. If contact with eyes, flush immediately with water and seek medical attention.

4. In case of spillage on clothes, wash off with water.

5. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzaldehyde preservatives or to some other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at: <http://www.geurope.com/>

They can also be obtained from your supplier.

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