Screw Posts are used to give retention for reconstruction of non vital teeth, where the root canal has been endodontically treated.

The gold plated screw posts should not be shortened or tampered with in any way that might take away part of the protective gold layer as this can cause the post to corrode. Titanium or stainless steel posts are not affected by this. It is of utmost importance that the root-canal has been prepared with a reamer of appropriate size and shape corresponding to the post to be used.

When using the reamers, start with a reamer number smaller than the diameter number desired and proceed from reamer number to reamer number step by step till desired number reamer can be used. Reamers are numbered 1 - 6 (7).

Reamers should be used at moderate r.p.m. speed and with a careful up- and down movement only.

If the precautions are not taken, reamer can get stuck and/ or break off in root canal.

Never use force when inserting post as this can cause root fracture.

Use appropriate cement to increase retention of post.

Screw posts should be used «Lege Artis» and with professional judgement.

Screw posts and reamers can be sterilized by all methods. Sterilization is recommended between interventions.

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