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P.U.M.A. hard

CE 0482

**Hersteller**

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg  
Telefon 040 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

**P.U.M.A. hard**

REF PUM5000 (P.U.M.A. hard, Systempackung)

REF PUK5010 (P.U.M.A. hard, Nachfüllpackung)

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REF PUM5000 (P.U.M.A. hard, system kit)

REF PUK5010 (P.U.M.A. hard, refill kit)

REF PUA5020 (P.U.M.A. hard ADHESIVE, refill kit)

**Description and indication**

P.U.M.A. hard is a cold-curing hard relining material on polymethacrylate basis for partial -, total - or interim prostheses (dentures).

P.U.M.A. hard shows an excellent long-term adhesion on dentures on methacrylate basis in combination with P.U.M.A. hard ADHESIVE. It can be used directly/chair-side or indirectly in the lab. P.U.M.A. hard is free of methyl methacrylate (PMMA). P.U.M.A. hard is delivered in KwikkMix®-System 10:1.

**Recommended procedure****1. Preparing the double chamber syringe**

First remove the original cap (do not re-use!) from the double chamber syringe. Every time as a check press out a small amount of material without mixing cannula. Then screw on the mixing cannula FANTESTIC® MIXING CANNULA MPN 2 (REF MIK1046) by turning clockwise by 90°. The double chamber syringe is ready for application.

**2. Preparation of the denture areas to be relined**

After a functional test (occlusion control) the areas to be relined (denture borders and/or denture base) will be roughened with a suitable bur, cleaned and dried.

Attention: Areas which should not be relined must be insulated, e. g. with vaseline. Anchors, clasps and attachments of partial dentures have to be blocked out with an easy flowing silicone or wax with a low melting point.

Then P.U.M.A. hard ADHESIVE (REF PUA5020) will be brushed extraorally as a thin coat on the denture areas and dried in the air for about 15 s. Do not contaminate the sticky oxygen inhibition layer, because this layer will be in adhesive contact to the relining material.

Attention: Store the P.U.M.A. hard ADHESIVE-bottle vertically. Methacrylate is volatile. Immediately close the bottle after use. The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

**3. Application of the relining material P.U.M.A. hard (chairside)**

Before application a small amount of material (size like rice corn to pea) should be pressed out and should be wasted immediately.

Apply the automatically mixed P.U.M.A. hard uniformly on the denture borders and/or denture base conditioned with P.U.M.A. hard ADHESIVE and insert the denture within 1 min into the mouth of the patient (working time at 23 °C (74 °F)).

Attention: Avoid that excess material will float behind the A-line (region from hard to soft palate) into the throat.

Then let the patient close his occlusion (ikp) for about 30 s and carry out active and passive functional excursions for about 1 min 30 s.

**3 a. Partial or complete dentures with undercuts**

Intraoral: Remove the denture from the mouth after about 3 min of starting the application. Remove the excess relining material with scalpel or scissors. Then introduce the denture for final occlusion and let it cure for about 2-3 min.

Extraoral: The complete curing can be accelerated in warm water or in a polymerizing device at 40 °C (104 °F) (to inhibit the oxygen inhibition layer, do not let it cure in the air).

**3 b. Partial or complete dentures without undercuts**

Remove the denture from the mouth after about 3 min of starting the application. Remove the excess relining material with scalpel or scissors. Then let the denture cure in the mouth for about 2-3 min. Optional the complete curing can be accelerated in warm water or in a polymerizing device at 40 °C (104 °F) (to inhibit the oxygen inhibition layer, do not let it cure in the air).

**4. Application of the relining material P.U.M.A. hard (labside)**

Embed the denture to be relined into a flask with counter. Then remove the denture from the flask and prepare as described under section 1 and section 2. Then isolate the model with an usual alginate based insulation. Apply the required automatically mixed P.U.M.A. hard uniformly on the denture borders and/or denture base to be relined. The layer of P.U.M.A. hard should be approximately 1 mm more than the base material has been ground off. Close flask with counter and press with light pressure for approximately 30 minutes (at 23 °C or 74 °F) or place flask in a pressure pot at 40-45 °C (104-113 °F) for approximately 10 minutes. Then deflask the relined denture and trim and polish as described under point 5.

**5. Finishing the relined denture**

The oxygen inhibition layer can be removed with alcohol. The relined areas can be finished with suitable instruments (burs, polishers and so on).

Attention: Do not breathe polishing dust. Use suitable mouth protective device or aspiration!

**Special storage and/or handling conditions**

Before first use screw off the original cap and dispose it. Before screwing on a new mixing cannula the double-chamber syringe should be tested if both pastes will be extruded equally.

Before every application a small amount of material (size like rice corn to pea) should be pressed out and should be disposed of immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Store the ADHESIVE-bottle vertically. Methacrylate is volatile. Immediately close the bottle after use. Ensure a good room ventilation. Methacrylate is highly flammable and irritant. The relined denture can be cleaned with commercial cleaners.

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Shelf life see label. Do not use after expiry date.

Do not store above 25 °C (77 °F)!

Avoid exposure to sunlight. Keep out of the reach of children!

The double chamber syringe should be stored horizontally (laying).

**Precautions**

Do not breathe polishing dust. Use suitable mouth protective device or aspiration!

Further on avoid contact of P.U.M.A. hard ADHESIVE and P.U.M.A. hard with skin or eyes. If there is an accidental contact with skin, flush immediately with soap and numerous amounts of water. If there is an accidental contact with eyes, flush immediately with numerous amounts of water and consult a physician if necessary.

P.U.M.A. hard is free of methyl methacrylate but contains other methacrylates. With susceptible patients, sensitization to P.U.M.A. hard cannot be excluded. If allergic reactions are observed, P.U.M.A. hard should not be used anymore. Do not use P.U.M.A. hard for patients with allergic reactions against acrylates. If swallowed accidentally, seek medical advice immediately.

**Technical data****P.U.M.A. hard ADHESIVE**

Time for drying P.U.M.A. hard ADHESIVE: about 15 s

**P.U.M.A. hard chairside**

Mixing time: auto-mixing

Working time incl. mixing time (23 °C (74 °F)): about 1 min

Subsequent recommended time in the mouth in closed occlusion (ikp): about 30 s

Subsequent recommended time in the mouth with excursion movements: about 1 min 30 s

Subsequent recommended curing time in the mouth: about 2-3 min

Time (start mixing - withdrawal from mouth): about 5-6 min

**P.U.M.A. hard labside**

Mixing time: auto-mixing

Working time incl. mixing time (23 °C (74 °F)): about 1 min

Subsequent recommended time, cuvette, 23 °C (74 °F): about 30 min

Subsequent recommended time pressure pot, 40 °C (104 °F): about 10 min

FANTESTIC is a registered trademark of R-dental Dentalerzeugnisse GmbH.

**Caution:** For dental use only.

**Warranty**

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

**Limitation of Liability**

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2019 R-dental Dentalerzeugnisse GmbH. All rights reserved.