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P.U.M.A. soft®

Hersteller:
R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

- REF PUM4000 P.U.M.A. soft®-Systempackung
- REF PUK4010 P.U.M.A. soft®-Nachfüllpackung (gingivafarben)
- REF PUK4015 P.U.M.A. soft®-Nachfüllpackung (transparent)

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- REF PUM4000 P.U.M.A. soft® System kit
REF PUK4010 P.U.M.A. soft® Refill (gingiva coloured)
REF PUK4015 P.U.M.A. soft® Refill (transparent)

Description and indication

P.U.M.A. soft® is a long-term, self-curing denture soft relining material based on vinyl polysiloxane base (A-silicones) for partial -, total - or temporary dentures.

P.U.M.A. soft® relieves pressure areas in the upper and lower jaw and can be used for compensating resiliencies of mucous membranes and for sealing of the A-Line in the case of adhesion problems and flabby ridges. The position of the upper and lower dentures are fastened by increased adhesion. With P.U.M.A. soft® sharp alveolar edges can be elastically covered. P.U.M.A. soft® is also suitable for cushioning of implants and as overdentures during healing processes.

P.U.M.A. soft® shows an excellent long-term adhesion on dentures on methacrylate base (PMMA) in combination with P.U.M.A. soft® ADHESIVE and can be used directly/chairside or indirectly in the lab.

P.U.M.A. soft® meets the requirements of DIN EN ISO 10139-2, Type A (Soft relining material for prostheses for long-term use for removable prostheses).

The P.U.M.A. soft®-kit consists of:

P.U.M.A. soft® ADHESIVE in a 5 ml-bottle
P.U.M.A. soft® (A-silicone, gingiva coloured) in a 50 ml automix cartridge
P.U.M.A. soft® SEAL A + B (base + catalyst) in 5 ml-bottles and accessories.

Recommended procedure

1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE® MIXING CANNULA. The mixing pistol with introduced cartridge is ready for application.

2. Preparation of the denture

Remove any old existing relining material. Then grind off the denture for about 1-2 mm. At the borders make a vestibular, lingual and dorsal sided groove of about 2-3 mm in height with a dept of minimum 1 mm. The edges of the denture should be rounded off and the surfaces to be relined must be roughened.

After the preparation, the denture should be clean and free of fat. The surface can be cleaned with medical alcohol (at least 90 vol.-%).

3. Application of adhesive

A thin coat of P.U.M.A. soft® ADHESIVE (REF PUA4020) will be brushed on the prepared areas of the denture where the soft lining material will be placed. Dry the adhesive for about 1 min in air. Close the bottle after use immediately.

Note: Contaminations with saliva, humidity or monomer liquid inhibits the adhesion of the relining material with the denture.

4a. Direct application

The working time is not more than 1 min (at 74 °F). The automatically mixed P.U.M.A. soft® will be applied uniformly in a layer of about 2 mm first onto the vestibular, lingual and dorsal sided grooves and then onto the denture base. All denture areas which are in contact with the gingiva should be lined.

Then insert the denture into the mouth of the patient (note the working time). The patient should close the mouth in normal occlusion (icp). Now the patient can carry out active and passive functional excursions.

The soft lining material is cured after about 4 min, and the denture can be removed from the mouth of the patient and can be trimmed as described in section 5. and 6.

4b. Indirect application

The denture to be relined will be invested into a flask with counter. Then remove the denture from the flask and prepare it as described in section 2. and 3. The model will be coated with a separating agent according to the instructions for use of the manufacturer.

Then apply the automatically mixed P.U.M.A. soft® uniformly in a layer of about 1 mm more than the base material has ground off on the denture and insert the denture within not more than 1 min (working time) and close the flask with counter and press with a light pressure for about 30 min (at room temperature, 23 °C, 74 °F).

Optionally the flask with counter can be hardened in a pressure pot at 40-45 °C (104-113 °F) for about 10 min.

Then the denture can be finished as described in section 5. and 6.

5. Finishing the denture

Then remove excess silicone with a scalpel and/or scissors. With the help of the enclosed grinding instrument (REF PSK4025) rough areas and borders can be smoothed with 8,000 r.p.m. under low pressure.

6. Sealing (smoothing)

Then the trimmed areas only can be sealed with P.U.M.A. soft® SEAL A + SEAL B (REF PSA403132). Mix equivalent drops of SEAL A and SEAL B in a trough of the mixing pallet with an enclosed brush (REF FFP5185) and trim. The hardening time is about 5 min.

Times mentioned for 23 °C (74 °F). Higher temperatures shorten, lower temperatures prolong these times.

7. Cleaning recommendation for patients

The relined denture should be cleaned several times a day under running water with a toothbrush and neutral soap, in the cleaning bath max. 10 min.

Avoid aggressive cleaning agents.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base - and catalyst paste and thus premature setting.

When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Do not use latex gloves, they may interfere the setting reaction of P.U.M.A. soft®.

Do not use P.U.M.A. soft® with standard condensation reaction type silicone impression materials (C-silicones).

Pay attention to the material safety data sheets (MSDS >> r-dental.com).

Expiration date see label. Do not use after expiration date.

Do not store above 25 °C (77 °F)!

The cartridge should be stored horizontally (laying).

Avoid exposure to sunlight. Keep out of the reach of children!

Technical data

Mixing time: auto-mixing
Working time incl. mixing time: about 1 min

Recommended time in the mouth (direct): about 4 min
Time (start mixing - withdrawal from mouth): about 5 min

Recommended time in articulator/fixator (indirect): about 30 min
Recommended time in pressure pot at 104 °F - 113 °F: about 10 min

P.U.M.A. soft and R-SI-LINE are registered trademarks of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2020 R-dental Dentalerzeugnisse GmbH. All rights reserved.