PermaFlo® DC

Composite Luting Resin

Description

PermaFlo® DC Composite Luting/Restorative Resin, in conjunction with PermaFlo DC Primers A and B, is designed to provide chemical polymerization of the luting resin beneath indirect restorations and endodontic posts which are inaccessible to light. It can be light cured for transparent restorations, i.e. onlays, inlays, and porcelain crowns. It is designed for direct restorations as an initial layer and/or bulk fill for posterior bonded composites and core material prior to crown cementation.

Indications for Use

PermaFlo® DC is used for cementation of metal, porcelain, and porcelain-fused-to-metal crowns, bridges, inlays, and onlays, for luting metal, fiber, and ceramic endodontic posts, and also for core fabrication.

Directions for Use

Cementation of metal, porcelain, and porcelain-fused-to-metal crowns, bridges, inlays, and onlays

1. Remove PermaFlo® DC cap from dual barrel syringe by twisting counterclockwise and discard.
2. Prior to attaching tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate orifices.
3. Attach mixing tip by lining up internal cap stems with syringe orifices. Twist tip clockwise to lock.
4. Clean preparation, rinse, and dry.
5. Verify prosthetic fit.
6. Prepare inside surface of prosthesis.
 - a. Metal based
 - i. Microabrade or sandblast inside surface of prosthesis
 - ii. Apply phosphoric acid to inside surface of prosthesis for 15 seconds, to clean, rinse 5 seconds, and dry.
 - b. Ceramic / porcelain
 - i. Apply hydrofluoric acid to inside surface of prosthesis for one minute, rinse, and dry.
 - ii. Apply silane to inside surface of prosthesis for one minute, dry, and set prosthesis aside. Do not rinse.

7. Prepare tooth surface and apply etch or self-etch primer as per manufacturer's instructions.
8. Place a puddle coat of Primer A for 10 seconds, do not dry. Follow with 2 – 4 drops of Primer B for 10 seconds. (Note: Always place Primer A first).
9. Thoroughly air dry 10 seconds. If a shiny coat is not observed, repeat step 8 until shiny.
10. Apply a thin layer of PermaFlo® DC cement in prosthesis and seat immediately.
11. Remove excess cement.
12. Apply glycerin to prevent oxygen inhibition formation.
13. Light cure margins for 10 seconds (20 seconds for lights with output less than 600mW/cm²).

Cementation of metal, fiber, and ceramic posts with core build-up

1. Prepare tooth and post space according to post manufacturer's instructions.
2. Apply etch or self-etch bonding agent to post space as per manufacturer's instructions.
3. Place a puddle coat of Primer A for 10 seconds, do not dry. Follow with 2 – 4 drops of Primer B for 10 seconds. (Note: Always place Primer A first).
4. Thoroughly air dry 10 seconds. If a shiny coat is not observed, repeat step 8 until shiny.
5. Thoroughly dry entire post space for 10 seconds using air/water syringe.
6. Attach Intraoral Tip to PermaFlo® DC Mixing Tip.
7. Verify flow before using intraorally.
8. Using even pressure, deliver PermaFlo® DC from the base of the post space, coronally.
9. Insert post and cut to length as per manufacturer's instructions.
10. Light cure 10 seconds.
11. Continue building layers of PermaFlo® DC around post to desired height. NOTE: Light-cure for 2-4 seconds between layers to set to gel and form core.

Precautions and Warnings

1. Carefully read and understand all instructions before using.
2. Verify flow of all syringes prior to applying intraorally. If resistance is met, replace tip and re-check. Use only recommended tips.
3. Establish adequate isolation.
4. Not all bonding agents and chemical/dual cure composites or luting resins are compatible. Test before using products outside the Ultradent or any specific manufacturer's product family.
5. Refrigerate PermaFlo® DC when not in use.
6. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, consult a physician.
7. Dispose of tips and the empty syringes properly.
8. Do not allow PermaFlo® DC or PermaFlo® DC Primers A and B to be ingested.
9. When using the PermaFlo® DC, the mixing tip will serve as a cap between patients. Do not remove mixing tip until next use. Disinfect and wipe dual barrel syringe and mixing tip with an intermediate level disinfectant between uses. Change mixing tip before each use.

NOTE: For MSDS and additional information about using PermaFlo® DC and PermaFlo® DC Primers A and B, please go to www.ultradent.com

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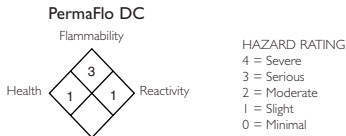
Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers.
The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date.
The first two numbers are the month, and the third number is the last number of the year.

**For product MSDS please see our
website: www.ultradent.com**

U.S. federal law restricts the sale of this device by or on the order of a dentist.
Keep out of reach of children.
For immediate reorder and/or complete descriptions of Ultradent's product line,
refer to Ultradent's catalog or call Toll Free **1-800-552-5512**.
Outside U.S. call (801) 572-4200
or visit www.ultradent.com.

© Copyright 2007 Ultradent Products, Inc. All Rights Reserved.