PermaSeal®

Composite Sealer and Bonding Agent

Description:
PermaSeal® composite sealer is a light-cured, unfilled methacrylate-based resin designed to cure in a very thin film and with one of the thinnest oxygen inhibition layers known.

Indications:
PermaSeal penetrates, seals cracks and Surface porosity in composite restorations. Finishing and polishing of composite develops microscopic defects in the surface, resulting in accelerated loss of composite, interpreted clinically as wear. PermaSeal, applied at the restoration placement and on an annual basis, can marginal integrity of composite restorations by filling microscopic cracks in composite surfaces.

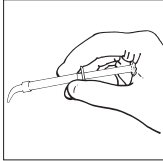
Fig. 1 Hold plunger in palm of hand for optimum control.

Procedure:

1. Remove luer lock cap from syringe.
2. Twist a Black Micro FX tip firmly onto syringe.
3. Check for uniform flow through tip by expressing a small amount on a paper pad before intraoral use.
4. For optimum control, hold plunger in palm of hand as shown in Fig. 1 rather than in conventional manner with thumb on plunger.
5. Add composite following recommended placement procedures, including incremental layering to maximize bonding. PermaSeal will function as a bonding agent between enamel or existing composite and the new composite material (note Figs. 4-5). We recommend Vitalescence® or Amelogen® composites to obtain a highly esthetic, wear-resistant repair.
6. Complete steps 1 through 6 of "New Restorations" to seal the repaired restoration.
7. To seal glass ionomers, etch, wash and dry. Place an even layer of PermaSeal and light cure. (Leave PermaSeal thicker than it would be before curing composite.)

Precautions:

1. Remove excess resin flash interproximally and subgingivally when present.
2. Take care not to bond adjacent restorations together.
3. Remember to establish adequate isolation before using PermaSeal. A clean, dry field is imperative for optimal resin bonding.
4. Be sure the outside orifice of the syringe as well as the luer taper on the tip is resin-free to ensure complete lightening of the tip against the syringe. Never force syringe plunger; if there is excess resistance to flow, replace tip with a new one and verify flow before proceeding.
5. Use oil-free, moisture-free air for drying after etching. We recommend Aqua4™ to evaluate moisture content of air.
6. Any resin can be sensitizing to the patient, causing an allergic reaction if individual is already sensitized. Wash exposed area completely to remove the material. If dermatitis, rash or any other allergic reaction is observed, consult a physician.
7. We highly recommend using Ultradent® Syringe Covers sealed with the Ultradent® Impulse Sealer to provide a protective barrier against cross-contamination.

EN-Composite Sealer & Bonding Agent
DE-Composite-Versiegler & Bonding-Material
FR-Composite Sealer & Agent Adhésif
NL-Compositesealer en -bonding

IT-Sigillante per Compositi e Agente Adesivo
ES-Sellante de Resina y Agente Adhesivo
PT-Selante para Compositos
SV-Kompositföregelning och Bond

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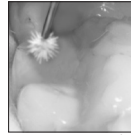
Fig. 2 Deliver a thin coat of PermaSeal with a Black Micro FX tip by aggressively rubbing for 5 seconds. Air thin and cure.

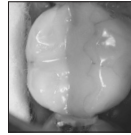
Fig. 3 Finished, sealed restoration.

B. New Restorations:

1. Contour composite restoration, finishing and polishing to the desired surface texture. Adjust occlusion as needed.
2. Isolate tooth or teeth being sealed.
3. Clean restoration and remove indicating medium markings. All composite surfaces to be sealed and adjacent tooth surfaces are etched 20 seconds with Ultra-Etch® 35% phosphoric acid, then rinsed and dried.
4. Rub a thin layer of PermaSeal for 5 seconds into the surface of composite and adjacent enamel (Figs. 2-3).¹ Adjust the brush length to vary the brush width for most effective use. Longer brush fibers will allow for greater fanning of the fibers, resulting in a wider spread of resin per brush stroke.
5. Gently air thin. Light cure PermaSeal for 20 seconds.²
6. Evaluate occlusion for prematurities. Usually, the occlusion does not require adjustment because of the minimal film thickness of PermaSeal

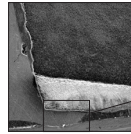
Fig. 4 Note SEM showing a slight flash of composite past the margin of a Class V composite.


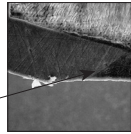
Fig. 5 Upon higher magnification, PermaSeal resin has infiltrated into the microspace between enamel and composite, sealing it.

C.

Previously Placed Composite Restorations:

1. Clean surfaces and margins to be sealed thoroughly with ConsepSis® Scrub, pumice, or a microetcher and rinse thoroughly.
2. Etch the enamel immediately adjacent to the restoration and all accessible composite surfaces for 20 seconds.
3. Complete steps 1 through 6 of "New Restorations" for sealing. When defects are too large to be filled with PermaSeal, follow "Composite Bonding and Repair" steps.
4. Composite Bonding and Repair:³
 1. Thoroughly clean surfaces and margins to be sealed. Freshen surfaces with a high-speed bur or a microetcher and rinse.
 2. Remove weakened portions of damaged, existing composite. Open defect to be repaired to limit air entrapment during placement of new composite.
 3. Etch the enamel immediately adjacent to the restoration and all accessible composite surfaces with Ultra-Etch for 15 seconds.
 4. Complete steps 4 through 6 of "New Restorations" to the right.

¹Use appropriate care not to bond to adjacent teeth.
²Studies (two examples below) and clinical experience show that quality of light varies from curing light to curing light. Effectiveness of light and light-conducting fibers, tubes, etc., decreases as unit ages. Light cure units should be checked to ensure that maximum light is being delivered for proper curing. Lower intensity lights may require additional exposure time. Check with Curing Radiometer.
³A. Dugan, William T. and Harlie, John H., "Influence of a Glutaraldehyde Disinfecting Solution On Curing Light Effectiveness." General Dentistry 37: 40-43, 1989.
⁴B. Clinical Research Associates Newsletter, 12, 3, February, 1988.
⁵If a significant portion of the old restoration is missing, it is advised to replace the restoration completely. We recommend "D1" Dentin Bonding System when bonding to dentin and enamel followed by Amelogen Plus Composite or Vit-C-essence.

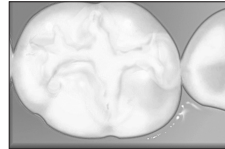EN Composite restoration completed.

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- EN 2** General Precautions:
- For Professional use only.
 - Review instructions, precautions, and MSDS before beginning treatment. Use only as directed.
 - Keep products out of heat/sunlight.
 - Avoid skin exposure to resin.
 - Isolate strong chemicals to area of treatment.
 - Confirm that patient has no known allergies to treatment materials.
 - Test flow of material from syringe and tip before using intraorally.
 - Never force syringe plungers.
 - Use Ultradrant™ Syringe Covers and/or clean and disinfect syringes between patients.
 - Use standard curing lights with all light polymerized/activated materials.

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Ultradrant syringes have an expiration date stamped on the side of the syringe consisting of one letter and two numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or the order of a dentist. For professional use only. Keep out of reach of children. For immediate reader and/or complete descriptions of Ultradrant's product line, refer to Ultradrant's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200, or visit www.ultradrant.com.

Manufactured by: Ultradrant Products Inc. 505 West 10200 South. Jordan, Utah 84095 USA Made in USA

For product SDS please see our website www.ultradrant.com 1-800-552-5512; 801-572-4200

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