Contenu

méthacrylate de méthyle, 2-Propenoic acid, reaction product with Pentaerythrite, Diphenyl (2,4,6-trimethylbenzoyl) phosphinooxide, Résine acrylique.

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Stand der Information: 04.2018

Intended use

Dental sealing lacquer for coating resin temporaries.

Indications for use

Varnish for final coating for resin temporaries to produce a smooth and shiny surface.

Processing instruction

As a ready for use 1-component-material **Plaquit** can be applied directly with a brush onto the area which has to be coated. For a safe adhesion of the lacquer the surface should be roughened (sanded or sandblasted), free of grease and dry. The layer should be as thin as possible which means that the surface of the denture should only be moistened. If a thicker layer is required it is advantageous to cure the first thin **Plaquit** layer for a short time (approx. 30 sec.) and afterwards to apply a second layer.

Polymerisation

The hardening should be effected in a polymerisation unit with a wave length of 300–400 nm. Depending on the intensity of light the hardening can lay between 3 and 5 minutes. All areas coated with **Plaquit** have to be completely polymerised as nonpolymerised lacquer can cause tissue irritations. This means that in some polymerisation units the coated denture – depending on the position of the light source has to be turned over for several times.

Hazard statements

H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H317 May cause an allergic skin reaction. H318 Causes serious eye damage. H361f Suspected of damaging fertility. H412 Harmful to aquatic life with long lasting effects. H335May cause respiratory irritation.

Precautionary statements

P210 Keep away from heat / sparks / open flames / hot surfaces. – No smoking. P303+P361+P353 IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.

Important advice

- Do not inhale and do not get in contact with skin or eyes.
- Due to the legal product liability we cannot quote binding polymerisation times for units of other manufacturers.
- Sensitive persons can show allergic reactions to some product components.
- The product is exclusively designed for the a. m. field of application and is only to be used by qualified and instructed persons.
- The general information of this product is based on detailed application practice and intensive research work. Due to further development we reserve the right of modification.

Keep away from children!
Only for use in the dental technique!

Storage conditions

Keep bottle closed well, protect from light. At room temperature, i. e. 18–28 °C.

Durability: see date of expiry.

Lot number / Expiry date: The lot number and the expiry date are indicated on each **Plaquit** packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

P305+P351+P338 IF ON SKIN (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower. P310 Immediately call a POISON CENTER / doctor. P405 Store locked up. P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

Contains

methyl methacrylate, 2-Propenoic acid, reaction product with Pentaerythrite, Diphenyl (2,4,6-trimethylbenzoyl) phosphinoxide, Acrylic resin.

Forms of delivery

 D38124	Plaquit 20 ml bottle with brush
 D38123	Plaquit 50 ml bottle with brush

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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