# Primo

DE

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15°C  
59°F

A thermometer icon with a scale showing 15°C (59°F) and 25°C (77°F).

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# Primo

Instructions for use

## Adhesive for Molloplast® B-relinings

By use of **Primo adhesive** the chemical bonding of **Molloplast® B** to the acrylic denture base is achieved. However, the adhesive is only applied onto already polymerised denture base material. Prior to the application of **Primo adhesive** the denture base is prepared according

to **Molloplast® B** instructions for use. Then **Primo adhesive** is brushed uniformly 1–2 times onto the clean and dried denture base. After reacting for approx. 60–90 minutes **Molloplast® B** is pressed onto the denture base and polymerised according to instructions.

### Important working hints

- **Primo** coating must cover the entire surface of denture acrylic in contact with **Molloplast® B**. After a reacting time of approx. 60–90 minutes **Molloplast® B** is pressed onto the denture base and polymerised according to instructions.
- Close bottle tightly after each use.
- Avoid contact with eyes, mucous membranes and skin. In case of accidental contact, rinse eyes immediately with running water and if necessary consult an ophthalmologist.
- Do not breathe vapours.
- Please follow the instructions of the safety data sheet!

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personell).

# DETAX

GmbH & Co. KG  
Carl-Zeiss-Str. 4 · 76275 Ettlingen/Germany  
Telefon: 07243/510-0 · Fax: 07243/510-100  
www.detax.de · post@detax.de

Made in  
Germany

02/2018

### Indications for use:

- **Primo adhesive** ensures an optimum bonding of **Molloplast® B** to polymerised acrylics.

### Technical data:

- **Volume:** 15 ml, vial with integrated brush
- **Drying time:** 2–3 minutes
- **Reacting time:** 60–90 minutes
- **Storage:** Keep container tightly closed. Keep away from moisture.

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