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POLYDENTIA SA
Swiss Products for Fine Dentistry

For use with Polydentia's Quick Ring Forceps and Quickmatrix Forceps

www.quickmatdeluxe.com

FOR PERFECT ANATOMICAL CONTACTS ON POSTERIOR COMPOSITE RESTORATIONS

Content: Quickmat Deluxe REF 5701

- 3 Quickrings, 1 Re-Force insert
- 6 Cylindrical Silicone Tubes
- 10 Delta Silicone Tubes
- 100 Contoured sectional matrices 0.04 mm thin, 50 pcs. 5 mm and 50 pcs. 6.4 mm high
- 40 Contoured sectional matrices microthin 0.025 mm, 20 pcs. 5 mm and 20 pcs. 6.4 mm high
- 140 Assorted Wood Wedges: 50 pcs 11mm (XS, pink), 50 pcs 12 mm (X, blue), 20 pcs 13mm (M, green), 20 pcs 15mm (L, yellow)

All items can be re-ordered separately as refills.

Made in Switzerland • Polydentia SA • 6805 Mezzovico-Vira • Switzerland
Phone +41 (0)91 946 29 48 • Fax +41 (0)91 946 32 03
Web www.quickmatdeluxe.com • E-mail info@polydentia.com

REV. 02.11.2013

The ideal sectional matrix system for perfect anatomical contacts on posterior composite restorations. For use with Quickmat Forceps (REF 5702).

INSTRUCTIONS FOR USE

1. Place a rubber dam, put a sectional matrix in place and wedge it.
2. Use Quickmat Forceps to place the Quickring.
3. Burnish the matrix against the adjacent tooth and proceed with the filling.
4. For large cavities place silicone tubes on the Quickrings.
- 5.-6. MOD restorations. Place sectional matrices and wedges on both sides of the tooth. Quickrings are designed not to interfere with one another.

Quickrings, **Re-Force** and silicone tubes are autoclavable at 134 °C (273 °F).

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




ORDER FORM Quickmat DELUXE

Please send me the following products:

Item No.	Description	Quantity
REF 5701	Quickmat Deluxe Kit	
Refills		
REF 5704	3 Quickrings + 3 Re-Force inserts + 8 Delta Silicone Tubes	
REF 5705	100 contoured sectional matrices (5 mm high, 0.04 mm thin) - Premolar	
REF 5706	100 contoured sectional matrices (6.4 mm high, 0.04 mm thin) - Molar	
REF 5735	100 contoured sectional matrices (5 mm high, 0.025 mm microthin) - Premolar	
REF 5736	100 contoured sectional matrices (6.4 mm high, 0.025 mm microthin) - Molar	
REF 5746	50 contoured sectional matrices with extension (6.4 mm high, 0.04 mm thin)	
REF 5756	50 contoured sectional matrices with extension (6.4 mm high, 0.025 mm microthin)	
REF 5710	5 Quickrings + 10 Cylindrical Silicone Tubes	
REF 5711	30 Cylindrical Silicone Tubes	
REF 5712	30 Delta Silicone Tubes	
Complementary products:		
REF 5765	100 transparent sectional matrices (5 mm high) - Premolar	
REF 5766	100 transparent sectional matrices (6.4 mm high) - Molar	
REF 5776	50 transparent sectional matrices with extension (6.4 mm high) - Molar	
REF 5703	Quickmatrix Forceps (for placement & removal of sectional matrices)	
REF 5702	Quickmat Ring Forceps (for easy placement of Quickrings)	

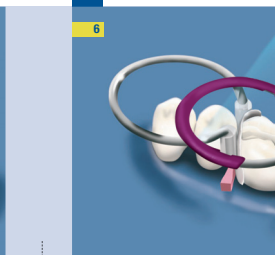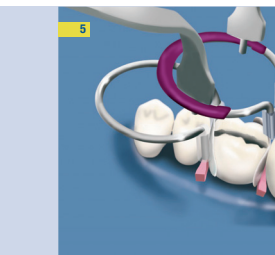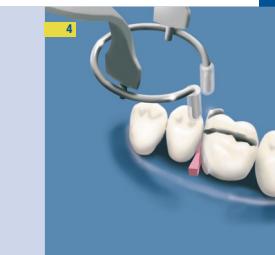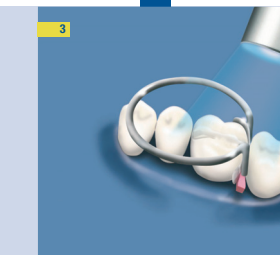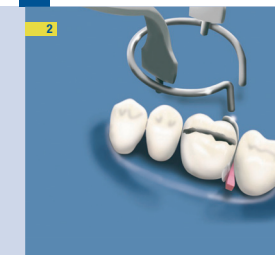POLYDENTIA SA
Swiss Products for Fine Dentistry