R-SI-LINE® C-HÄRTER Lab

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Hersteller

R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
 Telefon 0 40 - 30707073-0 • Fax 040 - 30707073-73 • www.r-dental.com

R-SI-LINE® C-HÄRTER Lab
Bestell-Nr.: KPH1008

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R-SI-LINE® C-HARDENER Lab
Order-No.: KPH1008

Description and indication

R-SI-LINE® C-HARDENER Lab is a paste hardener for hardening the condensation curing silicone R-SI-LINE® Lab-PUTTY used extraorally in the lab (plastic tube).

Recommended procedures**Dosage of C-HARDENER for R-SI-LINE® Lab-PUTTY**

R-SI-LINE® Lab-PUTTY (Order-No.: LPU1016) and R-SI-LINE® C-HARDENER Lab will be mixed in a ratio by weight of about 30 : 1. Then onto the spoon filled with Lab-PUTTY for every spoon diameter 3 strands of the catalyst paste R-SI-LINE® C-HARDENER Lab will be applied (see also the Instructions for use for R-SI-LINE Lab-PUTTY).

Then knead both materials for about 30 s until the colour is homogeneous. The working time is about 1 min 30 s.

The hardening time (Lab-PUTTY + C-HARDENER Lab) is about 8 min at room temperature.

Times mentioned for 74 °F. Higher temperatures shortens, lower temperatures prolongs these times.

Special storage and/or handling conditions

Do not use R-SI-LINE® C-HARDENER Lab for addition curing silicones (A-Silicones).

Avoid contact of R-SI-LINE® C-HARDENER Lab with skin or eyes. If there is an accidental contact with skin, flush immediately with copious amounts of water. If there is an accidental contact with eyes, flush immediately with copious amounts of water and consult an a physician.

Expiration date see label. Do not use after expiration date.

Do not store above 78 °F (25 °C). Avoid exposure to sunlight. Keep out of the reach of children!

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for defections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.

© 2016 R-dental Dentalerzeugnisse GmbH. All rights reserved.

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R-SI-LINE ist ein eingetragenes Warenzeichen der R-dental Dentalerzeugnisse GmbH.
© 2016 R-dental Dentalerzeugnisse GmbH. Alle Rechte vorbehalten.
R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg • DE