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R-SI-LINE® DUBLIER

**Hersteller**

R-dental Dentalerzeugnisse GmbH - Hamburg  
Telefon 0 40 - 22 75 76 17 - Fax 0 40 - 22 75 76 18 - www.r-dental.com

**R-SI-LINE DUBLIER**

**Bestell-Nr.: DUB10XX17 (Shore-A-Härte 17, blau)**  
**Bestell-Nr.: DUB10XX22 (Shore-A-Härte 22, grün)**  
**Bestell-Nr.: DUB10XX24 (Shore-A-Härte 24, grau-grün)**

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**R-SI-LINE DUBLIER**

**Order-No.: DUB10XX17 (Shore-A hardness 17, blue)**  
**Order-No.: DUB10XX22 (Shore-A hardness 22, green)**  
**Order-No.: DUB10XX24 (Shore-A hardness 24, grey-green)**

R-SI-LINE DUBLIER is used for duplicating models (preferably of gypsum). Available are materials with different Shore-A-hardnesses.

R-SI-LINE DUBLIER is a fluid, addition-curing precision impression duplicating material on vinylpolysiloxane base (A-silicone), which also duplicates undercuts very precisely. The high quality vinylpolysiloxanes have a minimal shrinkage and a very good stability of material and duplicating mold.

R-SI-LINE DUBLIER has a good tensile strength and is very deformable. The duplicating mold can be poured for several times.

R-SI-LINE DUBLIER meets the requirements of ADA Spec. No. 20.

**Recommended procedures****1. Preparation**

Before use shake base - and catalyst material very intensively. Then mix equal amounts (volume or weight) of base material (coloured) and catalyst material (white) in a clean mixing pot for about 45 s to a homogeneous and bubble-free mixture (if possible under vacuum). The use of an automatic mixer is also possible because of equal viscosities of base and catalyst material.

**2. Duplicating**

The mixed duplicating material will then be poured from a height of about 10-20 cm over one point of the model within the duplicating mold. The working time at room temperature is about 4 min. After about 30 min setting time the model can be taken out of the duplicating mold.

**3. Pouring**

The duplication mold should be poured with gypsum not earlier than 30 min after the setting time.

**4. Recycling**

The duplicating mold can be reduced to small particles and these particles can be used again.

**Precautions to application, product life and/or storage**

Do not interchange the caps of base - and catalyst material!

The shelf life is 3 years when stored at 74 °F. Do not store about 78 °F. Avoid exposure to sunlight. A cool storage prolongs shelf life.

Do not store the duplication materials and the duplicating mold near solvents or other volatile materials because of a swelling and a dimension loss of the mold.

® registered trademark of R-dental Dentalerzeugnisse GmbH

**Caution:** For dental use only. US Federal Law restricts this device to sale by or on the order of a dentist or licensed professional, to use or to order the use of this device.

The described product must be used according to the instructions for use and is determined for dental use only. There is no liability for defections resulting from inappropriate use or not intended purposes. The user is responsible for testing the suitability of the product for any purposes not explicitly stated in the instructions for use.

If there are damages to this product within the warranty period, R-dental is obliged to replace or to repair this product. There is no further claim. Outside of the liability by law there is no liability of R-dental for any loss or damages arising from this product, independently whether these damages are direct, indirect, special, incidental or consequential.