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R-SI-LINE® UltraLIGHT SH

**Hersteller:**R-dental Dentalerzeugnisse GmbH • Winterhuder Weg 88 • 22085 Hamburg
T 040-30707073-0 • F 040-30707073-73 • I www.r-dental.com

REF ULK1043SH (R-SI-LINE® UltraLIGHT SH)

REF ULK1043SHFS (R-SI-LINE® UltraLIGHT SH FS (fast set))

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REF ULK1043SHFS (R-SI-LINE® UltraLIGHT SH FS (fast set))

Description and indication

R-SI-LINE® UltraLIGHT SH (- SH FS) in auto-mixing cartridges is a super hydrophilic precision impression material for the putty-wash-technique (two steps) in conjunction preferred with the preliminary impression material R-SI-LINE® PUTTY (REF PUT1010), R-SI-LINE® PUTTY FS (REF PUT1010FS) or the base impression material R-SI-LINE® PUTTY MATIC (REF PMK1013).

Characteristics

R-SI-LINE® UltraLIGHT SH (- SH FS) is a super hydrophilic, very low viscosity addition curing precision impression material on vinyl polysiloxane base. It can be directly and bubblefree applied onto the teeth or the tray.

R-SI-LINE® UltraLIGHT SH (- SH FS) is very thixotropic and shows a controlled flow behavior. Thus it stays in the position on the tray, but under pressure it flows.

R-SI-LINE® LIGHT SH (- SH FS) is very hydrophilic and shows an excellent detail reproduction of subgingival areas.

R-SI-LINE® UltraLIGHT SH (- SH FS) meets the requirements of DIN EN ISO 4823, Type 3 (low consistency - light bodied).

Application

1. Preparing the auto-mixing cartridge/R-SI-LINE MIXING PISTOL N

Introduce the auto-mixing cartridge into the MIXING PISTOL N (REF MIP1081) and lock it. Before first use screw off the original cap and dispose it. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally. Then screw on the mixing cannula R-SI-LINE® MIXING CANNULA. The mixing pistol is ready for application.

For direct fine application of the R-SI-LINE® UltraLIGHT SH (- SH FS) onto the prepared areas additionally the R-SI-LINE® INTRA-ORAL-TIPS L (REF IOT2001) should be used.

For fine applications instead of R-SI-LINE® MIXING CANNULA LN (REF MIK1099) + INTRA-ORAL-TIP L (REF IOT2001) a NEEDLE MIXING CANNULA LNN (REF MIK1099N) with an integrated, curved application needle is available.

2. Impression taking

Before application a small amount of material (size of a pea) should be pressed out and should be wasted immediately.

Load the auto-mixed R-SI-LINE® UltraLIGHT SH (- SH FS) onto the R-SI-LINE® SILICONE ADHESIVE (REF SIA1070) coated tray and/or onto the teeth or the prepared areas.

Insert the tray into the mouth not later than 2 min (the fast set variant - UltraLIGHT SH FS not later than 1 min 30 s) after extruding (working time).

Maintain the position of R-SI-LINE® UltraLIGHT SH for at least 3 min (the fast set variant - UltraLIGHT SH FS at least 2 min) in the mouth (hardening time).

Times mentioned for 74 °F. Higher temperatures shorten, lower temperatures prolong these times.

3. Casting and disinfection

The impression should be stored at normal room temperature (< 25 °C, 77 °F) and can be poured 30 min after removal from the mouth. There are no other time limitations.

The impressions can be disinfected with standard disinfection solutions.

Special storage and/or handling conditions

Before first use screw off the original cap and dispose it. Closing the cartridge by turning on the original cap between use could cause cross contamination of base- and catalyst paste and thus premature setting. When screwing on a new mixing cannula the cartridge should be tested if both pastes will be extruded equally.

Before application a small amount of material (size of a pea) should be squeezed out and should be disposed immediately.

The filled MIXING CANNULA functions as a seal until the next application. Do not re-use the original cartridge cap.

Keep attention on the material safety data sheet (MSDS >> www.r-dental.com).

Shelf life see label. Do not use after expiry date.

Do not store above 77 °F! Avoid exposure to sunlight. Keep out of the reach of children!

Do not use R-SI-LINE® UltraLIGHT SH (- SH FS) with standard condensation reaction type silicone impression materials.

Do not use Latex gloves. These could inhibit the hardening reaction of the A-silicone in the case of contamination.

Technical data

| | |
|---|--------------|
| Mixing time: | auto-mixing |
| Working time incl. mixing time (- UltraLIGHT SH): | < 2 min |
| Working time incl. mixing time (- UltraLIGHT SH FS): | < 1 min 30 s |
| Recommended time in the mouth (- UltraLIGHT SH): | ≥ 3 min |
| Recommended time in the mouth (- UltraLIGHT SH FS): | ≥ 2 min |
| Time (start mixing - withdrawal from mouth) (- UltraLIGHT SH): | ≥ 5 min |
| Time (start mixing - withdrawal from mouth) (- UltraLIGHT SH FS): | ≥ 3 min 30 s |
| Dimensional change after 24 hours: | < 0.1 % |
| Elastic recovery: | > 99.6 % |
| Strain in compression: | about 5.7 % |
| Shore-A-Hardness: | about 42 |

R-SI-LINE is a registered trademark of R-dental Dentalerzeugnisse GmbH.

Caution: For dental use only.

Warranty

R-dental Dentalerzeugnisse GmbH warrants this product will be free from defects in material and manufacture. R-dental Dentalerzeugnisse GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. The user is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and R-dental Dentalerzeugnisse GmbH's sole obligation shall be repair or replacement of the product.

Limitation of Liability

Except where prohibited by law, R-dental Dentalerzeugnisse GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

© 2020 R-dental Dentalerzeugnisse GmbH. All rights reserved.