**PRINT SPECIFICATION PAGE ONLY.
DO NOT PRINT THIS PAGE IN FINAL IFU.**

INSTRUCTIONS FOR USE – PRINT SPECIFICATIONS

Instructions for Use – Sonic Fill 3

PART NUMBER	DMC3486	
REVISION	Rev01	
REVISION DATE	2019/08/15	
BINDING	Single stitch in center of 2.5 inch spine	
CORNER	N/A	
COLORS	1/1 Black	
SIZE	Sheet size - 22 inches (W) x 10.5 inches (H) / Finished size – 5.0 inches (H) x 2.5 inches (W)	
PAGE COUNT	48 pages – 96 panels	
MATERIAL	40-45 grams per square meter 30-32# offset	
ADHESIVE MATERIAL	Single wafer seal on the 2.5 inch open side	
COATING	N/A	
LANGUAGES: 36	en-ENGLISH	ENGLISH
	fr-FRENCH	FRANÇAIS
	es-SPANISH	ESPAÑOL
	de-GERMAN	DEUTSCH
	it-ITALIAN	ITALIANO
	da-DANISH	DANSK
	sv-SWEDISH	SVENSKA
	el-GREEK	ΕΛΛΗΝΙΚΑ
	nl-DUTCH	NEDERLANDS
	no-NORWEGIAN	NORSK
	fi-FINNISH	SUOMI
	pt-PORTUGUESE	PORTUGUÊS
	pl-POLISH	POLSKI
	ru-RUSSIAN	РУССКИЙ
	ro-ROMANIAN	ROMÂNĂ
	hu-HUNGARIAN	MAGYAR
	sl-SLOVENIAN	SLOVENSKO
	cs-CZECH	ČESKÝ
	tr-TURKISH	TÜRKÇE
	et-ESTONIAN	EESTI
	lv-LATVIAN	LATVIEŠU
	lt-LITHUANIAN	LIETUVIŲ
	sk-SLOVAK	SLOVENSKÝ
	ar-ARABIC	العربية
	bg-BULGARIAN	БЪЛГАРСКИ
	hr-CROATIAN	HRVATSKI
	sr-SERBIAN	SRPSKI
	ja-JAPANESE	日本語
	ko-KOREAN	한국어
	zh-MANDARIN SIMPLIFIED	简体中文
	zh-t-MANDARIN TRADITIONAL	中文繁體版
	id-INDONESIAN	BAHASA INDONESIA
	is-ICELANDIC	ÍSLENSKA
	th-THAI	ไทย
	vi-VIETNAMESE	TIẾNG VIỆT
	uk-UKRAINIAN	УКРАЇНСЬКА
SUPPLIER	Calico	

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90

range. Please refer to the table below for curing recommendations. All times refer to a single occlusal surface cure. In any posterior restoration, Kerr recommends additional 10 seconds cures on both the buccal and lingual surfaces of the tooth.

Curing Method	All Shades
Demi Ultra/Demi Plus (or light with output >1000 mW/cm ²)	10 Seconds

Storage and Shelf Life

SonicFill 3 should be stored at ambient temperature. The expiration date is marked on packaging. Do not use after expiration date. Every effort should be made to protect the composite from visible light prior to use.

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

User and Patient Notice: If a serious incident occurs with this medical device, report it to the manufacturer and to the competent medical authority for the user/patient's country.

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