HOFFMANN'S

HOFFMANN'S BASISPLATTEN
 Schellack Basisplatten, rosa de

HOFFMANN'S BASISPLATTEN
 Shellac Base Plates, pink en

HOFFMANN'S BASISPLATTEN
 Plaques Bases en Gomme Laque, rose fr

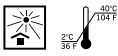
SHELLACK BASISPLATTEN
 Shellac Base Plates

For instructional film and for further languages visit our website www.hoffmann-dental.com

- 12x ReOrder-No.: 82825
- 12x ReOrder-No.: 82826
- 8x/4x ReOrder-No.: 82824

- 100x ReOrder-No.: 8282105
- 100x ReOrder-No.: 8282106
- 50x/50x ReOrder-No.: 8282104

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Instruction for use for the dentist/dental technician

Composition:
 base plates contain: shellac, stearin, talcum, mica

Areas of Application:
 Individual impression trays, basis for bite registration, basis for arranging and articulating artificial teeth.

Processing Recommendation:
 Soften the base plate with a fanned flame or heat gun/blow dryer and press onto the plaster cast with your fingers. Excess material can be removed with scissors or scalpel while soft. Soften the plate again and press the edges into place and round them off.

Notes:
 To prevent possible damage, never press an insufficiently heated base plate onto a plaster cast. Block out undercuts. Use milling cutters slowly.

- Available Package Sizes:**
- Standard size containing 12 plates, available in upper jaw, lower jaw or assorted.
 - Laboratory size containing 100 plates, available in upper jaw, lower jaw or assorted.

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HOFFMANN'S BASE PLATES

- for dental and dental laboratory use only
- keep cool and dry
- do not expose to direct insolation
- Do not use after the expiration date
- Material safety data sheet (MSDS) available at www.hoffmann-dental.com

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