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- SERASYNTH®
- SERASYNTH® LOC

## WARNINGS AND PRECAUTIONS

- Read the instructions for use in their entirety before using the product.
- The product should only be used by qualified and trained personnel. It is the physician's responsibility to be familiar with both appropriate surgical techniques and anatomy prior to use.
- The product is delivered sterile (sterilization method: ethylene oxide). 
- The product is intended for single use only. 
- Do not resterilize or re-use the product as critical changes in performance and safety are possible. 
- Do not use after expiration date. 
- Do not use if the product shows signs of damage (e.g. cuts or kinks) or if the outer packaging or sterile packaging has been damaged or unintentionally opened prior to use. 
- Use the product only for the intended indication.
- Please contact the manufacturer for further information.

## DESCRIPTION AND INTENDED PURPOSE

SERASYNTH® and SERASYNTH® LOC are synthetic, absorbable, monofilament surgical suture materials. The products are manufactured of the polymer poly-p-dioxanone (PDO). They are available dyed with D&C violet No. 2, Colour Index No. 60725. SERASYNTH® and SERASYNTH® LOC are available in various gauge sizes and lengths. They are supplied with various types of needles made of surgical steel. They may be supplied in combination with other components, such as a suture guiding tube. An anchor fixation system based on monodirectional barbs allows knotless fixation of SERASYNTH® LOC.

The underlying suture material complies with the current edition of the European Pharmacopoeia (EP) monograph for "Sutures, sterile synthetic absorbable (Fila resorbilia synthetica monofilamenta sterilia)", although SERASYNTH® LOC deviates from the pharmacopoeia in that it is the linear and not the knot tensile strength that forms the basis for suture stability.

SERASYNTH® is indicated for use in soft-tissue approximation and/or for ligation in cases where long-term (up to six weeks) absorbable suture material is indicated, including use in cardiovascular surgery, ophthalmic procedures and microsurgery.

SERASYNTH® LOC is intended for soft-tissue adaptation when long-term (up to six weeks) absorbable sutures are indicated.

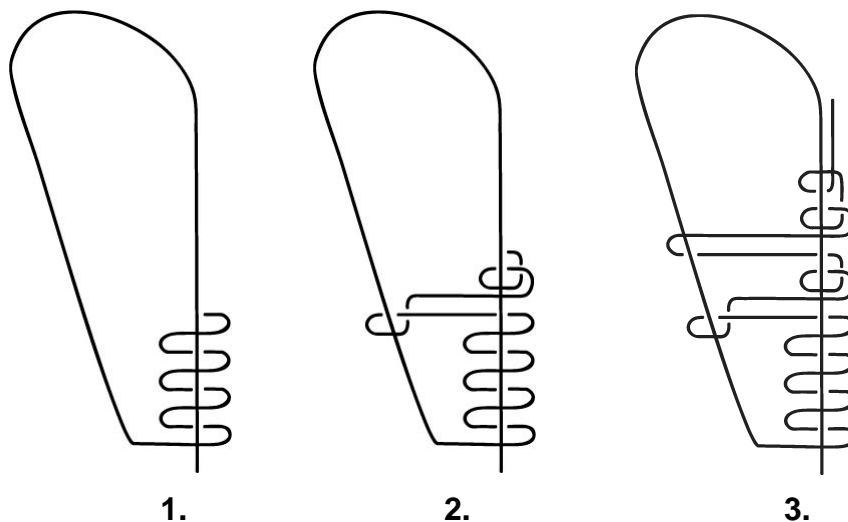
## INSTRUCTIONS FOR USE

In the choice and use of suture material, due account should be taken of the condition of the patient, the experience of the physician, the surgical procedure, and the type and size of the wound. Standard knot-tying techniques for synthetic monofilament sutures should

be used to ensure knot-holding security. In order to prevent needle damage during suturing it is recommended that the needle be grasped at an area between the end of its first third (as seen from its end) and its midpoint.

Unneeded suture material may be used for ligature or may be connected with surgical eye needles. For a list of suitable needles and application instructions, please refer to the Instructions for Use of Serag-Wiessner sterile eye needles. The suitability of other manufacturers' products has not been tested. When using other brands, their specific instructions must be observed.

**Endosuture SERASYNTH®:** due to the material characteristics of the suture base material PDO, not all the conventional knots are suitable for the SERASYNTH® Endosuture. Serag-Wiessner has developed a novel knot which is marked by its ease of tying and good knot security. Please proceed as shown in the diagram:

### **ACTION**

The function of a medical suture in wound care is to attach tissue to tissue or tissue to foreign material. Ligatures are a special type of suture intended to close hollow organs. The polymer PDO is degraded in the tissue by means of hydrolysis to 2-hydroxyethoxy acetic acid which is subsequently metabolised by the organism. This absorption expresses itself as a loss of tensile strength which is later accompanied by loss of mass. The tensile strength is reduced to about 60% of its initial value within 28 days (in-vitro). Absorption is completed after about 220 days (in-vivo).

### **CONTRAINDICATIONS**

SERASYNTH® / SERASYNTH® LOC must not be used where long-term stability of the suture material is required. The device must not be used in patients with allergies to any of the product components.

### **RESTRICTIONS AND LIMITATIONS OF USE / NOTES / INTERACTIONS**

The requirements of the suture material and the techniques required vary with the indication. In the choice of suture material, account should be taken of the characteristics of the suture material concerned. The use of the device should be carefully considered in patients with delayed wound healing. Considerations should be taken regarding the use of absorbable sutures in tissues with poor blood supply as suture extrusion and delayed absorption may occur. The use of absorbable sutures may be inappropriate in elderly, malnourished or debilitated patients. Like all foreign bodies, the device can give rise to calculus formation when in prolonged contact with salt-containing solutions (e.g. in the

biliary or urinary tract). Additional use of nonabsorbable suture materials or suitable immobilization techniques may be required with sutures that are under tension or require greater support. In intracutaneous sutures the suture material, in particular the knots, should be placed as deep as possible. Skin sutures which remain in place longer than 7 days may cause local irritation and should be snipped off or removed. To ensure that the anchor fixation system functions properly, SERASYNTH® LOC should be handled carefully and the suture finished off with a sufficiently long backstitch, for example. There is still only limited experience of use in the nervous and cardiovascular systems, for intestinal anastomoses, and the closure of fascia.

Surgical instruments should be handled with care in order to avoid damage to the thread. Bending of needles or grasping them outside of the recommended area can result in needle breakage. Because of the risk of infection from needle pricks, needles should be handled with particular care.

### **ADVERSE REACTIONS**

The following adverse reactions can occur in association with correct use of the device: local irritation, inflammatory reactions to the foreign body, formation of suture fistulas or granulomas.

Users and/or patients are encouraged to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority.

### **STORAGE CONDITIONS**

The storage conditions apply as indicated on the label.

### **DISPOSAL INFORMATION**

Disposal of unused products or waste material with particular care due to the potential risk of contamination and/or injury. In all other cases observe the specific national regulations.

### **MEANING OF THE SYMBOLS**

Symbols are explained at the end of the instructions for use.

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SERAG-WIESSNER GmbH & Co. KG