signum®

zirconia bond

KULZER
MITSUI CHEMICALS GROUP

(DE) Gebrauchsanweisung	3	(HU) Használati utasítás	49
(GB) Instructions for use	7	(LV) Lietošanas instrukcija	52
(FR) Mode d'emploi	10	(LT) Naudojimo instrukcija	55
(ES) Instrucciones de uso	14	(PL) Instrukcja obsługi	58
(IT) Istruzioni per l'uso	18	(EE) Kasutusjuhend	62
(PT) Instruções de uso	22	(HR) Upute za uporabu	66
(NL) Gebruiksaanwijzing	26	(RO) Instrucțiuni de utilizare	69
(SE) Bruksanvisning	30	(TR) Kullanma talimatı	73
(DK) Brugervejledning	33	(KR) 사용 설명서	76
(NO) Bruksanvisning	36	(BG) Инструкции за употреба ..	79
(FI) Käyttöohjeet	39	(ID) Petunjuk penggunaan	83
(GR) Οδηγίες χρήσης	42	(SI) Navodila za uporabo	87
(CZ) Návod k použití	46	(RS) Uputstva za upotrebu	91

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Stand: 2016-01

Signum zirconia bond is a zirconium dioxide resin adhesive for crowns and bridges.

Composition:

Signum zirconia bond I: acetone, bifunctional methacrylate-based components.

Signum zirconia bond II: MMA, initiators.

Indication:

- Adhesive primer for frameworks, implant structures and single crowns made of ZrO_2 for veneering with high-content veneer composite for a long term provisional solution.
- Intraoral and extraoral repairs in addition using Venus or Charisma.
- Used to condition the surface of Zirconia Oxide frameworks permitting the adhesion of resin cement.

Processing:

Laboratory:

Production of a veneer with a high-content veneer composite on copings and frameworks made of zirconium dioxide.

- Clean the object to be veneered with oil-free compressed air or clean tap water and carefully dry it to obtain a grease and dust-free surface.

- Signum zirconia bond I is dispensed into a dapping dish for each individually repaired unit. It is then applied with a suitable brush to the entire surface.
- Then Signum zirconia bond II is applied in the same way and light cured for 90 seconds with a suitable polymerisation device such as Heraflash HiLite Power or Uni XS.
- The prepared unit can then be prepared with a standard laboratory composite like Signum ceramis.

Dentist:

Repair and restoration of a veneered zirconium dioxide framework.

- The damaged area of zirconia ceramic needs to be abraded to the framework and cleaned using water and air (dental dam is recommended). **Attention:** For conditioning of silicate-ceramic (veneering) we recommend to use Signum ceramic bond according to instruction for use.
- The clean and prepared area needs to have Signum zirconia bond I applied using a suitable brush and carefully dried. Signum zirconia bond II is then applied in a thin layer with the appropriate brush and light cured with a suitable polymerisation light (Translux®) for 40 seconds.
- Now the damaged or broken site can be repaired with a composite material (Charisma or Venus).

Contraindication:

- If reliable drying is not possible in intraoral use.
- If the patient has a proven allergy to one of the ingredients.

Hazards/safety instructions:

Signum zirconia bond I

Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store locked up.

Signum zirconia bond II

Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat/sparks/open flames/hot surfaces. – No smoking. Use explosion-proof electrical/ventilating/lighting/equipment. Avoid breathing dust/fume/gas/mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Store locked up.

Safety data sheets and more information are available at our website www.kulzer-dental.com

Shelf life:

The maximum shelf life is printed on the label of the bottle. Do not use after the expiration date.

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® = registered trademark of Kulzer GmbH
® = registrovana robna oznaka Kulzer GmbH
Kulzer GmbH
Kulzer, LLC