## SPEIKOSEPT CHX 2 %-Gel

---

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---

### Composition:

Contains 2% chlorhexidine digluconate.

### Fields of application:

- For cleaning contaminated root canal instruments during root canal treatments
- For dental use only

### Contra-indications:

- Hypersensitivity to one of the ingredients

### Side effects/interactions:

None known if used as intended.

### Dosage, type and duration of treatment:

Put a sufficient amount of the gel into a suitable container, e.g. a stand for endo files.

The instruments used during treatment are placed in the container and simultaneously cleaned when they come into contact with the gel.

Thanks to the antibacterial effect of the chlorhexidine vis-à-vis most undesired endobacteria and fungi, the bacterial count is also greatly reduced. Additional rinsing is not necessary. The endo instruments can be directly reused during treatment.

Following treatment, the instruments and containers which are not designed for disposable use must be reprocessed as specified (cleaning, disinfection, sterilisation etc.).

### Notes:

Chlorhexidine-based products may not be mixed with sodium hypochlorite-based products as otherwise a not readily soluble, red brown sediment forms.

This medical device may no longer be used after the expiry date.

Reseal the syringe tightly after use and store at room temperature.

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