PPE Specification
Labeling Specification
389694R07 SURGICEL Fibrillar / TABOTAMP Fibrillar CE Marked
IFU

LAB-0010677 | Rev:7
Released: 22 Jun 2020
CE Marked
Release Level: 4. Production

SURGICEL™

FIBRILLAR

TABOTAMP™

FIBRILLAR

Absorbable Haemostat (oxidised regenerated cellulose)
Resorbierbares Hämostatikum (Oxidierter, regenerierter Zellulose)
Hémostatique résorbable (cellulose régénérée oxydée)
Emostatico assorbibile (cellulosa ossidata rigenerata)
Hemostático absorbible (celulosa regenerada oxidada)
Absorberbart hemostatikum (Oxidert regeneriert cellulosa)
Absorbeerbaar haemostaticum (geoxideerde geregenereerde cellulose)
Hemostático Absorvível (celulose regenerada oxidada)
Imukykyinen hemostaatti (hapetettua regeneroitua selluloosaa)
Resorberbart hæmostat (oxidert regenerert cellulose)
Απορροφήσιμο Αιμοστατικό (οξειδωμένη αναγεννηθείσα κυτταρίνη)
Absorberbar hemostat (oksidert regenerert cellulose)
Wchłaniany materiał hemostatyczny
(celuloza regenerowana w procesie utleniania)
Felszívódó vérzéscsillapító (oxidált regenerált cellulóz)
Vstřebateľný hemostat (oxidovaná regenerovaná celulóza)
Vstřebateľné hemostatikum (oxidovaná regenerovaná celulóza)
Sredstvo za zaustavitev krvavitve (oksidirana regenerirana celuloza)
Absorbējošs hemostatisks līdzeklis (oksidēta reģenerēta celuloze)
Sugeriamašis hemostatikas (oksiduota perdirbta celiuliozė)
Resorbeeruv hemostaatikum (oksideeritud regeneeritud tselluloos)
Emilebilir Hemostat (oksidize rejenerer selüloz)
止血膠棉 (氧化再生纖維素)
Рассасывающийся гемостатический материал
(окисленная восстановленная целлюлоза)

STERILE R

ETHICON™

PPE Specification
Labeling Specification
389694R07 SURGICEL Fibrillar / TABOTAMP Fibrillar
IFU

LAB-0010677 | Rev:7
Released: 22 Jun 2020
CE Marked
Release Level: 4. Production

PPE Specification
Labeling Specification
389694R07 SURGICEL Fibrillar / TABOTAMP Fibrillar
IFU

LAB-0010677 | Rev:7
Released: 22 Jun 2020
CE Marked
Release Level: 4. Production

English	Page 05
Deutsch	Seite 08
Français	Page 11
Italiano	Pagina 14
Español	Página 17
Svenska	Sidan 20
Nederlands	Pagina 23
Português	Página 26
Suomi	Sivu 29
Dansk	Side 32
Ελλάς	Σελίς 35
Norsk	Side 38
Polski	Strona 41
Magyar	44. oldal
Čeština	Strana 47
Slovenčina	Strana 50
Slovenščina	Stran 53
Latviešu	Lapa 56
Lietuvių	Puslapis 59
Eesti	Lehekülj 62
Türkçe	sayfa 65
中文	页 68
Русский	Страница 71
Symbols	Page 74

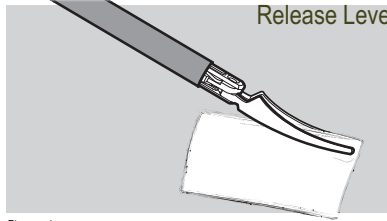
Figure 1

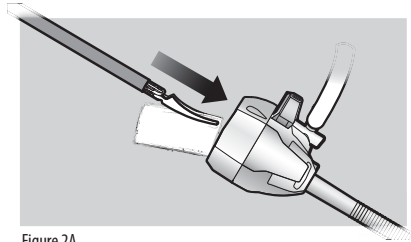
Figure 2A

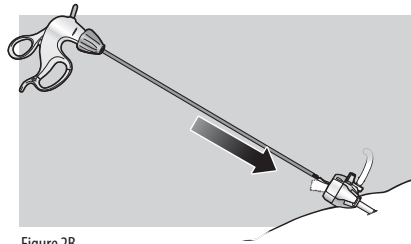
Figure 2B

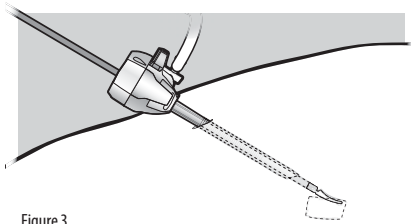
Figure 3

PPE Specification
Labeling Specification

LAB-0010677 | Rev:7
Released: 22 Jun 2020

38604R07 SURGICEL Fibrillar / TABOTAMP Fibrillar CE Marked
IFU Release Level 2 Production

SURGICEL™ FIBRILLAR Absorbable Haemostat is a sterile absorbable absorbent prepared by the controlled oxidation of regenerated cellulose. The fabric is white with a pale yellow cast and has a faint, caramel-like aroma. It is strong and can be sutured or cut without fraying. It is stable and should be stored at controlled room temperature. A slight discolouration may occur with age, but this does not affect performance.

The fibrillar form of the product allows the surgeon to grasp with forceps any amount of SURGICEL™ FIBRILLAR Haemostat needed to achieve haemostasis at a particular bleeding site. This makes it convenient for hard to reach or irregularly shaped bleeding sites. Although it is easy to pull the desired amount of SURGICEL™ FIBRILLAR Haemostat from the entire supply, the group of selected fibres continue to cohere to one another and application to the bleeding site is easily controlled. Unwanted dispersal over the operative site does not occur.

Actions

The mechanism of action whereby SURGICEL™ FIBRILLAR Haemostat accelerates clotting is not completely understood, but it appears to be a physical effect rather than any alteration of the normal physiologic clotting mechanism. After SURGICEL™ FIBRILLAR Haemostat has been saturated with blood, it swells into a brownish or black gelatinous mass which aids in the formation of a clot, thereby serving as a haemostatic adjunct in the control of local haemorrhage. When used properly in minimal amounts, SURGICEL™ FIBRILLAR Haemostat is absorbed from the sites of implantation with practically no tissue reaction. Absorption depends upon several factors including the amount used, degree of saturation with blood, and the tissue bed.

In addition to its local haemostatic properties, SURGICEL™ FIBRILLAR Haemostat is bactericidal *in vitro* against a wide range of gram positive and gram negative organisms including aerobes and anaerobes. SURGICEL™ FIBRILLAR Haemostat is bactericidal *in vitro* against strains of species including those of:

<i>Staphylococcus aureus</i>	<i>Bacillus subtilis</i>
<i>Staphylococcus epidermidis</i>	<i>Proteus vulgaris</i>
<i>Micrococcus luteus</i>	<i>Corynebacterium xerosis</i>
<i>Streptococcus pyogenes</i> Group A	<i>Mycobacterium phlei</i>
<i>Streptococcus pyogenes</i> Group B	<i>Clostridium tetani</i>
<i>Streptococcus salivarius</i>	<i>Clostridium perfringens</i>
<i>Branhamella catarrhalis</i>	<i>Bacteroides fragilis</i>
<i>Escherichia coli</i>	<i>Enterococcus</i>
<i>Klebsiella aerogenes</i>	<i>Enterobacter cloacae</i>
<i>Lactobacillus</i> sp.	<i>Pseudomonas aeruginosa</i>
<i>Salmonella enteritidis</i>	<i>Pseudomonas stutzeri</i>
<i>Shigella dysenteriae</i>	<i>Proteus mirabilis</i>
<i>Serratia marcescens</i>	
methicillin-resistant <i>Staphylococcus aureus</i> (MRSA)	
penicillin-resistant <i>Streptococcus pneumoniae</i> (PRSP)	
vancomycin-resistant <i>Enterococcus</i> (VRE)	
methicillin-resistant <i>Staphylococcus epidermidis</i> (MRSE)	

Studies conducted in animals show that SURGICEL™ FIBRILLAR Haemostat in contrast to other haemostatic agents does not enhance experimental infection. SURGICEL™ FIBRILLAR Haemostat is not, however, an alternative to systemically applied therapeutic or prophylactic antimicrobial agents.

SURGICEL™ FIBRILLAR Haemostat can be used in many areas of surgery, e.g. cardiovascular surgery, haemorrhoidectomy, implantation of vascular prostheses, biopsies, lung operations, surgery to the face and jaw, gastric resection, operations on the throat or nose, liver and gall bladder operations, gynaecological operations, thoracic and abdominal sympathectomies, neurosurgery, especially cerebral operations, thyroid operations, skin transplantations, and treatment of superficial injuries.

Indications

SURGICEL™ FIBRILLAR Haemostat is used adjunctively in surgical procedures to assist in the control of capillary, venous, and small arterial haemorrhage when ligation or other conventional methods of control are impractical or ineffective. SURGICEL™ FIBRILLAR Haemostat can be cut to size for use in endoscopic procedures.

Contraindications

- Although packing or wadding sometimes is medically necessary, SURGICEL™ FIBRILLAR Haemostat should not be used in this manner, unless it is to be removed after haemostasis is achieved. (See Warnings and Precautions).
- SURGICEL™ FIBRILLAR Haemostat should not be used for implantation in bone defects, such as fractures, since there is a possibility of interference with callus formation and a theoretical chance of cyst formation.
- When SURGICEL™ FIBRILLAR Haemostat is used to help achieve haemostasis in, around, or in proximity to foramina in bone, areas of bony confine, the spinal cord, or the optic nerve and chiasm, it must always be removed after haemostasis is achieved since it will swell and could exert unwanted pressure.
- SURGICEL™ FIBRILLAR Haemostat should not be used to control haemorrhage from large arteries.

Warnings

- SURGICEL™ FIBRILLAR Haemostat is supplied sterile and as the material is not compatible with autoclaving or ethylene oxide sterilization SURGICEL™ FIBRILLAR Haemostat should not be re-sterilised.
- SURGICEL™ FIBRILLAR Haemostat is not intended as a substitute for careful surgery and the proper use of sutures and ligatures.
- Closing SURGICEL™ FIBRILLAR Haemostat in a contaminated wound may lead to complications and should be avoided.
- The haemostatic effect of SURGICEL™ FIBRILLAR Haemostat is greater when it is applied dry; therefore it should not be moistened with water or saline.
- SURGICEL™ FIBRILLAR Haemostat should not be impregnated with anti-infective agents or with other materials such as buffering or haemostatic substances. Its haemostatic effect is not enhanced by the addition of thrombin, the activity of which is destroyed by the low pH of the product.
- Although SURGICEL™ FIBRILLAR Haemostat may be left *in situ* when necessary, it is advisable to remove it once haemostasis is achieved. It must **always** be removed from the site of application when used in, around, or in proximity to foramina in bone, areas of bony confine, the spinal cord, and/or the optic nerve and chiasm, and in proximity to tubular structures that could become constricted by swelling, regardless of the type of surgical procedure because SURGICEL™ FIBRILLAR Haemostat, by swelling, may exert pressure resulting in paralysis and/or nerve damage. Dislodgement of SURGICEL™ FIBRILLAR Haemostat could possibly occur by means such as repacking, further intraoperative manipulation, lavage, exaggerated respiration, etc. There have been reports that in procedures such as lobectomy, laminectomy and repair of a frontal skull fracture and lacerated lobe that SURGICEL™ FIBRILLAR Haemostat, when left in the patient after closure, migrated from the site of application into foramina in bone around the spinal cord resulting in paralysis and, in another case, the left orbit of the eye, causing blindness. While these reports cannot be confirmed, special care must be taken by physicians, **regardless of the type of surgical procedure**, to consider the advisability of removing SURGICEL™ FIBRILLAR Haemostat after haemostasis is achieved.
- Although SURGICEL™ FIBRILLAR Haemostat is bactericidal against a wide range of pathogenic microorganisms, it is not intended as a substitute for systemically administered therapeutic or prophylactic antimicrobial agents to control or prevent post-operative infections.

Precautions

- Use only as much SURGICEL™ FIBRILLAR Haemostat as is necessary for haemostasis, holding it firmly in place until bleeding stops. Remove any excess before surgical closure in order to facilitate absorption and minimize the possibility of foreign body reaction, such as encapsulation of the product, which may mimic artifacts on radiographic images, resulting in diagnostic errors and possible reoperation.
- In urological procedures, minimal amounts of SURGICEL™ FIBRILLAR Haemostat should be used and care must be exercised to prevent plugging of the urethra, ureter, or a catheter by dislodged portions of the product.
- Since absorption of SURGICEL™ FIBRILLAR Haemostat could be prevented in chemically cauterized areas, its use should not be preceded by application of silver nitrate or any other escharotic chemicals.
- If SURGICEL™ FIBRILLAR Haemostat is used temporarily to line the cavity of large open wounds, it should be placed so as not to overlap the skin edges. It should also be removed from open wounds by forceps or by irrigation with sterile water or saline solution after bleeding has stopped.
- Precautions should be taken in otorhinolaryngologic surgery to assure that none of the material is aspirated by the patient. (Examples: controlling haemorrhage after tonsillectomy and controlling epistaxis).
- Care should be taken not to apply SURGICEL™ FIBRILLAR Haemostat too tightly when it is used as a wrap during vascular surgery (see Adverse Reactions).

Directions for use in endoscopic procedures:

SURGICEL™ FIBRILLAR Haemostat can be cut to size for use in endoscopic procedures (see Figures 1, 2A, 2B, and 3 on page 04).

Figure 1. SURGICEL™ FIBRILLAR Haemostat should be cut to the appropriate size for endoscopic placement. Standard endoscopic procedures should be used up to the point of placement of the absorbable haemostat. Grasp the SURGICEL™ FIBRILLAR Haemostat at one corner.

Figures 2A and 2B. Slowly push the grasping instrument and material into the cavity.

Figure 3. With the use of grasping instruments in a second and/or third auxiliary site, placement can be made and the material repositioned as needed.

Adverse reactions

- "Encapsulation" of fluid and foreign body reactions have been reported.
- There have been reports of stenotic effect when SURGICEL™ FIBRILLAR Haemostat has been applied as a wrap during vascular surgery. Although it has not been established that the stenosis was directly

PPE Specification
Labeling Specification

LAB-0010677 | Rev:7
Released: 22 Jun 2020

38969407 SURGICEL Fibrillar Haemostat / TABOTAMP Fibrillar Haemostat
IFU

Release Level: 4 Production

Paralysis and nerve damage have been reported when SURGICEL™ FIBRILLAR Haemostat was applied around, in, or in proximity to foramina in bone, areas of bony confine, the spinal cord, and/or the optic nerve and chiasm. While most of these reports have been in connection with laminectomy, reports of paralysis have also been received in connection with other procedures. Blindness has been reported in connection with surgical repair of a lacerated left frontal lobe when SURGICEL™ FIBRILLAR Haemostat was placed in the anterior cranial fossa (see Warnings and Precautions).

- Possible prolongation of drainage in cholecystectomies and difficulty passing urine per urethra after prostatectomy have been reported. There has been one report of a blocked ureter after kidney resection, in which postoperative catheterization was required.
- Occasional reports of "burning" and "stinging" sensations and sneezing when SURGICEL™ FIBRILLAR Haemostat has been used as packing in epistaxis, are believed to be due to the low pH of the product.
- Burning has been reported when SURGICEL™ FIBRILLAR Haemostat was applied after nasal polyp removal and after haemorrhoidectomy. Headache, burning, stinging and sneezing in epistaxis and other rhinological procedures. Also stinging when SURGICEL™ FIBRILLAR Haemostat was applied on surface wounds (varicose ulcerations, dermabrasions and donor sites) have been reported.

Dosage and administration

Sterile technique should be observed in removing SURGICEL™ FIBRILLAR Haemostat from its sterile container. Minimal amounts of SURGICEL™ FIBRILLAR Haemostat in appropriate size are laid on the bleeding site or held firmly against the tissues until haemostasis is obtained.

The amount required depends on the nature and intensity of the haemorrhage to be stopped. The haemostatic effect of SURGICEL™ FIBRILLAR Haemostat is particularly pronounced when used dry. Moistening the material with water or physiological saline solution is not recommended.

Storage and shelf-life

SURGICEL™ FIBRILLAR Haemostat should be stored dry under controlled conditions (15°C-30°C) and protected from direct sunlight, in the original packaging.

Do not use if individual pack damaged/opened. Do not re-sterilise. The supplied SURGICEL™ FIBRILLAR Haemostat is available sterile as an original pack in variable sizes. The use by date of this product is printed on the packaging. Do not use SURGICEL™ FIBRILLAR Haemostat after this date.

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© ETHICON, INC. 2019
ETHICON, LLC
Johnson & Johnson Medical GmbH