# VITA INTERNO®

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# VITA INTERNO®

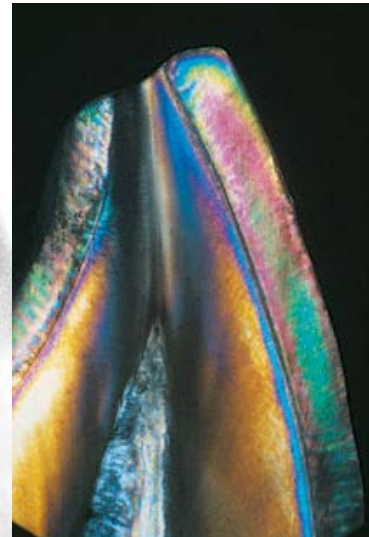
For natural, in-depth colour effects

VITA – perfect match.

**VITA**

**The VITA INTERNO ceramics enable the ceramist to achieve an even more perfect reproduction of in-depth shade effects.**

	VITA INTERNO <b>white smoke - Int 01</b>
	VITA INTERNO <b>sand - Int 02</b>
	VITA INTERNO <b>lemon chrome - Int 03</b>
	VITA INTERNO <b>sunflower - Int 04</b>
	VITA INTERNO <b>coral blush - Int 05</b>
	VITA INTERNO <b>gold earth - Int 06</b>
	VITA INTERNO <b>misty lilac - Int 07</b>
	VITA INTERNO <b>marina - Int 08</b>
	VITA INTERNO <b>cream pearl - Int 09</b>
	VITA INTERNO <b>burnt orange - Int 10</b>
	VITA INTERNO <b>clove - Int 11</b>
	VITA INTERNO <b>splinter - Int 12</b>
	VITA INTERNO <b>lilac - Int 13</b>
	VITA INTERNO <b>khaki - Int 14</b>

**The VITA INTERNO® Assortment contains:**

<b>Material</b>	<b>Order No.</b>	<b>Quantity</b>
<b>INTERNO ceramics</b>	<b>Int 01 - Int 12</b>	<b>12 x 3 g</b>
<b>INTERNO FLUID</b>		<b>1 x 20 ml</b>
<b>AKZENT / INTERNO mixing plate</b>		<b>1 piece</b>
<b>Glass spatula</b>		<b>1 piece</b>
<b>a&amp;e brush No. 3/0</b>		<b>1 piece</b>
<b>Shade sample blade</b>		<b>1 piece</b>
<b>Working instructions</b>	<b>Nr. 770E</b>	<b>1 piece</b>

# VITA INTERNO® Working Instructions

## Area of applications

The porcelains of this assortment are shaded according to the anomalies occurring in natural teeth. These porcelains enable the user to reproduce natural shade effects and obtain an even more perfect match of ceramic restorations to natural teeth.

VITA INTERNO porcelains can be used with all types of ceramic techniques in combination with the following products: VITA VM 7, VM 9, VM 13, VM 15, VITA VMK Master, VMK 95 and VITA TITANIUM CERAMICS. These stains, however, are not intended for shading opaque materials.

## Properties

The VITA INTERNO porcelains are fine-grained and strongly fluorescent and are distinguished by their shade intensity. It is possible to mix these with one another. They can be used either on their own or mixed into opaque dentine, dentine, enamel and translucent porcelains.

## Working procedure

When mixed with opaque dentine, the VITA INTERNO porcelains increase the fluorescence and natural luminosity of a ceramic restoration in varying light conditions. When INTERNO porcelains are mixed with dentine, the shade character of the porcelain can be altered. The INTERNO/dentine mixture (Int 01-03 and Int 09) is particularly suitable for mamelon effects. Due to their fine-grained structure, the INTERNO porcelains are also very suitable for washing into the dentine, base dentine or opaque dentine in their unmixed state to achieve in-depth shade effects.

A mixture of translucent and INTERNO porcelains (e.g. Int 01, Int 03, Int 08) enhances the translucent effect in the incisal area.

Using the INTERNO porcelain Int 12 (splinter), either on its own or mixed with other INTERNO porcelains, it is possible to reproduce natural enamel cracks. The INTERNO porcelains Int 10 and Int 11 have no fluorescence; these are particularly suitable for reproducing the effects resulting from dentine hardening in abraded teeth. The firing temperature is that of the dentine firing of the ceramic material used.

## Important

When grinding dental ceramic products it is necessary to use a suction unit, wear a dust mask or keep the workpiece wet.

**Outstanding results can be obtained with Interno ceramics, particularly when reproducing colour effects (contrasts) occurring in the incisal area of natural teeth.**

Their high fluorescence means shades with increased luminosity and even better light transmission - just as in natural dentition.

VITA INTERNO ceramics can be mixed with dentine and translucent porcelains. This intensifies the chromatic effect.

In special cases the INTERNO ceramics can be used to create an additional impression of depth when space is lacking.

**Study on  
natural teeth:**

Typical shade reflection  
on the incisal edge

Clearly visible internal  
structures in the dentine of  
a natural tooth after removing  
the enamel layer

The cross-section  
illustrates the diversity  
of colour effects within  
the dentine

**Various factors alter the structures in natural teeth. Contrasts and colour effects are to be found particularly in the teeth of older patients.**

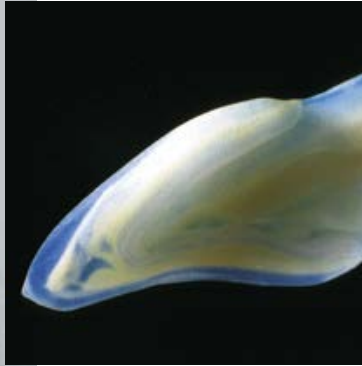
**Example of the application of a mixture of INTERNO and transparent porcelain to imitate orange incisal shading. The INTERNO mixture should be "washed" into a well-moistened ceramic. It is also important to make sure the INTERNO ceramics are inlaid or "washed in" only in a very thin layer.**

Lifelike impression of depth with INTERNO ceramics in the crown.

**Observations on natural dentition:  
The intense luminosity of the  
INTERNO ceramics matching the  
highly fluorescent effect of  
natural teeth:**

above:

VITA VM 7 crown

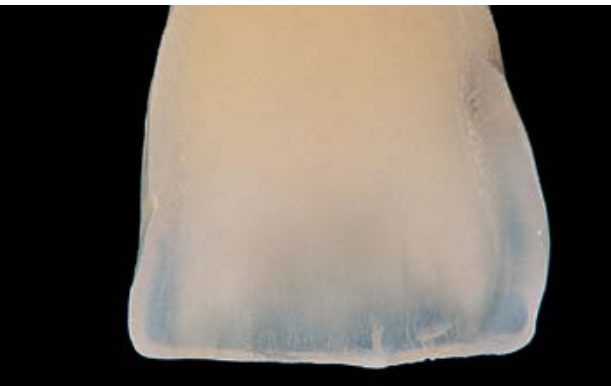
centre:

A natural tooth;

below:

Fluorescence of a VITA VM 7 crown  
with INTERNO ceramics.

**Bright reflection in a natural tooth.**

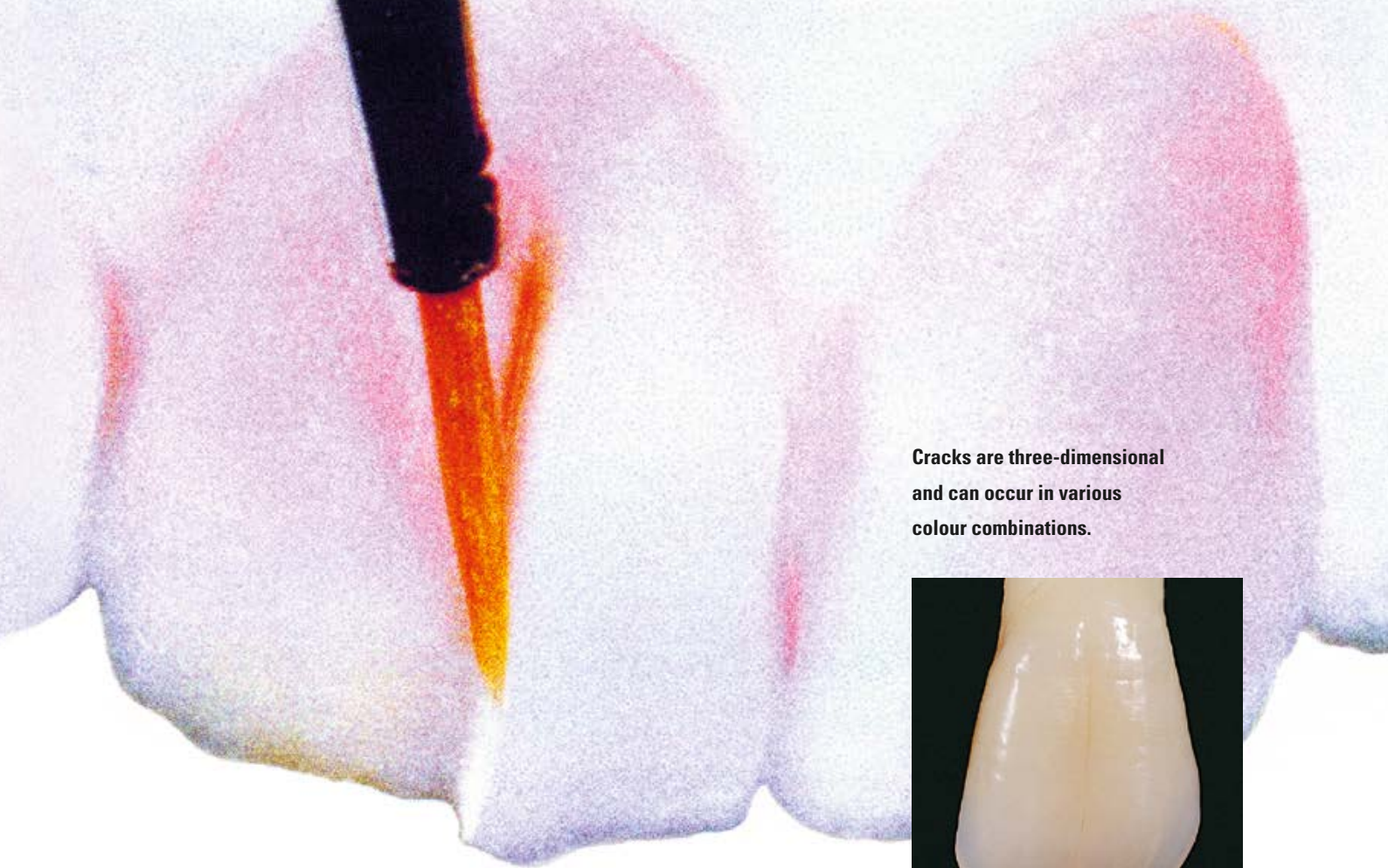
**Three case examples**

**The INTERNO ceramics demonstrate remarkably lifelike interdigital and incisal effects in the lateral incident light.**

**Cracks are three-dimensional and can occur in various colour combinations.**

**By mixing with other INTERNO ceramics, the shade of INTERNO 12 (splinter) can also be altered, and individualised.**

**Restoration with  
VITA INTERNO – the results  
speak for themselves.**

The VITA INTERNO ceramics were developed and created in close co-operation with the ceramist Claude Sieber, Basel/Switzerland. The concept of this brochure bears his stamp, the photographs and patient cases portray his philosophy.

## Recommended firing for:

VITAVM®7	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	500	6.00	7.27	55	910	1.00	7.27

VITAVM®9	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	500	6.00	7.27	55	910	1.00	7.27

VITAVM®13	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	500	6.00	6.55	55	910	1.00	6.55

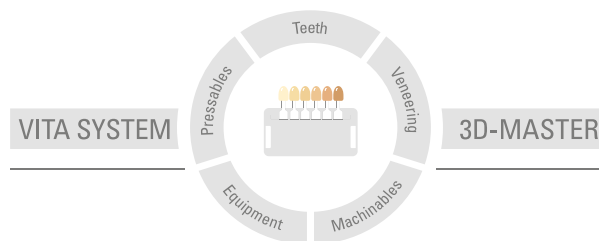
VITAVM®15	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	400	6.00	8.00	50	800	1.00	8.00

VITA VMK Master®	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	500	6.00	7.49	55	930	1.00	7.49

VMK 95	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	600	6.00	6.00	55	930	1.00	6.00

VITA TITANKERAMIK	Pre-drying °C	→ min.	↗ min.	↗ °C/min.	Temp. approx. °C	→ min.	VAC min.
Dentine firing	400	6.00	7.00	53	770	1.00	8.00

With the unique VITA SYSTEM 3D-MASTER all natural tooth shades are systematically determined and completely reproduced.

**Please note:** Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product. Furthermore, our liability for the accuracy of this information is independent of the legal basis and, in as far as legally permissible, shall always be limited to the value as invoiced of the goods supplied, excluding value-added tax. In particular, as far as legally permissible, we do not assume any liability for loss of earnings, indirect damages, ensuing damages or for third-party claims against the purchaser. Claims for damages based on fault liability (culpa in contrahendo, breach of contract, unlawful acts, etc.) can only be made in the case of intent or gross negligence. The VITA Modulbox is not necessarily a component of the product. Date of issue of this product information: 09.17

After the publication of these working instructions any previous versions become obsolete. The current version can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

VITA Zahnfabrik is certified in accordance with the Medical Device Directive and the following products mentioned in this brochure bear the mark **CE** 0124 :

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VITA Zahnfabrik H. Rauter GmbH & Co.KG