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DIRECTIONS FOR USE

■ For Opaque porcelain

1. Mix Opaque porcelain powder with VINTAGE ANTI-GREENING LIQUID FOR OPAQUE to obtain optimal consistency.
2. When the mixture loses optimum moisture, add some distilled water for alteration.
3. When building up opaque porcelain additionally, apply VINTAGE ANTI-GREENING LIQUID FOR OPAQUE instead of distilled water before subsequent build-up.

■ For Body porcelain

1. Mix Body porcelain, Incisal porcelain or Translucent porcelain powder with VINTAGE ANTI-GREENING LIQUID FOR BODY to obtain optimal consistency.
2. When the mixture loses optimum moisture, add some distilled water for alteration.
(Note) Do not use VINTAGE ANTI-GREENING LIQUID FOR BODY to regain moisture.
3. When building up additionally, apply VINTAGE ANTI-GREENING LIQUID FOR BODY instead of distilled water before subsequent build-up.
4. After contouring, rinse with water and dry the surface, upon which VINTAGE ANTI-GREENING LIQUID FOR BODY is applied. Then, perform self-glazing or apply stain or glaze porcelain.
5. When performing post-soldering or re-firing, apply VINTAGE ANTI-GREENING LIQUID FOR BODY thinly to the interface between porcelain and metal or porcelain surface.

NOTE ON USE

1. VINTAGE ANTI-GREENING LIQUID is designed exclusively for silver-containing alloys. Use opaque liquid or distilled water when using other materials.
2. Marginal shrinkage occurs more frequently when using VINTAGE ANTI-GREENING LIQUID comparing with distilled water. Exercise caution to minimize marginal shrinkage.
3. Tightly close the bottle cap after each use to avoid evaporation.
4. Do not use this product for any purposes other than specifically outlined in these instructions for use.

INDICATIONS FOR USE

Color resistant porcelain mixing liquid for silver-containing alloys

PACKAGE

VINTAGE ANTI-GREENING LIQUID FOR BODY 50 mL

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SHOFU DENTAL GmbH

1225 Stone Drive, San Marcos, California 92078-4059, USA

10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

An der Pönt 70, 40885 Ratingen, Germany

Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/

72461

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SHOFU DENTAL ASIA-PACIFIC PTE.LTD.