- en** Instructions for use
- ru** Инструкция по применению
- de** Gebrauchsanleitung
- pl** Instrukcja obsługi
- fr** Mode d'emploi
- ro** Instrucțiuni de utilizare
- nl** Gebruiksaanwijzing
- sv** Bruksanvisning
- es** Instrucciones de uso
- el** Οδηγίες χρήσης
- it** Istruzioni per l'uso
- pt** Instruções de uso

03.0:2020-09
72486

Manufacturer
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.
10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

EC REP
SHOFU DENTAL GmbH
An der Pönt 70, 40885 Ratingen, Germany

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en Metalbond Porcelain READ CAREFULLY BEFORE USE

PRODUCT DESCRIPTIONS
VINTAGE PRO is a metal bond porcelain system indicated for veneering of precious, semi-precious and non-precious alloys as well as palladium-based alloys with a CTE range of 13.5 to 14.8 x 10⁻⁶ K⁻¹. VINTAGE PRO is also suitable for metal frameworks made with CAD/CAM and electroforming/sintering. The shade system includes VITA® classical shades with a variety of effect shades to reproduce a wide range of shade.

INDICATIONS FOR USE
Build-up porcelain for fabrication of metal bonded ceramic restorations

- PRECAUTIONS AND WARNINGS**
- If any inflammation or other allergic reactions occur on the operator while using this product, immediately discontinue use and seek medical advice.
 - Avoid contact with skin or eyes. In case of accidental contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
 - Use local dust extractor or mask etc, while grinding or polishing this product to avoid harmful influence of dust on the human body.
 - Wear protective glasses etc, while grinding or polishing this product to avoid damage to eyes.
 - Tightly close the bottle cap after each use.
 - Follow the instructions for use of each dental material, instrument or equipment to be used in conjunction with this product.
 - Do not use this product for any purposes other than specifically outlined in these instructions for use.
 - Use this product within the expiration date indicated on the package and container. (Example ↗ YYYY-MM-DD → Year-Month-Date of the expiration date)
 - This product is intended for use by dental professionals only.

- DIRECTIONS FOR USE**
Refer to the separate user's manual or detailed instructions for use.
- Use VINTAGE PRO in conjunction with alloys having coefficient of thermal expansion of 13.5 to 14.8 x 10⁻⁶ K⁻¹.
 - The firing conditions may vary due to the different designs and operating voltages of the porcelain furnaces. It is essential to carry out test firings before using the porcelain for actual restorations.
 - Apply the first Opaque layer (BASE) in a thin layer and fire.
 - The second layer of Opaque must be applied evenly in a sufficient thickness to mask the color of the underlying metal coping.
 - Mix Opaque pastes sufficiently well with a plastic spatula before use.
 - Opaque porcelains must be mixed with dedicated VINTAGE PRO opaque mixing liquids only. Do not mix with other VINTAGE mixing liquids.
 - When Opaque porcelain is applied generously, set longer drying time at the furnace entrance and preheating time in order to burn the liquid components thoroughly.
 - Select the powders based on the combination chart and build up the restoration.
 - The Body build-up should be done to the final anatomical shape of the tooth in order to ensure the exact dimension for the Enamel porcelain.
 - When large amount of porcelain is built up, set longer drying time at the furnace entrance in order to avoid explosion of the porcelain from the inside resulting from remaining liquid components.
 - Enamel porcelain should be built up slightly oversized in consideration of firing shrinkage.
 - When the area requiring an additional build-up is dry, wet the area with water by using a brush.
 - Use distilled water or VINTAGE MARGIN HARDENING LIQUID for Margin porcelains.
 - Use distilled water or VINTAGE CPM MODELLING LIQUID for ADD-ON porcelains: ADD-ON B and ADD-ON T. Do not use VINTAGE MIXING LIQUID or MIXING LIQUID HC for these porcelains since they affect the shade of the restoration.

STORAGE
Store at room temperature (1-30 °C/34-86 °F). Avoid direct sunlight.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.
Location of the symbols glossary: https://www.shofu.com/en/symbols_glossary/
* VITA is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen, Germany

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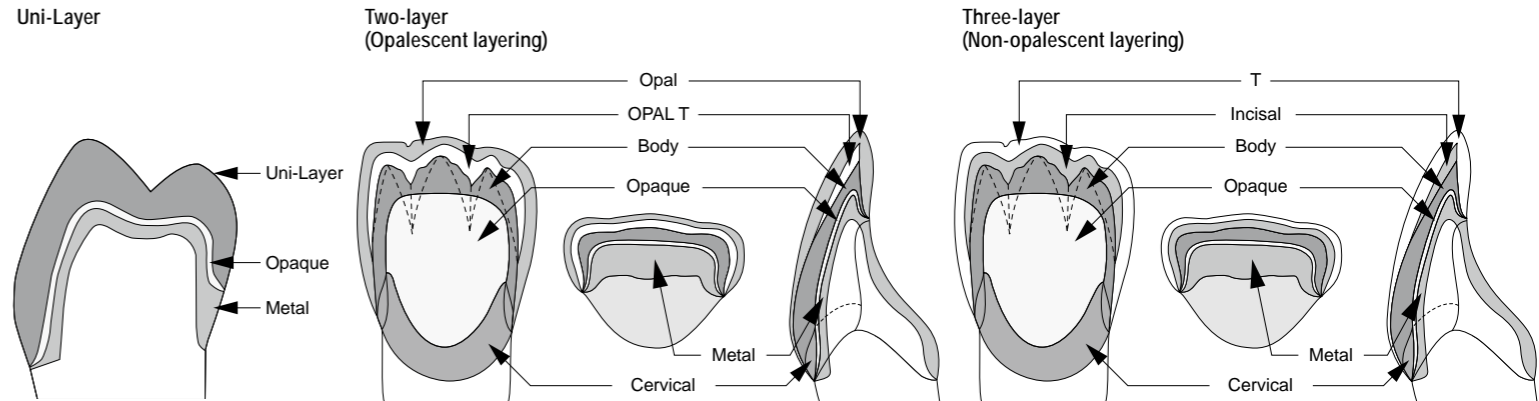
4. PHYSICAL PROPERTIES

	Type of firing	Coefficient of thermal expansion (25 - 500 °C)	Glass transition point
Base Opaque	2 nd firing	11.5 x 10 ⁻⁶ K ⁻¹	598°C
	4 th firing		
Shade Opaque	2 nd firing	12.4 x 10 ⁻⁶ K ⁻¹	594°C
	4 th firing		
Margin	2 nd firing	13.0 x 10 ⁻⁶ K ⁻¹	594°C
	4 th firing		
Cervical, Body, Opaque Dentin, Opal, Incisal, Opal Effect, Enamel Effect, Uni-Layer, Color Effect, Gum	2 nd firing	13.0 x 10 ⁻⁶ K ⁻¹	580°C
	4 th firing		
Correction	2 nd firing	11.0 x 10 ⁻⁶ K ⁻¹	585°C
	4 th firing		

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* VITA är ett registrerat varumärke som tillhör VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen, Tyskland