VOCO Admira® Bond

GB

CE 0482

Instructions for use

Product description:

In general, minimal preparation according to the rules of the adhesive dentine-enamel bond. **Admira Bond** contains specially developed adhesive ORMOCER®s as well as additive aliphatic and aromatic dimethacrylates. ORMOCER®s as a material class are protected by patent. **Admira Bond** cures under halogen light/LED light (blue light).

Indications:

Admira Bond is suited for direct restorations of all filling classes using light-curing restorative material.

Directions:

Cavity preparation:

Cleaning and drying of the cavity is carried out with special preparations or hydrogen peroxide solution. Due to the subsequent application of light-curing restorative material, drying of the working field is required. Prevent any contamination of the cavity with blood or saliva.

Cleaning/Drying:

Cleaning and drying of the cavity is carried out with special preparations or hydrogen peroxide solution. Due to the subsequent application of light-curing restorative material, drying of the working field is required. Prevent any contamination of the cavity with blood or saliva.

Lining:

A calcium hydroxide lining should be applied in close proximity to the pulp. A lining is recommended for very deep cavities. Due to their fluoride release, glass ionomer cements are particularly suitable for this purpose.

Beveling/etching:

Use **Admira Bond** in the "total-etch technique". Bevel the enamel and then etch the entire cavity, starting from the enamel margin (enamel approx. 20 - 30 s, dentine for a maximum of 15 s with **Vococid**). In case of fluoridated enamel or enamel with a low density of prisms, prolong the etching time **on the enamel only** accordingly (up to 90 s). Then remove etchant, rinse for 20 s and remove excess moisture with the air syringe. The dentine surface should remain slightly moist (not too dry) and the enamel surface should have a chalky white appearance.

Application of Admira Bond:

Dispense **Admira Bond** onto a **Single Tim** or a disposable brush. As an alternative, dispense **Admira Bond** onto a mixing pallet. Apply **Admira Bond** evenly on all prepared enamel and dentine surfaces using a **Single Tim** or a disposable brush and allow to act for 30 s. Then dispense **Admira Bond** with a faint airjet and polymerise with halogen light/LED light for 20 s. Conventional polymerisation devices are suited for light-curing this material. The light output should be a minimum of 500 mW/cm² for halogen polymerisation devices and 300 mW/cm² for LED devices. Do not remove or contaminate the inhibition layer since it is important for the adhesion to the restorative material.

Filling:

Use the restorative material according to the manufacturer's instructions.

Information, precautionary measures:

- There are no known side effects. However, hypersensitive persons may have a reaction.
- **Admira Bond** contains ORMOCER®, Bis-GMA, HEMA, BHT, acetone and organic acids. **Admira Bond** should not be used in case of known hypersensitivities (allergies) to any of these ingredients. Avoid contact with the oral mucosa. In case of contact of **Admira Bond** with the eyes, rinse immediately with plenty of water and seek medical advice.
- Materials containing thymol, eugenol or oil of cloves interfere with the curing of restorative composites and thus should not be used in combination with composite fillings.

Storage:

Tightly close **Admira Bond** bottle immediately after use. Store at temperatures of 4 °C to 23 °C (storage at 5 °C - 8 °C prolongs the shelflife of the product). Do not expose **Admira Bond** to direct sunlight or the chair light. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 BB2475 E1 0117 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VOCO GmbH