



Expert information

CALCIPRO

Manufacturer

lege artis Pharma GmbH + Co. KG
P. O. Box 60, D-72132 Dettenhausen
Breitwasenring 1, D-72135 Dettenhausen
Telephone +49 (0) 71 57 / 56 45 - 0
Fax +49 (0) 71 57 / 56 45 50
E-Mail: info@legeartis.de
www.legeartis.de

CE 0124

Name

CALCIPRO

Powder for the preparation of a calcium hydroxide paste, visible by X-ray, in any desired consistency
For dental use only

Composition

90 % calcium hydroxide, barium sulfate

Indications

CALCIPRO, mixed to a smooth paste with purified water immediately prior to use, for direct dental pulp capping as well as temporary root-canal filling.

Contra-indications

There are no contra-indications in respect of the use of CALCIPRO for direct capping, provided that the pulp has been diagnosed as being able to be preserved. When used correctly as a temporary root-canal filling, there are no contra-indications.

Side effects

When used correctly, none known.

Notice

When being used as a temporary root-canal filling, it must be ensured that CALCIPRO is not applied beyond the apex.

Calcium hydroxide has an irritant effect on the skin and mucosae. Contact with the eyes must be avoided. After accidental eye contact, remove the particles mechanically, if possible, then rinse immediately with copious amounts of water.

Mode of application

Immediately prior to being used, the user can prepare the calcium hydroxide paste quickly and easily to the desired consistency by placing a few drops of purified water onto a glass plate or mixing block and then gradually adding CALCIPRO powder, which is then mixed with the water using a spatula, until the desired consistency is obtained.

The X-ray visibility of CALCIPRO facilitates quality control and increases the safety of the product when used.

Shelf life

CALCIPRO should not be used after the expiry date.

Storage instructions

CALCIPRO powder must always be kept well closed. When removing the powder from the storage container, always use a dry instrument such as a mixing spatula.

Administrative form and package size

10 g of powder Item number 0032318

Date of revision

2018-06

Symbol of "Expiry date"



Symbol of "Batch number"



Symbol of "Pay attention to the directions for use"



Symbol of "Manufacturer"



Symbol of "Catalogue number"

Shows Item number of the manufacturer

