VOCO**GB****Grandio®****CE** 0482**light-curing nano-hybrid filling material
in accordance with EN 24049/ISO 4049****Instructions for use**

Grandio® is a light-curing filling material combining proven composite and innovative nano technology. **Grandio®** is radiopaque, contains 87 % w/w inorganic fillers (Δ 71,4 Vol. %) in a methacrylat matrix (Bis-GMA, TEGDMA) and cures under halogen or LED lights (blue light).

Grandio® is used with a dentine/enamel bond.

Grandio® is available in practical syringes and Caps for direct application.

Shades:
A1, A2, A3, A3.5, A4, B1, B2, B3, C2, C3, D2, D3, I, OA2, OA3.5, BL

Directions:

- class I to V fillings
- reconstruction of traumatically affected anteriors
- faceting of discoloured anteriors
- correction of shape and shade for better aesthetic appearance
- locking, splinting of loose anteriors
- repairing veneers
- filling deciduous teeth
- core-build-up under crowns
- composite inlays

Method of use:**Preparation/shade selection:**

Clean the teeth to be treated with a fluoride-free cleaning paste. Mark occlusal contact points (posterior area); a minimal separation facilitates the design of the approximal contact and matrix placing. Before anaesthesia, moisten the **Grandio®** shade guide and match the shade to the moist, clean tooth in daylight.

Drying:

Ensure that the work field is dry. Using a rubber dam is recommended.

Cavity preparation:

In principal, cavity preparation should be carried out according to the rules of adhesive filling therapy, i.e. minimally invasive to conserve healthy tooth substance. Bevel the enamel margins on anteriors and round off the preparation margins on posteriors. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here. Clean and dry the cavity afterwards.

Lining:

A calcium hydroxide lining should be applied in the proximity of the pulp. Apply a layer of stable cement over the area. Due to their fluoride release, glass polyalkenoate (glass ionomer) cements are advantageous.

Bond material:

Grandio® is used in adhesive technique with a dentine/enamel bond. All light-curing bonding materials can be used. Follow the respective instructions for use with regard to preparation (etch technique) and application. Using **Futurabond NR** in the self-etch-technique or **Solobond M** in total-etch-technique is recommended. Follow **Futurabond NR/Solobond M** instructions for use.

Application of Grandio®:

Let the material reach room temperature before application.

Place a matrix on cavities in the approximal area. Using translucent matrices is advantageous (smooth surfaces). Apply the chosen shade of **Grandio®** (see shade selection) and adapt it with a suitable instrument. Apply and polymerize fillings of more than 2 mm in layers.

Light-curing:

Conventional polymerization devices are suited for light-curing this material. The light output should be a minimum of 500 mW/cm² on halogen polymerization devices and 300 mW/cm² on LED devices. The curing time is a minimum of 20 s per layer, for opaque shades OA2 and OA3.5 a minimum of 40 s. Hold the light emission tip of the device as close as possible to the surface of the filling. If the distance is more than 5 mm, the curing depth may be reduced. Incomplete curing may lead to discoloration and pulpitis-like complaints.

Finishing:

The filling can be finished and polished immediately after removing the shaping aids (e.g. fine or extra fine diamond polisher, polishing disc), with cooling provided. As a final step, the margin of the filling or the entire tooth should be fluoridated.

Grandio® can be used for direct and indirect inlays according to customary methods. The physical stability can be improved by the usual external (additional) curing.

Information/precautionary measures:

- No known side effects. However, sensitivity in hypersensitive patients cannot be excluded.
- **Grandio®** should not be used if the prescribed application technique cannot be used or if patients have known allergies to its ingredients (Bis-GMA, BHT, TEGDMA).
- Phenolic substances, especially eugenol- or thymol-containing preparations, interfere with the curing of filling composites. The use of zinc oxide eugenol cements or other eugenol-containing materials with filling composites must be avoided.
- Fillings exposed to occlusal forces should be checked at least once a year to detect any changes over time.

Storage:

Store at temperatures between 4°C - 23°C. If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and possible polymerization, syringes should be closed immediately after dispensing. Do not use **Grandio®** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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