voco

Grandio® disc

Instructions for use

Product description:

Grandio disc is a nano-hybrid composite CAD/CAM disc for the fabrication of permanent single-tooth restorations.
Grandio disc contains 86 % w/w inorganic fillers in a polymer matrix. The restorations produced must be placed with an adhesive composite luting agent in combination with a dentine-enamel bonding agent. **Grandio disc** is available in one size (\varnothing 98.4 x 15 mm).

- **Grandio disc** contains methacrylates. **Grandio disc** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Storage:

Store at temperatures of 4 °C to 23 °C. If the material is stored in the refrigerator, allow it to reach room temperature before use. Do not store the product in a place where it is exposed to direct artificial light or sunlight. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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- Crowns, inlays, onlays, veneers
- Implant-supported crowns

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- Präparationswinkel 4° - 6°
- Minimum okklusal wall thickness 1.5 mm
- Minimum bukkal wall thickness 0.8 mm
- Minimum wall thickness below load-bearing cusp 1.5 mm
- Minimum cervical wall thickness 0.8 mm

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- Minimum cervical wall thickness 0.4 mm
- Minimum labial wall thickness 0.5 mm
- Minimum incisal wall thickness 0.5 mm

Cement space:

70 µm (\pm 10 µm)

Milling process:

–

Select the correct disc size for the planned restoration as well as the milling parameters for **Grandio disc** or for composites. When doing so, pay attention to the software settings of the respective CAD/CAM systems. **Processing requires diamond instruments**. First of all, the settings for disc sizes and the milling parameters need to be created, if they are not already included in the software settings of the CAD/CAM systems. Please contact the CAD/CAM system provider for assistance. The operating instructions from the manufacturers must be observed and followed.

Check the processed restoration for defects such as fissures or material chipping. If the restoration displays any defects, it must be rejected.

Note: Grandio disc is an already completely cured composite and must not be fired.

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- Mindestwandstärke zervikal 0,4 mm
- Mindestwandstärke labial 0,5 mm
- Mindestwandstärke inzisal 0,5 mm

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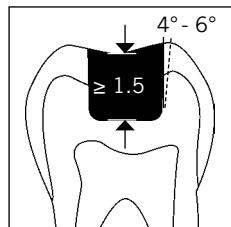
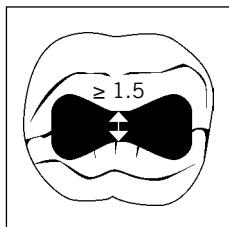
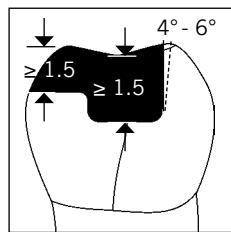
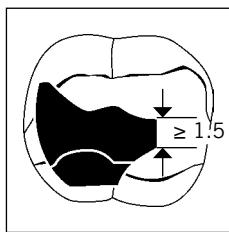
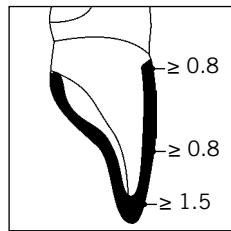
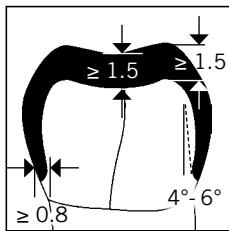
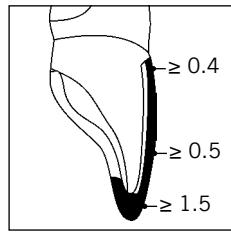
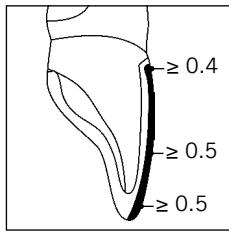
Modifications:

Grandio disc is a nano-hybrid composite CAD/CAM restorative material which can be modified, characterised or repaired at any time using a composite/ORMOCER®. Roughen the surface of the restoration (for example with a diamond bur) and apply a suitable adhesive system (e.g., **Futurabond U**) in accordance with the instructions for use. Then apply the composite/ORMOCER® (e.g., **GrandioSO**, **GrandioSO Flow**, **FinalTouch**) in accordance with the respective instructions for use.

Information, precautionary measures:

- No known side effects. However, the sensitisation of hypersensitive patients cannot be ruled out.

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VC 60 AA6051 E1 0719 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VITA® is a registered trademark of VITA Zahnfabrik H. Rauter GmbH & Co.
VITA® is a registered trademark of VITA Zahntfabrik H. Rauter GmbH & Co.
VOCO GmbH