# greenbite colour

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# greenbite colour

**GB**  
Instructions for use

VPS bite registration material, fast setting – extreme high final hardness - thixotropic - with colour change

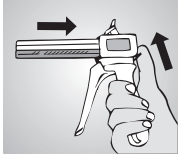
Fig. 1

## 1. Mixing and Dosing

Extrude material from the cartridge with Automix2-system gun (Fig. 1). Place cartridge into mixing gun and remove closure cap by rotation. Eject a small amount of material (bleeding) until equal flow is being extruded from both orifices (Fig. 2). Mount the mixing cannula by using the guides on the cannula and the cartridge and lock in place by rotating in the opposite direction (Fig. 3). Feed material applying even pressure. Prior to use, dispense a small amount of material and visually check that the base and catalyst paste are mixed omogeneously (Fig. 4). Only then dosing can be performed individually. Store cartridge with mixing cannula attached for subsequent use.

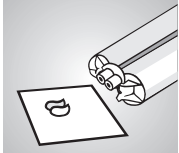
Fig. 2

## 2. Instruction for use

Extrude material from cartridge with mixing cannula or with attached contouring tip onto teeth (Fig. 5). At terminal occlusion application from vestibular is possible. Let registration harden in terminal occlusion. While curing in the mouth the colour of the occlusal record changes from green at the beginning to yellow. This colour change serves as an additional indicator of the degree of setting. The occlusal record is removed after an intraoral setting time of approx. 50 sec. and after completion of the colour change.

Fig. 3

## 3. Disinfection

After removal from the mouth rinse registrate under lukewarm water. Disinfection for 15 min. in 2 % glutaraldehyde or other usual solutions.

## Important working hints

- Remove any retraction astringents or mouth wash solutions prior to the registration – could affect setting reaction.
- Do not intermix with condensation curing silicones.
- Avoid staining of clothing. Hardened silicone materials cannot be removed.
- Remove all residues of registration material from the patient's mouth.
- Please follow the instructions of the safety data sheet!

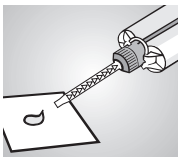
Fig. 4

Eventually separated silicone oil (blue) during storage of the product does not have any effect on the working process. The setting reaction is not influenced from that.

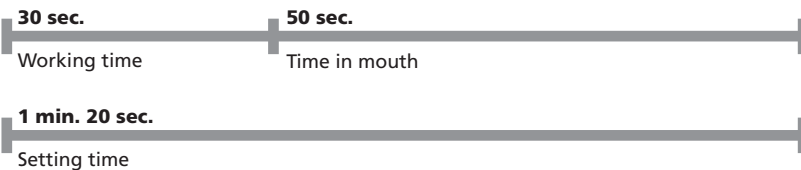
Fig. 5

Further information:

Silicone based materials are proven a million times. On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

Only for dental use by qualified personnel.

## Indications

for use:

- Bite registration with crown and bridge impressions and for orthodontic indications, key for intraoral and gnathological registrations

## Technical Data:

- DIN 13903
- **Mixed volume:** 50 ml (cartridge)
- **Mixing ratio:** 1:1
- **Colour code:** base: blue catalyst: yellow
- **Mixing time:** omitted (Automix2- system)
- **Working time:** approx. 30 sec.\*
- **Setting time:** approx. 1 min. 20 sec.\*
- **Time in mouth:** approx. 50 sec.
- **Final hardness:** approx. 45 Shore D
- **Linear dimensional change:** < 0.2 %
- **Application:** At 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5% rel. humidity
- **Storage:**

\* from beginning of mixing at 23 °C ± 2 °C / 73 °F ± 4 °F, 50 ± 5 % rel. humidity. Increased temperatures accelerate, decreased temperatures retard a. m. times.

## Ordering information:

- **greenbite colour Standard packing 02207**  
2 x 50 ml cartridges  
8 mixing cannulas, green  
8 contouring tips for mixing cannulas green
- **Ecopacking 02726**  
8 x 50 ml cartridges  
32 mixing cannulas, green  
32 contouring tips for mixing cannulas green
- **Mixing cannulas green, 48 pcs. 02705**
- **Contouring tips for mixing cannulas green 12 pcs. 02648**

**DETAX**  
GmbH & Co. KG  
Postfach 100225 · 76256 Ettlingen / Germany  
Telefon 07243/510-0 · Fax 07243/510-100  
www.detax.de · post@detax.de

Caution: Federal U.S. law restricts this device to sale by or on the order of a dentist (or trained specialist personell).

VM05

Made in Germany

01/2018

# greenbite colour

*VPS bite registration material, fast setting – extreme high final hardness - thixotropic - with colour change*

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