i BOND[®] Ceramic Primer

KULZER
MITSUI CHEMICALS GROUP

DE	Gebrauchsanweisung	3
GB	Instructions for use	7
FR	Mode d'emploi	11
ES	Instrucciones de uso	15
IT	Istruzioni per l'uso	19
PT	Instruções de uso.....	23
NL	Gebruiksaanwijzing	27
SE	Bruksanvisning	31
DK	Brugervejledning	35
NO	Bruksanvisning	39
FI	Käyttöohjeet	43

Produktbeschreibung

Haftvermittler für Silikatkeramik

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Primer for silica based ceramics

Composition

iBOND Ceramic Primer is a isopropanol/acetone based solution of methacrylate monomers and silane.

Indications

Surface conditioning of silicate/glass ceramic.

- For the fixation of indirect restorations with luting composites.
- For intraoral repair of ceramic veneerings, as well as full ceramic restorations.

Contraindications

- The use of this product is contraindicated in cases of known or suspected allergies to (meth)acrylate compounds.
- If a secure isolation cannot be guaranteed when being used intraorally.

Side effects

This product or one of its components may, in particular cases, cause hypersensitive reactions. If suspected, information about the ingredients can be obtained from the manufacturer. In the case of a suspected allergy, we recommend taking an allergy test prior to use.

Warning indications

- The product contains acetone and is highly flammable.
- Avoid static charging. Keep away from all sources of ignition.
- Use only in sufficiently well-ventilated rooms. Do not inhale vapours.
- The product may irritate eyes.
- Avoid contact with eyes. Immediately and thoroughly rinse with plenty of water in case of contact with eyes and contact an ophthalmologist.
- Avoid direct contact with the mucosa. On contact with the skin, clean immediately with soap and water.

Precautions

- Wear appropriate protective clothing, eye protection, a mask and gloves. Eye protection is recommended for the patient.
- Close iBOND Ceramic Primer bottle carefully after each use.
- Do not reuse bottles after contamination with bodily fluids in order to prevent cross-contamination.
- Saliva or blood in the field of application during treatment may lead to failure. Use rubber dam (e.g. Ivory) or other suitable means of isolation.

Special instructions

Only to be used by dentists and for its intended use.

Storage advice

- Do not expose to direct sunlight. Store in a sufficiently well ventilated place. Do not store above room temperature 25°C (77°F). After first opening the storage in the refrigerator 8°C (46°F) is recommended.
- Do not use after the expiry date. Expiry date and batch number: see note on packaging. Please quote the batch number and expiry date in all correspondence about the product.
- Expiry date Σ and batch number \square : See note on bottle / packaging.

Disposal information

Dispose of in accordance with local regulations.

Application steps

Fixation of indirect restorations

- Preparation of the material surfaces

Silicate / glass ceramic:

Caution: Ceramic materials require different surface treatments; ask the manufacturer or laboratory about the correct procedure.

Always apply iBOND Ceramic Primer after pre-treating the ceramic surfaces.

- Measure out the desired quantity of drops of iBOND Ceramic Primer into the mixing well.
- Use the applicator tip or brush to apply to the whole surface of the material without delay and allow to dry for 20 seconds.
- Then dry briefly with an oil-free air flow.

Provide and apply iBOND Universal immediately as indicated in the correspondent instructions for use.

Intraoral repair

- **Preparation of the material surfaces**

Silicate / glass ceramic:

Use rubber dam (e.g. Ivory) or other suitable means of isolation. Roughen the surface to be repaired with a diamond bur. Remove the ceramic glaze approx. 1 mm beyond the margin of the fracture site. Rinse and dry with oil-free air.

Apply iBOND Ceramic Primer as described previously.

Provide and apply iBOND Universal immediately as indicated in the correspondent instructions for use.

Information which deviates from these directions for use is invalid without the authorisation of Kulzer.

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Kulzer GmbH
Kulzer, LLC