



Expert information

## **LEGASED natur**

CE

### **Manufacturer**

**lege artis** Pharma GmbH + Co. KG  
P. O. Box 60, D-72132 Dettenhausen  
Breitwasenring 1, D-72135 Dettenhausen  
Telephone +49 (0) 71 57 / 56 45 - 0  
Fax +49 (0) 71 57 / 56 45 50  
E-Mail: [info@legeartis.de](mailto:info@legeartis.de)  
[www.legeartis.de](http://www.legeartis.de)

### **Name**

LEGASED natur  
Oral wound dressing

### **Composition**

Natural resins (myrrh, benzoin, colophony), ethanol, modified cellulose and macrogol

### **Indications**

Viscous, film-forming fluid applied as a protective coating in cases of mild inflammation in the oral cavity, sore ulcer and aphthous stomatitis. Offers protection against mechanical pressure and chemical irritation, thus relieving existing pain.

### **Contra-indications**

Known hypersensitivity to natural resins, particularly colophony, or to any of the other ingredients.  
Not for large, open wounds (e.g. Alveoli).

### **Side effects**

When used correctly, side effects are not known.

### **Warnings**

Contains approx. 23 % alcohol.

In case of eye contact, rinse eyes with plenty of water. Call a physician if necessary.

Remove any spilled or splashed solution as quickly as possible with alcohol or another suitable solvent (attention: do not use water or water-based cleaning agents, as this leads to immediate precipitation), as the natural resins contained in the solution will otherwise become sticky and can leave stains behind.

### **Mode of application**

#### At the dental surgery:

Dab dry the area to be treated and keep as dry as possible to ensure the fluid adheres well. Then apply the solution, using a suitable instrument (e.g. Apply-Tip), to the dry mucous membrane (do not rub in).

If necessary, the solution can also be filled into a disposable syringe and applied to specific areas using a blunt and not too narrow cannula.

#### At home and away on holiday:

If possible, dab dry the area to be treated to enable the protective coating to adhere better (e.g. with a paper tissue). Then quickly apply a generous amount of the solution using a cotton-wool bud or, if necessary, clean fingertip (do not rub in). To speed up the film-forming process, the mouth may be briefly rinsed out with water. The solution can be applied several times daily, particularly after meals and before going to bed.

